


# S1263 One Piece Wooden Footboard and Hardware

P93, P93E, P93M, PW93, PW93E

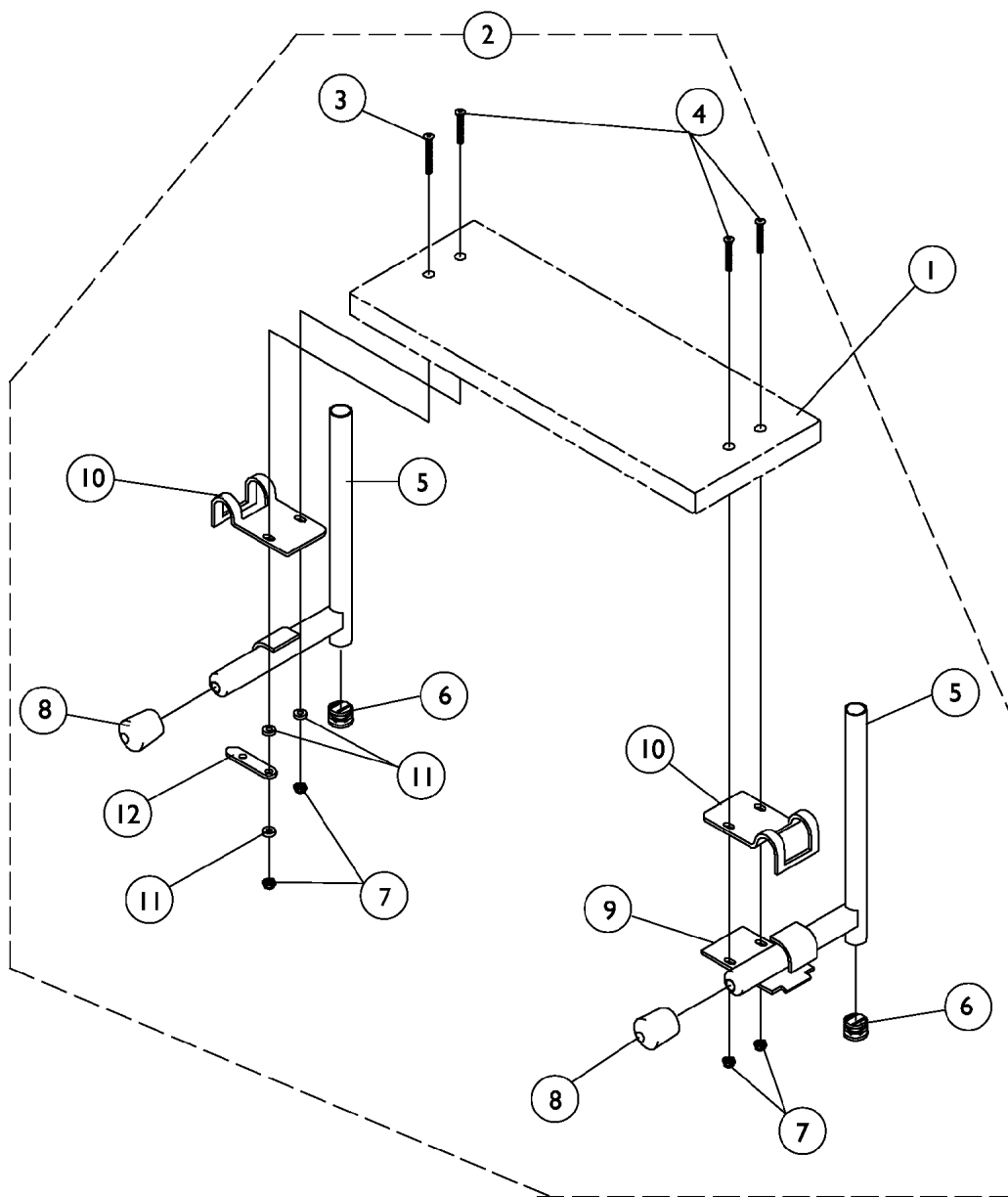

# S1263 One Piece Wooden Footboard and Hardware

## P93, P93E, P93M, PW93, PW93E

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 One Piece Wooden Footboard without Hardware	1	1
	A	See Chart	Heel Loop Hardware Packages (not shown)	1	1
2	B	1032013-NLA	NLA - Kit, Wooden Footboard Mounting Hardware	1	1
3			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/2")	1	1
4			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	1	3
5			NLA - Tube, Slide - Right	1	1
5			NLA - Tube, Slide - Left	1	1
6			Plug Button, Black (3/4")	1	2
7			Locknut (#10-32)	1	4
8			Cap, End, Tube, Black (7/8")	1	2
9			Lock, Pad Plate, Legrest, Black	1	1
10			Clamp, Black	1	2
11			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
12			NLA - Link, Locking, Chrome	1	1
2	C	1054573	Kit, Wooden Footboard Mounting Hardware	1	1
3			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/2")	1	1
4			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	1	3
5			Tube, Pivot/Slide, Chrome - Right (9-1/2" L)	1	1
5			Tube, Pivot/Slide, Chrome - Left (9-1/2" L)	1	1
6			Plug Button, Black (3/4")	1	2
7			Locknut (#10-32)	1	4
8			Cap, End, Tube, Black (7/8")	1	2
9			Lock, Pad Plate, Legrest, Black	1	1
10			Clamp, Black	1	2
11			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
12			NLA - Link, Locking, Chrome	1	1

NOTE: A - Not Shown.  
 B - Includes items 3-12 and used for P93, P93E, P93M and P904  
 C - Includes items 3-12 and used for PW93 and PW93E only.

### #1 One Piece Wooden Footboard without Hardware

Footrest-Footboard w/ Heel Loop Holes Part Number	Footrest-Footboard w/o Heel Loop Holes Part Number	Chair Width	Footboard Width	Footboard Depth
---	--	-------------	-----------------	-----------------

# S1263 One Piece Wooden Footboard and Hardware

## P93, P93E, P93M, PW93, PW93E

### #1 One Piece Wooden Footboard without Hardware

Footrest- Footboard w/ Heel Loop Holes Part Number	Footrest- Footboard w/o Heel Loop Holes Part Number	Chair Width	Footboard Width	Footboard Depth
1034245-NLA	48247X003-NLA	14"	11-5/8"	6"
1034247-NLA	48247X001-NLA	16"	13-3/4"	6"
1034249-NLA	48247X000-NLA	18"	15-9/16"	6"
1034251-NLA	1031913-NLA	20"	17-1/2"	6"

# S1263 One Piece Wooden Footboard and Hardware

## P93, P93E, P93M, PW93, PW93E

### Heel Loop Hardware Packages (not shown)

(All hardware is included to attach heel loops to footboard.)

Chair Width	Part Number	Package Includes
14-15" Wide	1033904-NLA	2 each Heel Loops 4 each Screw, PPH (1/4-20 x 3") 4 each Spacer (1/4 x 11/32 x 1-3/4") 4 each Locknut (1/4-20)
16"-24" Wide	1033905-NLA	2 each Heel Loops 4 each Screw, PPH (1/4-20 x 3") 4 each Spacer (1/4 x 11/32 x 1-3/4") 4 each Locknut (1/4-20)