


S1263 One Piece Wooden Footboard and Hardware

PW93, PW93E

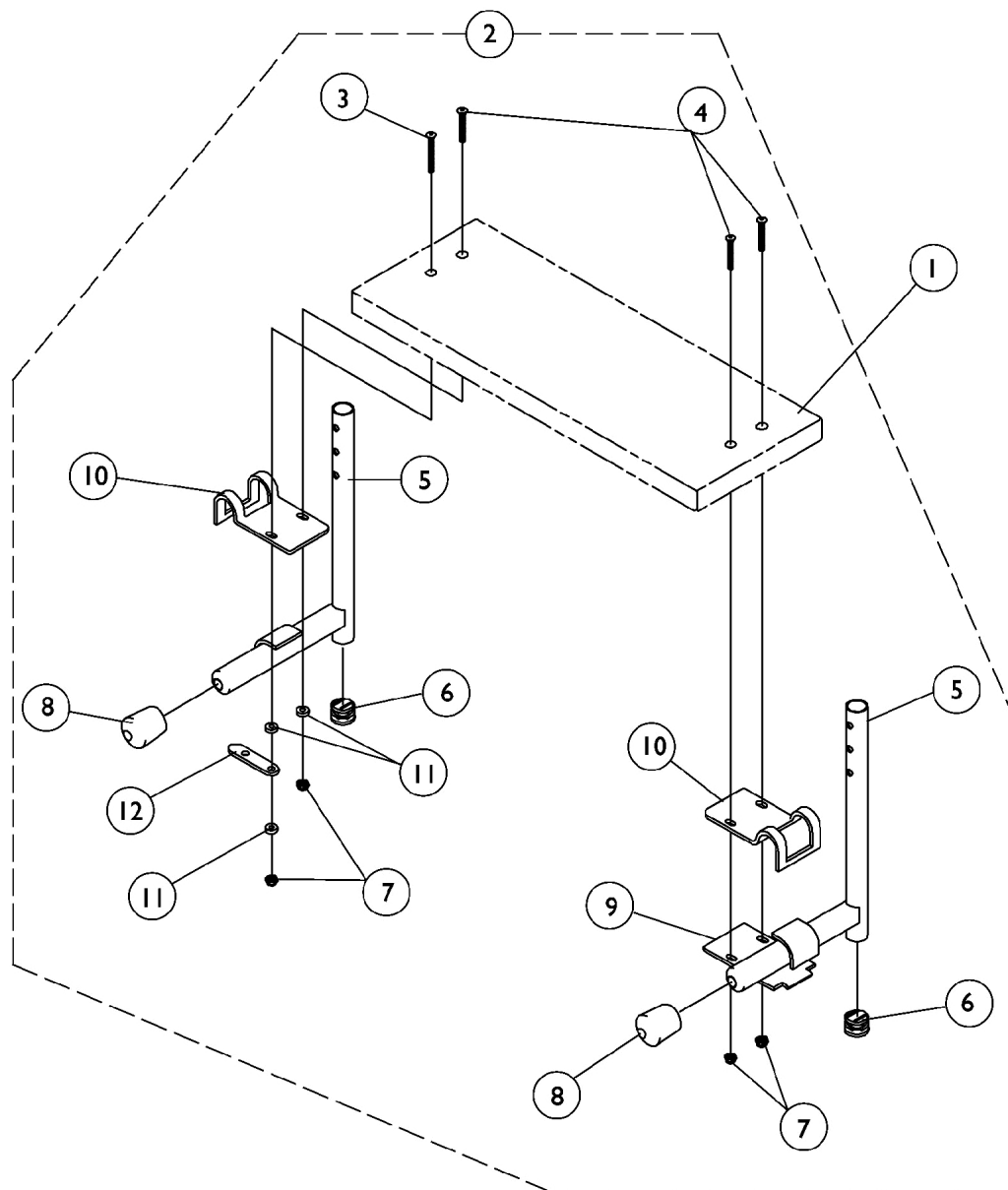

S1263 One Piece Wooden Footboard and Hardware

PW93, PW93E

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 One Piece Wooden Footboard without Hardware	1	1
		See Chart	Heel Loop Hardware Packages (not shown)	1	1
2	B	1054573	Kit, Wooden Footboard Mounting Hardware	1	1
3			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/2")	1	1
4			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	1	3
5			Tube, Pivot/Slide, Chrome - Right (9-1/2" L)	1	1
5			Tube, Pivot/Slide, Chrome - Left (9-1/2" L)	1	1
6			Plug Button, Black (3/4")	1	2
7			Locknut (#10-32)	1	4
8			Cap, End, Tube, Black (7/8")	1	2
9			Lock, Pad Plate, Legrest, Black	1	1
10			Clamp, Black	1	2
11			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
12			NLA - Link, Locking, Chrome	1	1

NOTE: A - Not Shown.
B - Includes items 3-12

#1 One Piece Wooden Footboard without Hardware

Footrest- Footboard w/ Heel Loop Holes Part Number	Chair Width	Footboard Width	Footboard Depth
1034245-NLA	14"	11-5/8"	6"
1034247-NLA	16"	13-3/4"	6"
1034249-NLA	18"	15-9/16"	6"
1034251-NLA	20"	17-1/2"	6"

S1263 One Piece Wooden Footboard and Hardware

PW93, PW93E

Heel Loop Hardware Packages (not shown)

(All hardware is included to attach heel loops to footboard.)

Chair Width	Part Number	Package Includes
14-15" Wide	1033904-NLA	2 each Heel Loops 4 each Screw, PPH (1/4-20 x 3") 4 each Spacer (1/4 x 11/32 x 1-3/4") 4 each Locknut (1/4-20)
16"-20" Wide	1033905-NLA	2 each Heel Loops 4 each Screw, PPH (1/4-20 x 3") 4 each Spacer (1/4 x 11/32 x 1-3/4") 4 each Locknut (1/4-20)