


ASBA Seat Frame, Seat Pan, Adapter, & Arm Socket Hardware Before 8/08/05

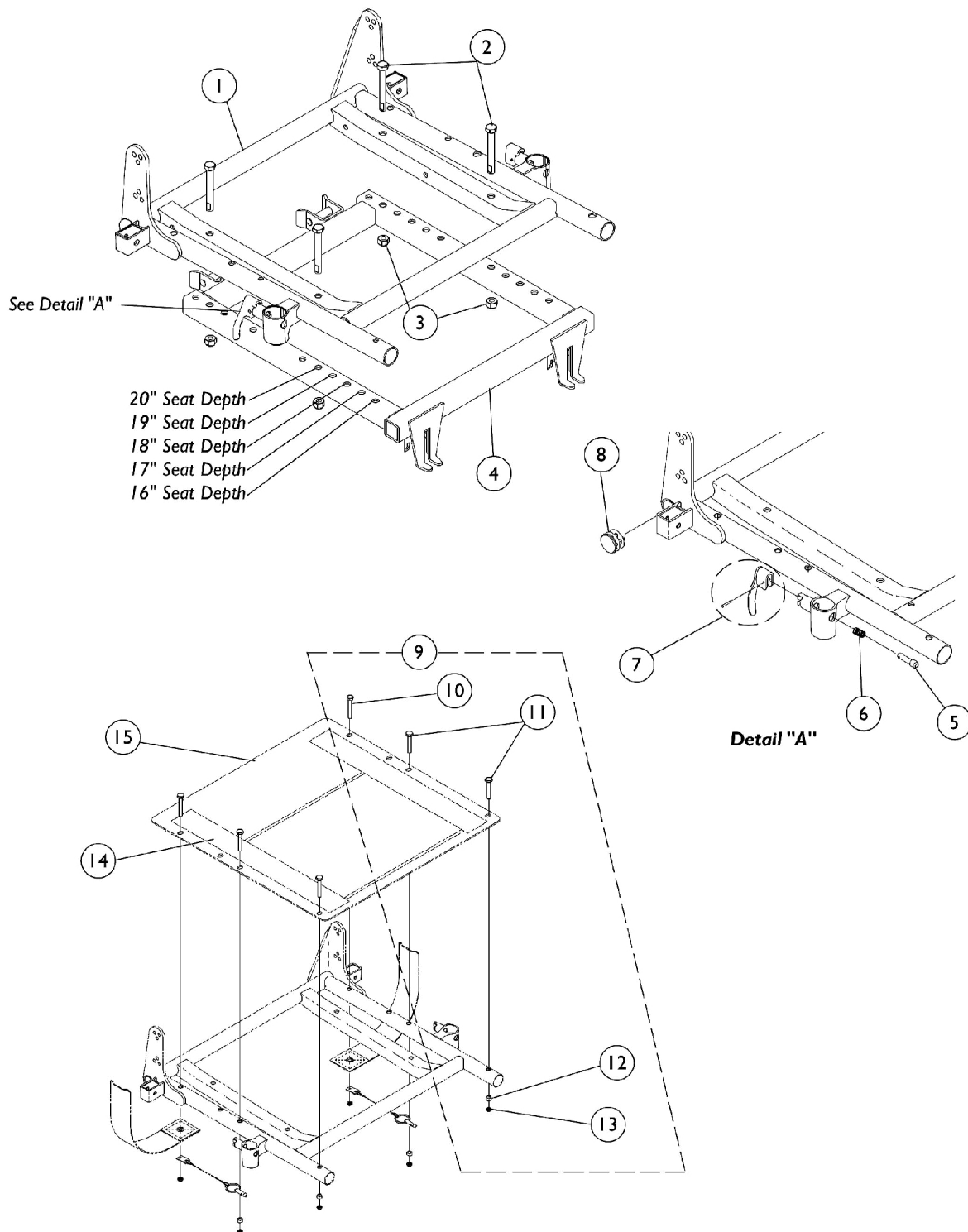

ASBA Seat Frame, Seat Pan, Adapter, & Arm Socket Hardware Before 8/08/05

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Seat Frames	1	1
2		1043628-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 2-1/2")	1	4
3		1026158	Locknut (5/16-18)	1	4
4		1106480	Adapter, Seat Frame	1	1
5		1028594	Pin, Arm Lock, Black	1	2
6		47471X074	Spring, Arm Lock	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		1029501	Plug Button, Black (1")	1	2
9	B	1070459-NLA	NLA - Kit, Seat Pan Mounting Hardware	1	1
10			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
11			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
12			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
13			Locknut (1/4-20)	1	6
14		1039949-NLA	NLA - Fastener, Hook Adhesive Back (2" W x 16" L)	1	2
14		1044578-NLA	NLA - Fastener, Loop Adhesive Back (2" W x 16" L)	1	2
15		See Chart	#15 Seat Pans	1	1
NOTE: A - Includes items 5-8 B - Not Shown. Includes items 10-13					

#1 Seat Frames

Chair Depth	16" Wide	17" Wide	18" Wide	19" Wide	20" Wide
16-17" Deep	1070672-NLA	1070673-NLA	1070674-NLA	1070675-NLA	1070676-NLA
18-19" Deep	1070681-NLA	1070682-NLA	1070683-NLA	1070684-NLA	1070685-NLA
20" Deep	1070690-NLA	1070691-NLA	1070692-NLA	1070693-NLA	1070694-NLA

#15 Seat Pans

Chair Depth	16" Wide	17" Wide	18" Wide	19" Wide	20" Wide
16" Deep	1066013-NLA	1066014-NLA	1066015-NLA	1066016-NLA	1066017-NLA
17" Deep	1066024	1066025-NLA	1066026	1066027-NLA	1066028-NLA
18" Deep	1066035-NLA	1066036-NLA	1066037-NLA	1066038-NLA	1066039-NLA
19" Deep	1066046-NLA	1066047-NLA	1066048-NLA	1066049-NLA	1066050-NLA
20" Deep	1066057-NLA	1066058-NLA	1066059-NLA	1066060	1066061-NLA