

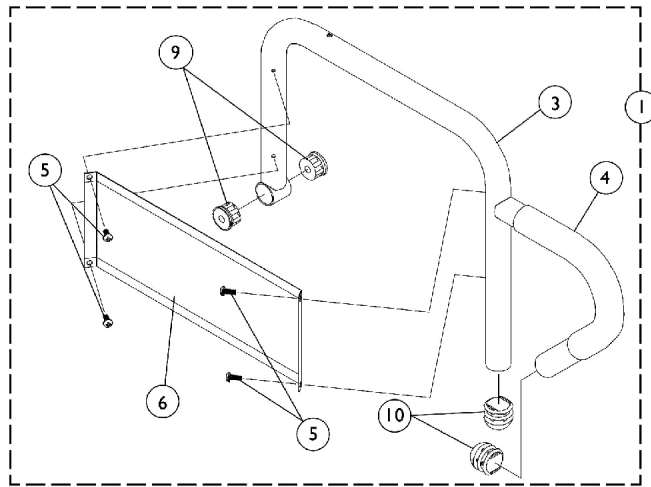



Arms - Fixed Height

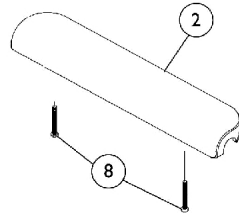

Desk Length Fixed Height Arm

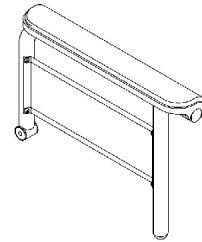

**Full Length
Fixed Height Arm**

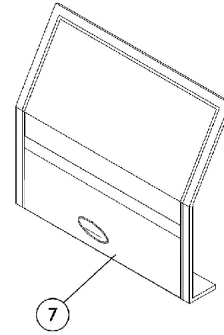

7

Arms - Fixed Height

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Fixed Height Arms w/o Pad	1	1
2	A,B	1102585-NLA	NLA - Pad, Arm, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
2		1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
2	A,C	1102586-NLA	NLA - Pad, Arm, Desk Length, Urethane Waterfall, Black	1	1
2		1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
3		See Chart	#3 Arm Tube - Fixed Height	1	1
4		1032244	Grip, Foam Hand (9" L)	1	1
5		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
6		1035472-NLA	NLA - Guard, Clothing, Desk Length	1	1
6		1044572-NLA	NLA - Guard, Clothing, Full Length (16 - 19" D)	1	1
6		1044005-NLA	NLA - Guard, Clothing, Full Length (20" and Deeper)	1	1
7		1035065	Cover, Clothing Guard w/ Pocket, FXDESK & ADJFULL/ADJDESK	1	1
7		1044588	Cover, Clothing Guard with Pocket Fixed Height Full Length (20" and Deeper)	1	1
8		1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
9		1065936	Plug, Pivot	1	2
10		1067734	Plug Button, Black (5/8")	1	2

NOTE: A - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs with screws for chairs built before 1/3/02
 B - No longer available, use 1110818 as a replacement.
 C - No longer available, use 1110817 as a replacement.

#1 Fixed Height Arms w/o Pad

Chair Seat Depth	Arm Style	Part Number
16-19"	Desk -Right	1068566-NLA
16-19"	Desk -Left	1068567-NLA
16-19"	Full -Right	1068559-NLA
16-19"	Full -Left	1068560-NLA
20"	Desk -Right	1068540-NLA
20"	Desk -Left	1068541-NLA
20"	Full -Right	1068528-NLA
20"	Full -Left	1068529-NLA

Arms - Fixed Height

#3 Arm Tubes - Fixed Height

Chair Seat Depth	Arm Style	Part Number
16-19"	Desk -Right	1068564
16-19"	Desk -Left	1068565-NLA
16-19"	Full -Right	1068490
16-19"	Full -Left	1068491-NLA
20"	Desk -Right	1068414-NLA
20"	Desk -Left	1068415-NLA
20"	Full -Right	1068473-NLA
20"	Full -Left	1068474-NLA