


# Front Rigging Hanger Assembly


## ASBA (Before 8/08/05) and Van Seats

NOTE: Seat pan and Front rigging hanger assembly share the same hardware on ASBA seats. See ASBA seat pan and hardware page.


NOTE: Van seat front rigging mounting holes changed from a vertical to horizontal position 5/16/02.

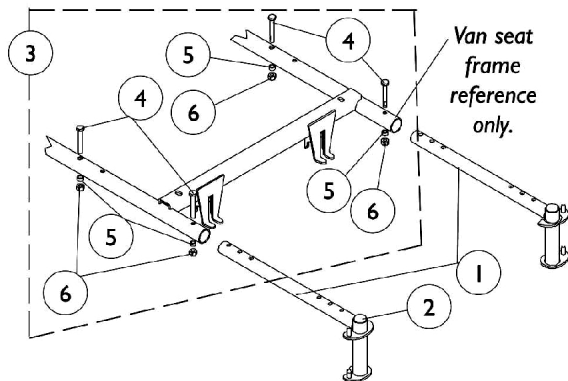


**Front Rigging Hanger  
(A.S.B.A. Seat Before 9/1/02)  
and (Van Seat Before 5/16/02)**

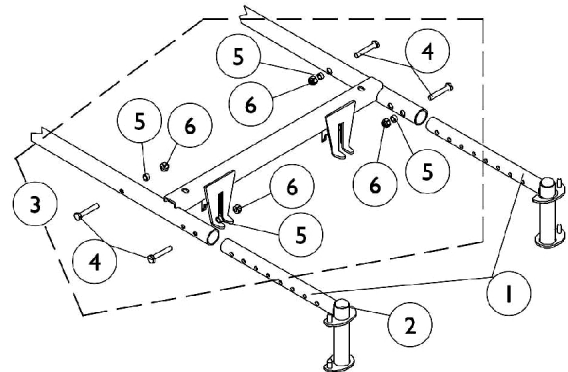


**Front Rigging Hanger  
5/16/02 Thru 8/31/02  
Van Seat**

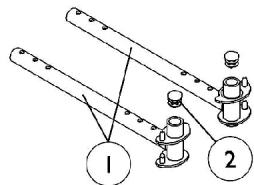

**Front (Hemi) Rigging Hanger  
A.S.B.A. Seat 9/1/02-1/5/03**

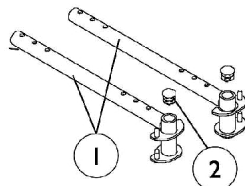

**Front (Hemi) Rigging Hanger  
A.S.B.A. Seat 1/6/03-8/07/05**

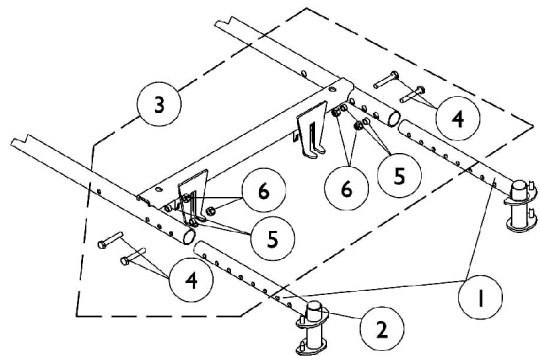

**Front (Hemi) Rigging Hanger  
Van Seat  
After 8/31/02**

# Front Rigging Hanger Assembly

## ASBA (Before 8/08/05) and Van Seats

**NOTE:** Seat pan and Front rigging hanger assembly share the same hardware on ASBA seats. See ASBA seat pan and hardware page.

**NOTE:** Van seat front rigging mounting holes changed from a vertical to horizontal position 5/16/02.

| Item Number | Note | Part Number | Description                                                                               | Quantity Per |          |
|-------------|------|-------------|-------------------------------------------------------------------------------------------|--------------|----------|
|             |      |             |                                                                                           | Package      | Assembly |
| 1           | A    | 1109009-NLA | NLA - Front Rigging Hanger Assembly w/ Plug Button, Adult, Black - Right                  | 1            | 1        |
| 1           | A    | 1109010     | Front Rigging Hanger Assembly w/ Plug Button, Adult, Black - Left                         | 1            | 1        |
| 1           | B    | 1111690-NLA | NLA - Front Rigging Hanger Assembly w/ Plug Button, Adult, Black - Right                  | 1            | 1        |
| 1           | B    | 1111691-NLA | NLA - Front Rigging Hanger Assembly w/ Plug Button, Adult, Black - Left                   | 1            | 1        |
| 1           | C    | 1113026     | Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Right                         | 1            | 1        |
| 1           | C    | 1113027     | Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Left                          | 1            | 1        |
| 1           | G    | 1114782-NLA | NLA - Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Right                   | 1            | 1        |
| 1           | G    | 1114783-NLA | NLA - Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Left                    | 1            | 1        |
| 1           | H    | 1114766     | Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Right (12 - 21" D) | 1            | 1        |
| 1           | H    | 1114767     | Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Left (12 - 21" D)  | 1            | 1        |
| 2           |      | 1027097     | Plug Button, Black (3/4")                                                                 | 1            | 1        |
| 3           | D,E  | 1113589-NLA | NLA - Hardware, Front Rigging Hanger Attaching (Pair)                                     | 2            | 1        |
| 3           | D,F  | 1113045     | Kit, Front Rigging Hanger Hardware                                                        | 2            | 1        |
| 4           |      |             | Screw, Hex Head (1/4-20 x 1-1/2")                                                         | 1            | 4        |
| 5           |      |             | NLA - Spacer (1/4 x 3/8 x 3/16")                                                          | 1            | 4        |
| 6           |      |             | Locknut (1/4-20)                                                                          | 1            | 4        |
| 4           | E    | 1088898-NLA | NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-3/4")                                          | 1            | 4        |

**NOTE:** A - ASBA Before 9/1/02 and Van Seat Before 5/16/02

B - 5/16/02 thru 8/31/02 for Van Seat Only

C - After 8/31/02 for Van Seat

D - Includes items 4-6

E - Before 5/16/02

F - After 5/15/02

G - 9/1/02 thru 1/5/03 for ASBA seat

H - 1/5/03 thru 8/07/05 for ASBA seat