


Controller, Cables, Programmer & Hardware

MK5 NX-75, MK5 NX-80 and MK5 EX

Effective 12/19/03, MK5 NX-80 replaced MKIV RII as standard electronics on 400 Lb. limit chairs and PTO seating system chairs.

From 4/01/04 to 8/08/05 MK5 NX-80 was standard electronics on 300 Lb. limit chairs. As of 8/09/05 MK5 NX-75 is available on 300 Lb. limit chairs.

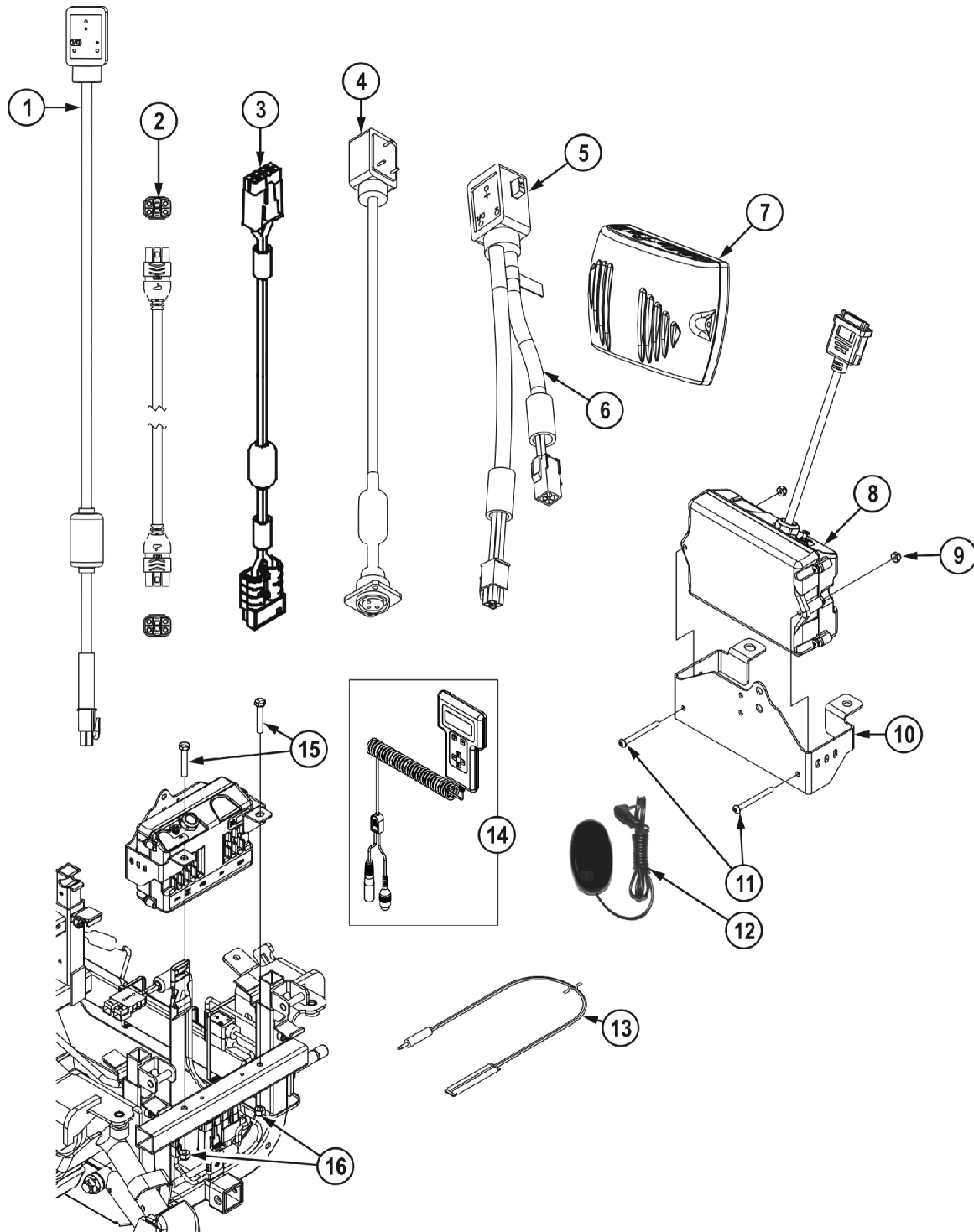

Controller, Cables, Programmer & Hardware

MK5 NX-75, MK5 NX-80 and MK5 EX

Effective 12/19/03, MK5 NX-80 replaced MKIV RII as standard electronics on 400 Lb. limit chairs and PTO seating system chairs.

From 4/01/04 to 8/08/05 MK5 NX-80 was standard electronics on 300 Lb. limit chairs. As of 8/09/05 MK5 NX-75 is available on 300 Lb. limit chairs.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1119418	Cable, Auxiliary Power	1	1
2	B	1127293	Cable, Joystick SPJ+	1	1
3		1122199	Cable, Battery Extension, Red Connector	1	1
4	A	1114500-NLA	NLA - Cable Option with Charger Connector	1	1
5		1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	1
6	C	1127429-NLA	NLA - Cable, Splitter Auxiliary Power	1	1
7	D,E	1126182-NLA	NLA - Controller, MK5 NX-75	1	1
8	F	1122191-NLA	NLA - Controller, MK5 NX-80	1	1
8	G	1115750-NLA	NLA - Controller, MK5 EX	1	2
9	H	1025731	Locknut (#10-32)	1	2
9	D	1056098	Locknut (#8-32)	1	2
10	K	1110665-NLA	NLA - Bracket, Controller Mounting, Black	1	1
	K	1112783-NLA	NLA - Bracket with Hardware, Controller, M91/M94	1	1
			NLA - Screw (#8-32 x 1/2")	1	1
			NLA - Screw, Hex Head (5/16-18 x 3/4")	1	1
			Washer, Steel (5/16 x 3/4 x 1/16")	1	1
			Locknut (5/16-18)	1	1
			NLA - Bracket, Controller Mounting, Black	1	1
10			Tie Wrap (11-1/2" L)	1	1
			NLA - Screw, Socket Button Head (10-32 x 2")	1	2
11	H	1121493-NLA	NLA - Screw, Socket Button Head (10-32 x 2")	1	2
11	D	1127352	Screw, Phillips Pan Head (#8-32 x 1")	1	2
12	I	1123276	Kit, Switch, Black Egg Reset w/ Hook/Loop Adhesive Back Fastener & Attaching Screws (EGSBLK)	1	1
13	J	1020818	Switch, Red Flex Ribbon (Tape) Reset	1	1
14		1109091-NLA	NLA - Programmer, Remote	1	1
15		1044939-NLA	NLA - Screw, Hex Head (5/16-18 x 1-3/4")	1	2
16		1026158	Locknut (5/16-18)	1	2
	K	1095204-NLA	NLA - Cable, Virtual Service, Charger/Modem	1	1

NOTE: A - These items are only required when using on board charger p/n 1123102. This would apply to Non PTO chairs built Before 11/01/04 and PTO Plus chairs built between 7/15/04 and 5/31/05.
 B - Used on NX-80 or EX 80 only.
 C - Used only when PTO Plus is present. Item 4 (fuse) is included.
 D - Used on MK5 NX-75 controller
 E - After 8/08/05
 F - Standard on 400 Lb. limit chairs and chairs w/ PTO Seating System as of 12/19/03; Standard on 300 Lb. limit chairs between 4/1/04 and 8/8/05.
 G - Available as an upgrade on all M91chairs before 8/9/05.
 H - Used on MK5 NX-80 and MK5 EX controllers
 I - Not Shown. After 5/23/04 the Egg Reset Switch replaced the Red Flex Ribbon (Tape) Reset Switch
 J - Before 5/24/04
 K - Part #1110665 is no longer available for sale individually. Order kit part #1112783 Controller Bracket with Hardware. Kit Hardware inside carton is for other Controller applications. Order 2 ea. of items 9 & 11 also depending on Controller type used.