


Pivot Tube Assemblies w/ Footplates - Adjustable Angle
Footplates - AT5543
(Before 9/12/03)

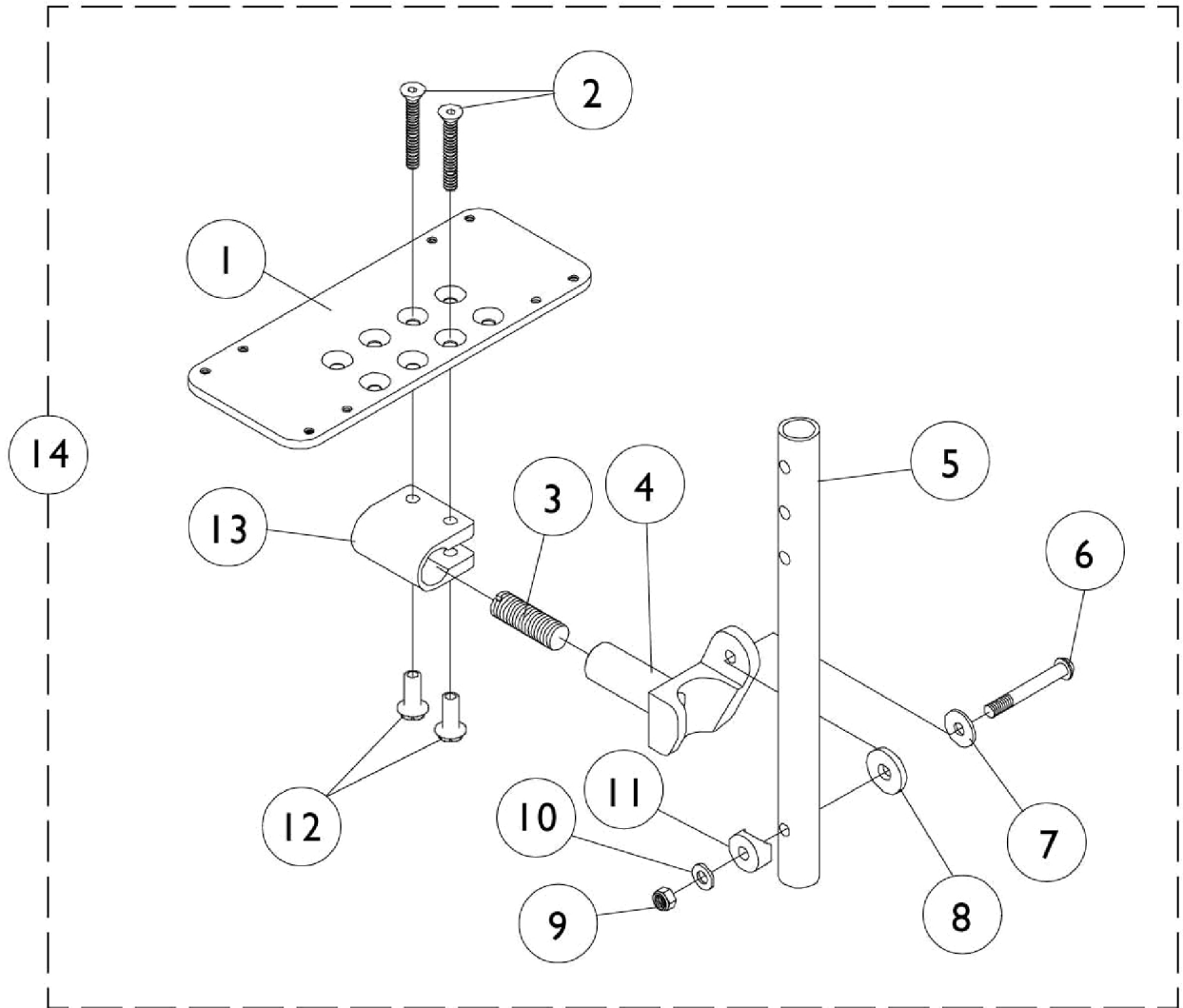

Pivot Tube Assemblies w/ Footplates - Adjustable Angle

Footplates - AT5543

(Before 9/12/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060426-NLA	NLA - Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
1		1060427-NLA	NLA - Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
1		1062569-NLA	NLA - Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
2	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
4		1052102	Hinge, Footplate - Right	1	1
4		1052103	Hinge, Footplate - Left	1	1
5		1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
5		1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
5		1091745-NLA	NLA - Tube, Pivot/Slide, Straight, Steel, Chrome (13-7/8" L)	1	1
6		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
7		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
8		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
9		7930333	Locknut (1/4-20)	25	1
10		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
11		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
12	A	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
13	A	1058517	Clamp, Footplate	1	1
14		See Chart	#14 - Adjustable Angle Footplates with Pivot Tube	1	1

NOTE: A - Items 2, 12 and 13 must be ordered together for chairs manufactured before 8/10/99 due to a hardware change.

#14 - Adjustable Angle Footplates with Pivot Tube

60°, 70° Non-Tapered and Elevating Legrest

Chair Width & Footplate	Right	Left
	Straight Slide Tube (9-7/8")	Straight Slide Tube (9-7/8")
14"-15" (5-1/4" Footplate)	1060432	1060433
16"-20" Wide (6-1/4" Footplate)	1062572	1062573
	ALPT Straight Slide Tube (11-7/8")(Alum.)	ALPT Straight Slide Tube (11-7/8")(Alum.)
14"-15" Wide (5-1/4" Footplate)	1060440-NLA	1060441
16"-20" Wide (6-1/4" Footplate)	1062578	1062579
	ALPT Straight Slide Tube (11-7/8")(Steel)	ALPT Straight Slide Tube (11-7/8")(Steel)
14"-15" Wide (5-1/4" Footplate)	1062758	1062759
16"-20" Wide (6-1/4" Footplate)	1062760	1062761