



Pivot Tube Assemblies w/ Footplates - A1562 Aluminum

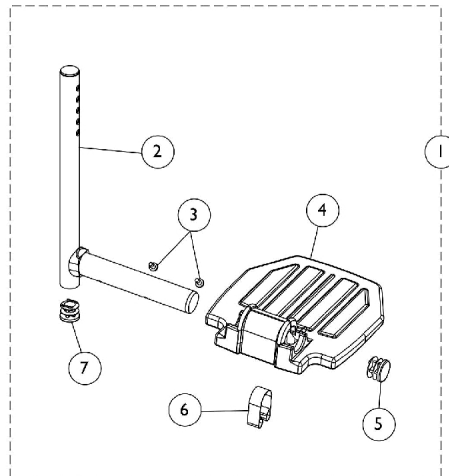

**Used on all front riggings except
Wedge Style AT5044 Elevating Legrests**

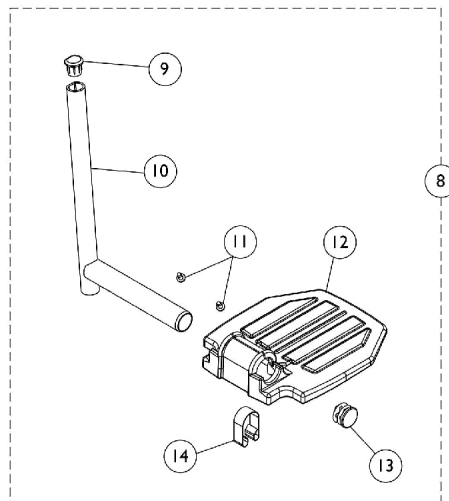

Used on Wedge Style AT5044 Elevating Legrests

Pivot Tube Assemblies w/ Footplates - A1562 Aluminum

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Aluminum Footplate w/ Pivot Tube Assembly	1	1
2		1120330	Pivot/Slide Tube w/ Plug, Black	1	1
3		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
4		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
4		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
5		1130582	Package, Plug Button, Black (7/8")	10	1
6		1006981	Spring, Tension	1	1
7	B	1134426	Package, Plug Button, Black (3/4")	10	1
8		1074568	Package, Push Plug, Footplate	2	1
9	C	See Chart	#8 Aluminum Footplate w/ Pivot Tube Assembly	1	1
9		1134427	Package, Plug Button, Black, "D" Shaped (3/4")	10	1
10		1134431	Pivot/Slide Tube w/ Plug Buttons, Black	10	1
11		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
12		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
12		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
13		1130582	Package, Plug Button, Black (7/8")	10	1
14		1006981	Spring, Tension	1	1
NOTE: A - Includes items 2-7 B - Not Shown C - Includes items 9-14					

Pivot Tube Assemblies w/ Footplates - A1562 Aluminum

#1 Aluminum Footplate w/ Pivot Tube Assembly

All Front Riggings except Wedge Style AT5044 Elevating Legrests

Description	Footplate Size	Chair Width	Right	Left
All Footrests Except 70° Tapered	Small	14"	1120335	1120336
All Footrests Except 70° Tapered	Medium	16-20"	1120337	1120338
70° Tapered Footrests	Small	16"	1120335	1120336
70° Tapered Footrests	Medium	17-20"	1120337	1120338
Elevating Legrest (Ratchet Style)	Small	14"	1120335	1120336
Elevating Legrest (Ratchet Style)	Medium	16-20"	1120337	1120338
Articulating Legrest	Small	16"	1023905-NLA	1023906-NLA
Articulating Legrest	Medium	17-20"	1023907	1023908
Includes items 2-7				

#8 Aluminum Footplate w/ Pivot Tube Assembly

Wedge Style AT5044 Elevating Legrests

Footplate Size	Chair Width	Right	Left
Small	14"	1132113	1132114
Medium	16-20"	1132073	1132074
Includes items 9-14			