



Pivot Tube w/ Footplate Assemblies - 1651 Composite

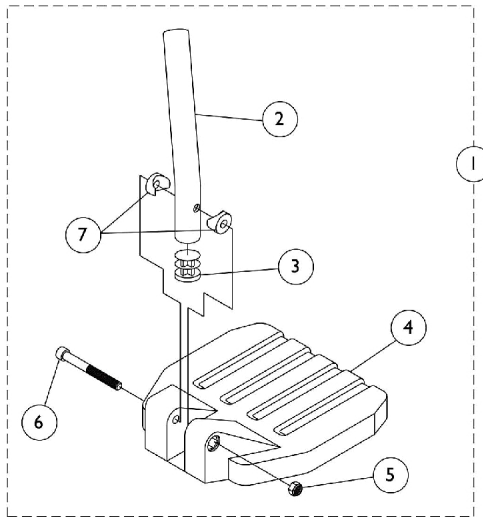

Used on all front riggings except
Wedge Style Elevating Legrests AT5044

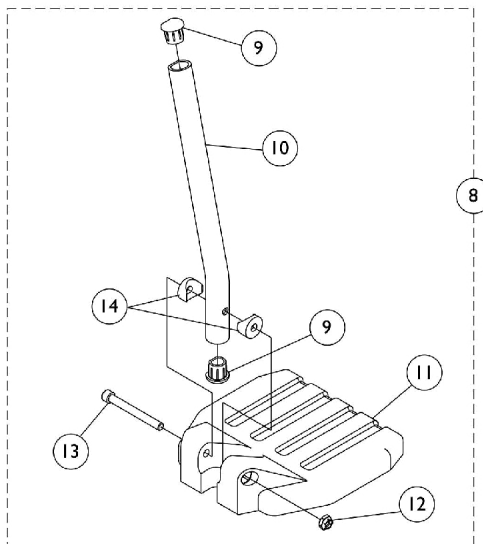

Used on Wedge Style Elevating Legrests AT5044

Pivot Tube w/ Footplate Assemblies - 1651 Composite

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|--|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | See Chart | #1 PHW93 Footrests | 1 | 1 |
| 1 | A | See Chart | #1 PH904A Legrests | 1 | 1 |
| 1 | A | See Chart | #1 60° Footrests (9-3/4-14") | 1 | 1 |
| 1 | A | See Chart | #1 70° Non Tapered Footrest (14-19") | 1 | 1 |
| 1 | A | See Chart | #1 AT5044 Elevating Legrest - Ratchet Style (16-20") | 1 | 1 |
| 2 | B | 1057852 | Tube, Pivot/Slide, Curved End, Steel, Black (9-7/8") | 1 | 1 |
| 2 | C | 1120382 | Tube, Pivot/Slide, Curved End, Black - Long (9-7/8") | 1 | 1 |
| 2 | D | 1120389 | Tube, Pivot/Slide, Straight, Black - Long (9-7/8") | 1 | 1 |
| 2 | E | 1120390 | Tube, Pivot/Slide, Straight, 2" Longer ALPT, Black (11-7/8") | 1 | 1 |
| 2 | F | 1120391 | Tube, Pivot/Slide, Curved End, 2" Longer ALPT, Black (11-7/8") | 1 | 1 |
| 3 | | 1134426 | Package, Plug Button, Black (3/4") | 10 | 1 |
| 4 | | 1026879 | Footplate, Composite, Small, Black (6" W x 6" L) | 1 | 1 |
| 4 | | 1026880 | Footplate, Composite, Medium, Black (7" W x 6" L) | 1 | 1 |
| 4 | | 1026881 | Footplate, Composite, Large, Black (8" W x 6" L) | 1 | 1 |
| 5 | | 1025195 | Locknut (1/4-20) | 1 | 1 |
| 6 | | 60285X006 | Screw, Socket Head (1/4-20 x 2-1/4") | 1 | 1 |
| 7 | | 1025457 | Spacer, Coved (1/4 x 3/4 x 11/32") | 1 | 2 |
| 8 | G | See Chart | #8 AT5044 Elevating Legrest - Wedge Style (16-20") | 1 | 1 |
| 9 | | 1134427 | Package, Plug Button, Black, "D" Shaped (3/4") | 10 | 1 |
| 10 | H | 1134428 | Pivot/Slide Tube w/ D Insert, Black | 1 | 1 |
| 10 | H | 1134429 | Pivot/Slide Tube w/ D Insert, Black (2" Longer) | 1 | 1 |
| 11 | | 1026879 | Footplate, Composite, Small, Black (6" W x 6" L) | 1 | 1 |
| 11 | | 1026880 | Footplate, Composite, Medium, Black (7" W x 6" L) | 1 | 1 |
| 11 | | 1026881 | Footplate, Composite, Large, Black (8" W x 6" L) | 1 | 1 |
| 12 | | 1025195 | Locknut (1/4-20) | 1 | 1 |
| 13 | | 60285X006 | Screw, Socket Head (1/4-20 x 2-1/4") | 1 | 1 |
| 14 | | 1025457 | Spacer, Coved (1/4 x 3/4 x 11/32") | 1 | 2 |

NOTE: A - Includes items 2-7
 B - PHW93 Footrests
 C - PH904A Legrest, 60° Footrest, and AT5044 Ratchet Style Legrest
 D - 70° Non-Tapered Footrest
 E - PHW93 w/ ALPT, PH904A w/ ALPT, 70° Non-Tapered Footrest w/ ALPT
 F - AT5044 Ratchet Style Legrest w/ ALPT
 G - Includes items 9-14
 H - AT5044 Wedge Style Legrest

Pivot Tube w/ Footplate Assemblies - 1651 Composite

#1 PHW93 Footrests

| Right/Left | Small Short 14" W | Small 2" Longer 14" W | Medium Short 15-16" W | Medium 2" Longer 15-16" W | Large Short 17-22" W | Large 2" Longer 17-22" W |
|------------|----------------------|--------------------------|--------------------------|------------------------------|-------------------------|-----------------------------|
| Right | 1057859 | 1120410 | 1057869 | 1120412 | 1057879 | 1120414 |
| Left | 1057860 | 1120411 | 1057870 | 1120413 | 1057880 | 1120415 |

#1 PH904A Legrests

| Right/Left | Small Short 14" W | Small 2" Longer 14" W | Medium Short 15-16" W | Medium 2" Longer 15-16" W | Large Short 17-22" W | Large 2" Longer 17-22" W |
|------------|----------------------|--------------------------|--------------------------|------------------------------|-------------------------|-----------------------------|
| Right | 1028391 | 1120410 | 1028387 | 1120412 | 1028383 | 1120414 |
| Left | 1028392 | 1120411 | 1028388 | 1120413 | 1028384 | 1120415 |

#1 60° Footrests (Curved End) 9-3/4 - 14" Curved End

| Right/Left | Small Long 14" wide | Medium Long 15-16" wide | Large Long 17-22" wide |
|------------|------------------------|----------------------------|---------------------------|
| Right | 1028391 | 1028387 | 1028383 |
| Left | 1028392 | 1028388 | 1028384 |

#1 70° Non Tapered; 70LIFTNT Footrest 14-19" Straight

| Right/Left | Small Short 14" wide | Small 2" Longer 14" wide | Medium Short 15-16" wide | Medium 2" Longer 15-16" wide | Large Short 17-22" wide | Large 2" Longer 17-22" wide |
|------------|-------------------------|-----------------------------|-----------------------------|---------------------------------|----------------------------|--------------------------------|
| Right | 1120404 | 1120410 | 1120406 | 1120412 | 1120408 | 1120414 |
| Left | 1120405 | 1120411 | 1120407 | 1120413 | 1120409 | 1120415 |

Pivot Tube w/ Footplate Assemblies - 1651 Composite

#1 AT5044 Elevating Legrest - Ratchet Style

16-20"

Curved End

| Right/Left | Small Long 14" wide | Small 2" Longer 14" wide | Medium Long 15-16" wide | Medium 2" Longer 15-16" wide | Large Long 17-22" wide | Large 2" Longer 17-22" wide |
|------------|------------------------|-----------------------------|----------------------------|---------------------------------|---------------------------|--------------------------------|
| Right | 1028391 | 1120416 | 1028387 | 1120418 | 1028383 | 1120420 |
| Left | 1028392 | 1120417 | 1028388 | 1120419 | 1028384 | 1120421 |

#8 AT5044 Elevating Legrest - Wedge Style

16-20"

Curved End

| Right/Left | Small Long 14" wide | Small 2" Longer 14" wide | Medium Long 15-16" wide | Medium 2" Longer 15-16" wide | Large Long 17-22" wide | Large 2" Longer 17-22" wide |
|------------|------------------------|-----------------------------|----------------------------|---------------------------------|---------------------------|--------------------------------|
| Right | 1132078 | 1132107 | 1132080 | 1132109 | 1132082 | 1132111 |
| Left | 1132079 | 1132108 | 1132081 | 1132110 | 1132083 | 1132112 |