


SureStep Walking Beams and Hardware

300 & 400 Lb. limit chairs (Before 10/04/05)

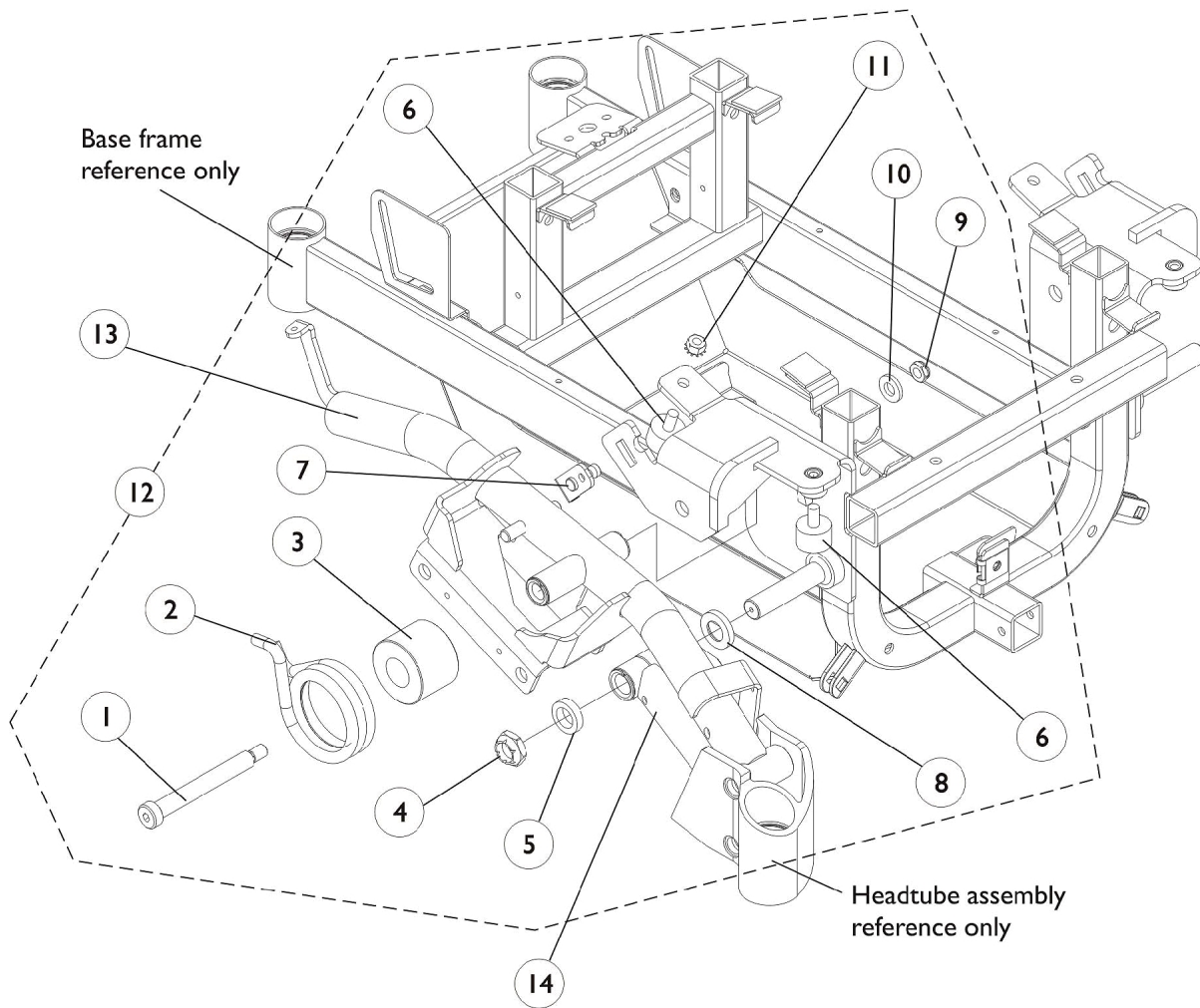

SureStep Walking Beams and Hardware

300 & 400 Lb. limit chairs (Before 10/04/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1110667-NLA	NLA - Screw, Socket Head Shoulder (1/2 x 3-3/4", 3/8-16)	1	1
2	D	1110718	Spring, Torsion - Right	1	1
2	D	1110719	Spring, Torsion - Left	1	1
2	E	1117033	Spring, Torsion - Right	1	1
2	E	1117034	Spring, Torsion - Left	1	1
3		1112361	Spacer, Spring, Black	1	1
4		1028550	Locknut (5/8-18)	1	1
5		1110685-NLA	NLA - Spacer, Bushing	1	1
6		1108437-NLA	NLA - Bumper, Rubber (1" Diameter x 5/8")	1	2
7		1113420	Plate, Lock	1	1
8		1106068	Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
9		1028486	Locknut, Low Profile (3/8-16)	1	1
10		1069548-NLA	NLA - Washer, Steel (7/16 x 13/16 x 1/8")	1	1
11		000037-NLA	NLA - Nut, Keps (5/16-18)	1	1
12	C,D	1110686-NLA	NLA - Hardware, Walking Beam Mounting, SureStep - Right	1	1
12	C,D	1110687-NLA	NLA - Hardware, Walking Beam Mounting, SureStep - Left	1	1
12	C,E	1117035-NLA	NLA - Hardware, Walking Beam Mounting, SureStep- Right	1	1
12	C,E	1117036-NLA	NLA - Hardware, Walking Beam Mounting, SureStep - Left	1	1
13	B	1110702	Beam, Walking w/ Bearings - Right	1	1
13	B	1110703	Beam, Walking w/ Bearings - Left	1	1
14	B	1112568	0	1	2
	B,A	1110669	Bearing, Flange (1/2 x 13/16 x 1/2")	1	4
NOTE: A - Not Shown B - Flanged bearings are included in items 13 and 14. C - Includes items 1-11 D - 300 Lb. limit chairs E - 400 Lb. limit chairs					