


# ASBA Seat Frame, Seat Pan, Adapter, and Hardware

Before 10/04/05

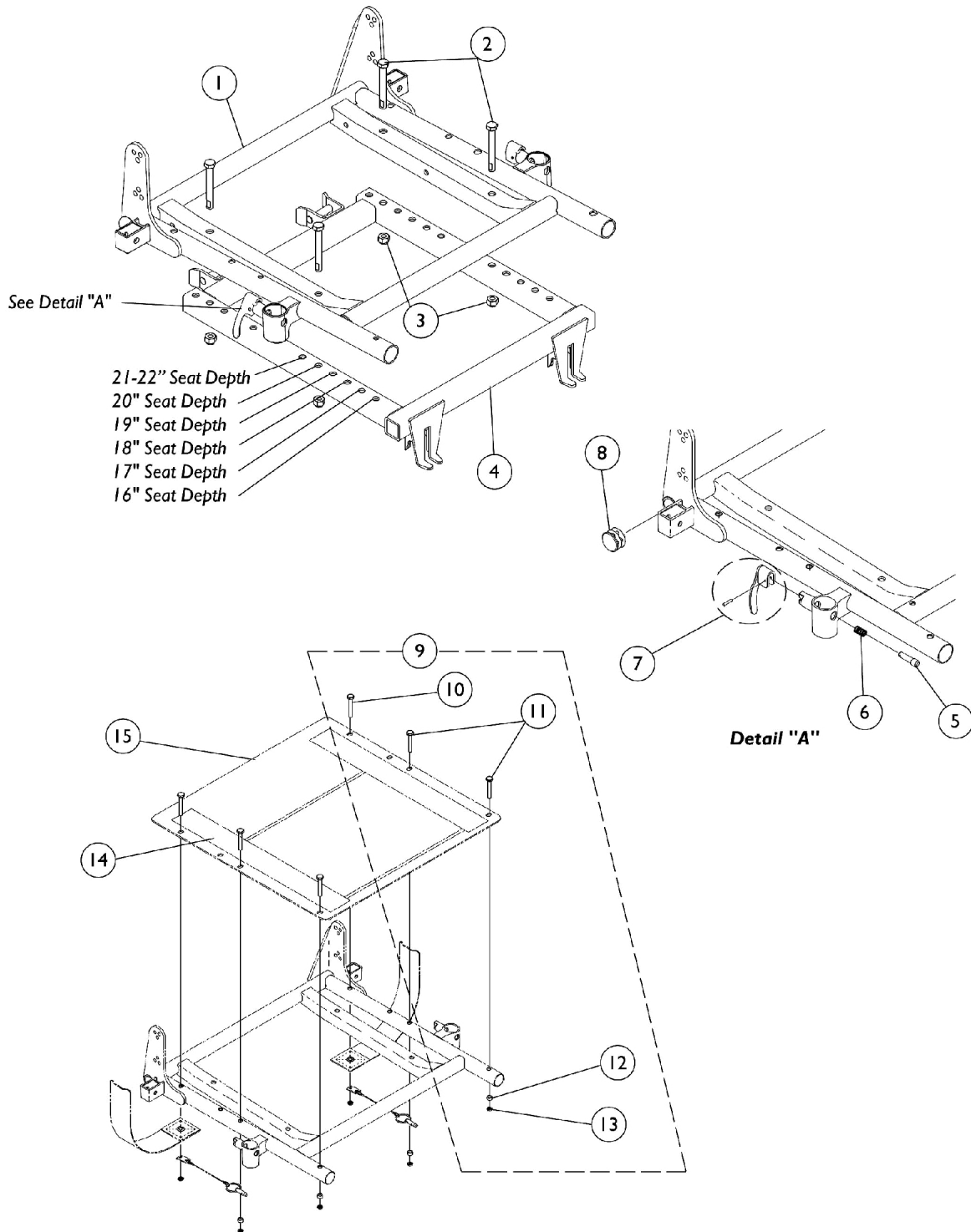

## ASBA Seat Frame, Seat Pan, Adapter, and Hardware Before 10/04/05

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Seat Frame	1	1
2		1043628-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 2-1/2")	1	4
3		1026158	Locknut (5/16-18)	1	4
4		1106480	Adapter, Seat Frame	1	1
5		1028594	Pin, Arm Lock, Black	1	2
6		47471X074	Spring, Arm Lock	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		1029501	Plug Button, Black (1")	1	2
9	B	1070459-NLA	NLA - Kit, Seat Pan Mounting Hardware	1	1
10			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
11			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
12			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
13			Locknut (1/4-20)	1	6
14		1039949-NLA	NLA - Fastener, Hook Adhesive Back (2" W x 16" L)	1	2
14		1044578-NLA	NLA - Fastener, Loop Adhesive Back (2" W x 16" L)	1	2
15		See Chart	#15 Seat Pan	1	1

NOTE: A - Includes items 5-8  
B - Not Shown. Includes items 10-13

### #1 Seat Frame

Chair Width	16"-17" Deep	18"-19" Deep	20"-21" Deep	22" Deep
16" Wide	1070672-NLA	1070681-NLA	1070690-NLA	1070699-NLA
17" Wide	1070673-NLA	1070682-NLA	1070691-NLA	1070700-NLA
18" Wide	1070674-NLA	1070683-NLA	1070692-NLA	1070701-NLA
19" Wide	1070675-NLA	1070684-NLA	1070693-NLA	1070702-NLA
20" Wide	1070676-NLA	1070685-NLA	1070694-NLA	1070703-NLA
21" Wide	1070677-NLA	1070686-NLA	1070695-NLA	1070704-NLA
22" Wide	1070678-NLA	1070687-NLA	1070696-NLA	1070705-NLA
23" Wide	1070679-NLA	1070688-NLA	1070697-NLA	1070706-NLA
24" Wide	1070680-NLA	1070689-NLA	1070698-NLA	1070707-NLA

**ASBA Seat Frame, Seat Pan, Adapter, and Hardware**  
**Before 10/04/05**  
**#15 Seat Pan**

Chair Width	16" Deep	17" Deep	18" Deep	19" Deep	20" Deep	21" Deep	22" Deep
16" Wide	1066013-NLA	1066024	1066035-NLA	1066046-NLA	1066057-NLA	1066068-NLA	1066079-NLA
17" Wide	1066014-NLA	1066025-NLA	1066036-NLA	1066047-NLA	1066058-NLA	1066069-NLA	1066080-NLA
18" Wide	1066015-NLA	1066026	1066037-NLA	1066048-NLA	1066059-NLA	1066070-NLA	1066081-NLA
19" Wide	1066016-NLA	1066027-NLA	1066038-NLA	1066049-NLA	1066060	1066071-NLA	1066082-NLA
20" Wide	1066017-NLA	1066028-NLA	1066039-NLA	1066050-NLA	1066061-NLA	1066072-NLA	1066083-NLA
21" Wide	1066018-NLA	1066029-NLA	1066040-NLA	1066051-NLA	1066062-NLA	1066073-NLA	1066084-NLA
22" Wide	1066019-NLA	1066030-NLA	1066041-NLA	1066052-NLA	1066063-NLA	1066074-NLA	1066085-NLA
23" Wide	1066020-NLA	1066031-NLA	1066042-NLA	1066053-NLA	1066064-NLA	1066075-NLA	1066086-NLA
24" Wide	1066021-NLA	1066032-NLA	1066043-NLA	1066054-NLA	1066065-NLA	1066076-NLA	1066087-NLA