



Arm Mounting Swingaway Hardware

ASBA and ADJASBA Seats

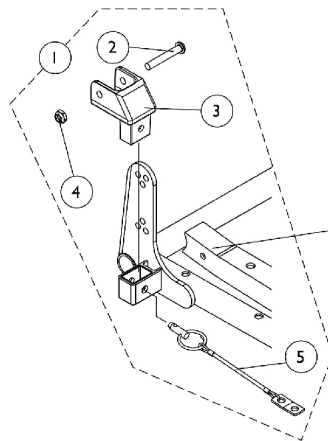

ASBA Seat Frame
Before 10/04/05 shown
for reference only.

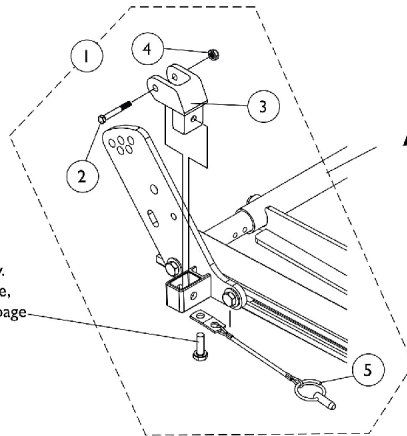

ADJASBA Seat Frame
After 10/03/05 shown for
reference only.

Hex Cap Screw
shown for reference only.
See ADJASBA Seat Frame,
Seat Pan and Hardware page

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1068488	Kit, Swingaway Armrest Mounting Hardware	1	1
2		1019590	Screw, Socket Button Head (1/4-20 x 1-5/8")	1	1
3		1068025	Clevis, Pivot	1	1
4		1025195	Locknut (1/4-20)	1	1
5		1068039	Pin, Pull	1	1

NOTE: A - Includes items 2-5