


Front Rigging Hanger Assembly

ASBA (Before 10/04/05) and Van Seats (All dates)

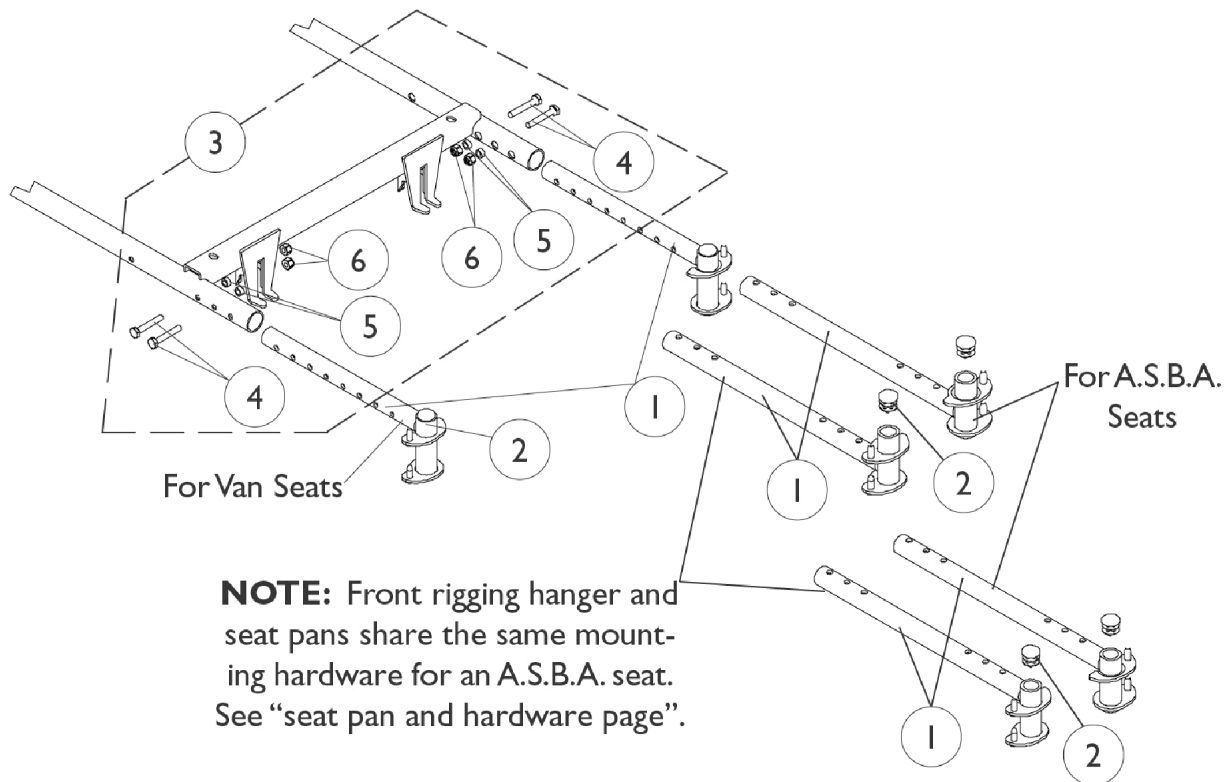

Front Rigging Hanger Assembly

ASBA (Before 10/04/05) and Van Seats (All dates)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1114782-NLA	NLA - Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Right	1	1
1	A	1114783-NLA	NLA - Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Left	1	1
1	B	1114766	Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Right (12 - 21" D)	1	1
1	B	1114767	Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Left (12 - 21" D)	1	1
1	D	1132339-NLA	NLA - Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Right (22" D)	1	1
1	D	1132340-NLA	NLA - Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Left (22" D)	1	1
1	C	1113026	Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Right	1	1
1	C	1113027	Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Left	1	1
2		1027097	Plug Button, Black (3/4")	1	1
3		1113045	Kit, Front Rigging Hanger Hardware	1	1
4			Locknut (1/4-20)	1	4
5			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
6			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
NOTE: A - For all A.S.B.A. seats Before 1/6/03 B - 16-21" deep chairs with A.S.B.A. seats After 1/5/03 C - For van seats D - 22" deep chairs with A.S.B.A. seats After 1/5/03					