
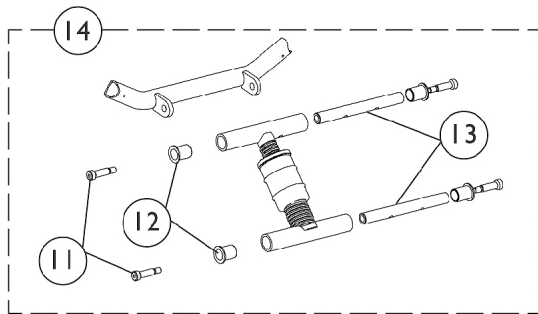
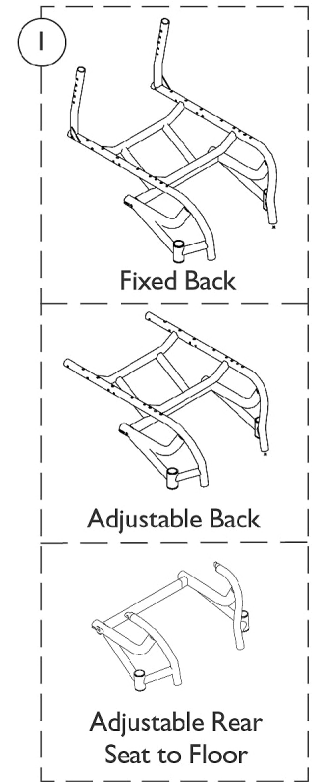


Seat Frames and Frame Hardware

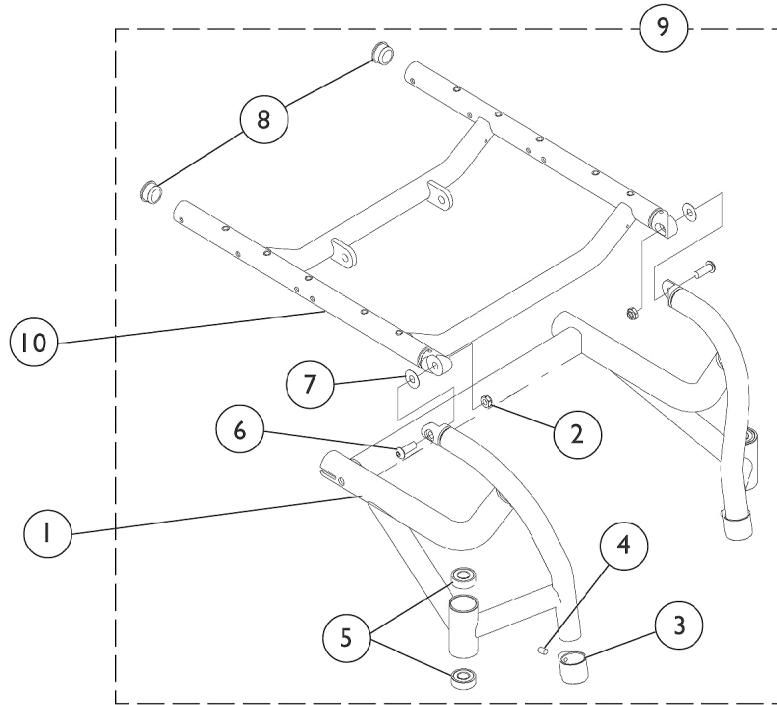

Seat Frames and Frame Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881101592-NL A	NLA - Frame, Aluminum, Color	1	1
2		1028769	Locknut (5/16-18)	1	2
3		1111949-NLA	NLA - Clamp, Footrest	1	2
4		1026911-NLA	NLA - Screw, Socket Set Cone Point (1/4-20 x 1/2")	1	2
5		40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	4
6	B	1000208-NLA	NLA - Screw, Shoulder Hex Socket Head (5/16" x 3/8", 1/4-20)	1	2
6	C	1046494	Screw, Socket Button Head (5/16-18 x 7/8")	1	2
7		1046497-NLA	NLA - Washer, Nylon (5/16 x 7/8 x 1/64")	1	4
8		1056328	Plug Button, Black (1")	1	2
9	D	1098112-NLA	NLA - Hardware, Frame, A6, Both Sides	2	1
10		See Chart	#10 Seat Frames	1	1
11		1069421-NLA	NLA - Screw, Shoulder Hex Socket Head (3/8 x 1", 5/16-18)	1	4
12		1093116-NLA	NLA - Flange	1	4
13		1095638-NLA	NLA - Rod, Attaching, Adjustable Height	1	2
14		See Chart	#14 Seat Adjustment Assembly	2	1

NOTE: A - Configured Item
 B - Chairs built Before 9/16/02. Due to design improvement, this screw was replaced with part nos. 1046494 and 1028769.
 C - Chairs built After 9/15/02.
 D - Includes items 2-8.

Seat Frames and Frame Hardware

#10 Seat Frames

Chair Width	14" Deep	15" Deep	16" Deep	17" Deep	18" Deep	19" Deep	20" Deep
14"	8881093166-NLA	8881093173-NLA	8881093180-NLA	8881093187-NLA	8881093194-NLA	N/A	N/A
15"	8881093167-NLA	8881093174-NLA	8881093181-NLA	8881093188-NLA	8881093195-NLA	N/A	N/A
16"	8881093168-NLA	8881093175-NLA	8881093182-NLA	8881093189-NLA	8881093196-NLA	8881093203-NLA	N/A
17"	8881093169-NLA	8881093176-NLA	8881093183-NLA	8881093190-NLA	8881093197-NLA	8881093204-NLA	N/A
18"	8881093170-NLA	8881093177-NLA	8881093184-NLA	8881093191-NLA	8881093198-NLA	8881093205-NLA	N/A
19"	N/A	N/A	8881093185-NLA	8881093192-NLA	8881093199-NLA	8881093206-NLA	N/A
20"	N/A	N/A	8881093186-NLA	8881093193-NLA	8881093200-NLA	8881093207-NLA	8881093972-NLA

Due to design improvement, when ordering a Seat Frame for chairs built Before 9/16/02 also order 2 each of item 2 Locknut (1028769) and item 6 Screw (1046494).
Specify frame color when ordering.

#14 Seat Adjustment Assembly

Front STF	22" Wheels	24" Wheels	26" Wheels
17"	1095431-NLA	1098519-NLA	N/A
18"	1095431-NLA	1095431-NLA	1098519-NLA
19"	1095432-NLA	1095431-NLA	1095431-NLA
20"	1095432-NLA	1095432-NLA	1095431-NLA
21"	1095432-NLA	1095432-NLA	1095432-NLA