


Adjustable Height Arms

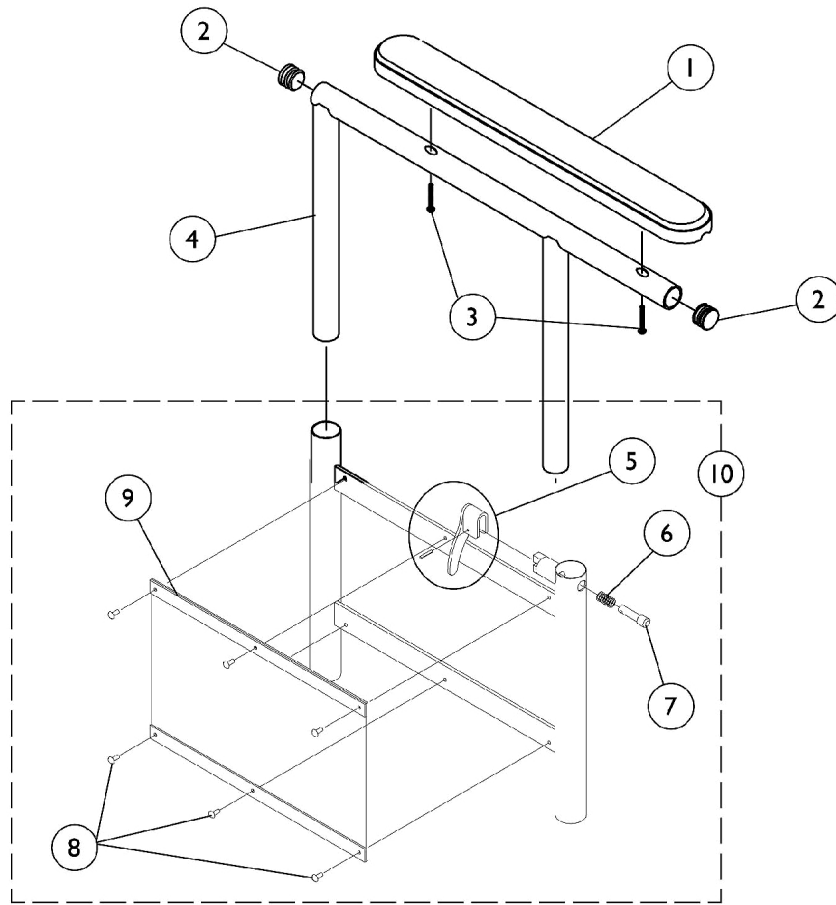

Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053578	Kit, Arm Pad w/ Screws, Desk Length (9" L x 2" W) (Pair)	2	1
1	A	8881053577	Kit, Arm Pad w/ Screws, Full Length (14" L x 2" W) (Pair)	2	1
2		1027095	Plug Button, Black (7/8")	1	2
3		1025714	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
4		1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
4		1041522	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
10	B	1035892	Adjustable Height Arm Base Assembly - Right	1	1
			Base, Armrest, Adjustable Height	1	1
5			Kit, Lock Handle w/Rivets (Pair)	1	1
6			Spring, Arm Lock	1	1
7			Pin, Arm Lock, Black	1	1
8			Rivet	1	6
9			NLA - Guard, Clothing	1	1
10	B	1035893	Adjustable Height Arm Base Assembly - Left	1	1
			Base, Armrest, Adjustable Height	1	1
5			Kit, Lock Handle w/Rivets (Pair)	1	1
6			Spring, Arm Lock	1	1
7			Pin, Arm Lock, Black	1	1
8			Rivet	1	6
9			NLA - Guard, Clothing	1	1

NOTE: A - Specify upholstery color U67 Black when ordering
B - Includes Arm Base plus items 5-9