




Adjustable Height Arms

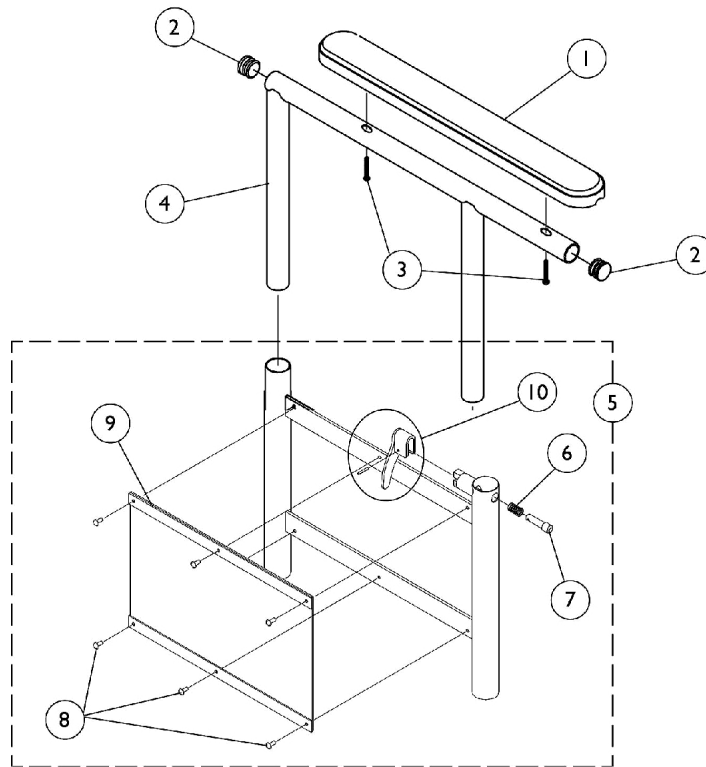

Arm Attaching Hardware

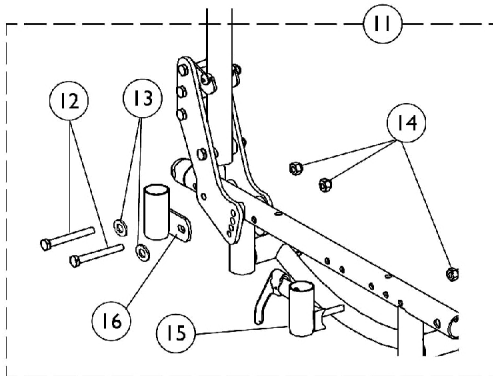

**All chairs without
Hand Operated Hublock Wheel Locks**

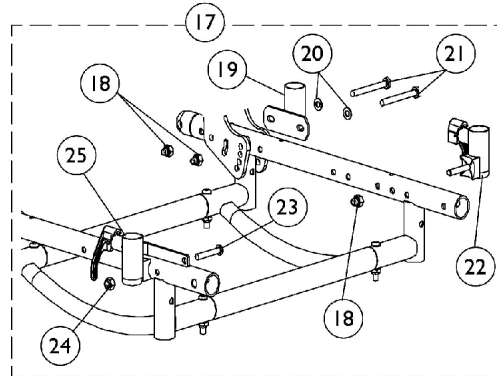

**Chairs with Hand Operated
Hublock Wheel Locks**

Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053578	Kit, Arm Pad w/ Screws, Desk Length (9" L x 2" W) (Pair)	2	1
1	A	8881053577	Kit, Arm Pad w/ Screws, Full Length (14" L x 2" W) (Pair)	2	1
2		1027095	Plug Button, Black (7/8")	1	2
3		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1
4		1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
4		1041522	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
5	B	1035892	Adjustable Height Arm Base Assembly - Right	1	1
6			Base, Armrest, Adjustable Height	1	1
6			Spring, Arm Lock	1	1
7			Pin, Arm Lock, Black	1	1
8			Rivet	1	6
9			NLA - Guard, Clothing	1	1
10			Kit, Lock Handle w/Rivets (Pair)	1	1
5	B	1035893	Adjustable Height Arm Base Assembly - Left	1	1
6			Base, Armrest, Adjustable Height	1	1
6			Spring, Arm Lock	1	1
7			Pin, Arm Lock, Black	1	1
8			Rivet	1	6
9			NLA - Guard, Clothing	1	1
10			Kit, Lock Handle w/Rivets (Pair)	1	1
11	D	1133487	Kit, Arm Attaching Hardware	1	1
12			Screw, Hex Head (1/4-28 x 2-1/4")	1	4
13			Washer (1/4 x 1/2 x 1/16")	1	4
14			Nut, Hex with Nylon Cap (1/4-20)	1	6
15			Socket, Arm, Front - Right	1	1
15			Socket, Arm, Front - Left	1	1
16			Socket, Arm, Rear - Right/Left	1	2
17	E	1133488	Kit, Arm Attaching Hardware for Hand Oper. Hublock	1	1
18			Nut, Hex with Nylon Cap (1/4-20)	1	6
19			Socket, Arm, Rear - Right/Left	1	2
20			Washer (1/4 x 1/2 x 1/16")	1	4
21			Screw, Hex Head (1/4-28 x 2-1/4")	1	4
22			Socket, Arm, Front - Left	1	1
23			Screw, Socket Button Head (1/4-28 x 1-1/2")	1	1
24			Locknut (1/4-28)	1	1
25			Arm Socket Assembly, Hublock - Right	1	1

NOTE: A - Specify upholstery color U67 Black when ordering
B - Includes items 6-10
D - Includes items 12-16. Contains enough hardware for both arms
E - Includes items 18-25. Contains enough hardware for both arms