



Forks and Hardware

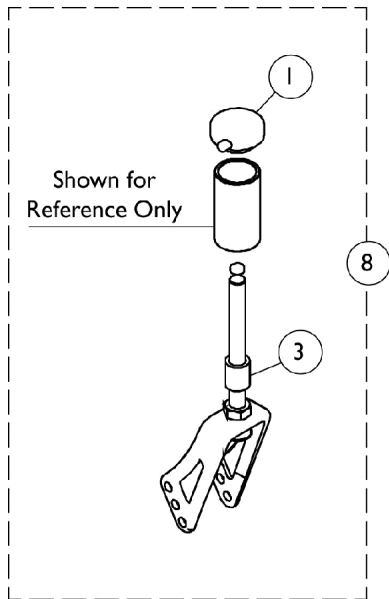

**Non
Suspension**

Suspension

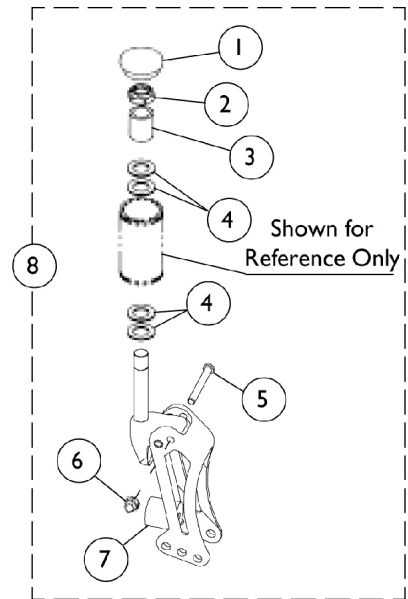

Forks and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1088523	Plug Button, Head Tube, Black	1	1
1		1042044-NLA	NLA - Cap, Head Tube, Quick Release Caster, Black	1	1
2		1025737	Locknut (1/2-20)	1	6
3		1046495	Spacer (1/2 x 3/4 x 1")	1	1
4		1026174	Washer (1/2 x 7/8 x 3/64")	1	1
5		1030957-NLA	NLA - Screw, Shoulder Hex Socket Head (1/4 x 1-3/4", #10-24)	1	1
6		1100857-NLA	NLA - Nut, Hex with Nylon Cap (#10-24)	1	1
7		See Chart	#7 Polyurethane Springs	2	1
7		See Chart	#7 Polyurethane Spring Kits	2	1
8		See Chart	#8 Caster Forks with Hardware	1	2

#7 Polyurethane Springs

User's Weight Range	Spring Color	Part Number
0 lbs to 140 lbs	Blue	1100506-NLA See Note(s) A
141 lbs to 220 lbs	Black	1100507-NLA See Note(s) A
221 lbs to 250 lbs	Purple	1100508-NLA See Note(s) A

NOTE: A - Includes 2 Springs

#7 Polyurethane Spring Kits

User's Weight Range	Spring Quantity/Color	Part Number
0 Lbs. to 100 Lbs.	4 Blue	1100509-NLA See Note(s) A
101 Lbs. to 140 Lbs.	2 Blue / 2 Black	1100510-NLA See Note(s) A
141 Lbs. to 180 Lbs.	4 Black	1100511-NLA See Note(s) A
181 Lbs. to 220 Lbs.	2 Black / 2 Purple	1100512-NLA See Note(s) A
221 Lbs. to 250 Lbs.	4 Purple	1100513-NLA See Note(s) A

NOTE: A - Includes 4 Springs

Forks and Hardware

#8 Caster Forks with Hardware

Fork Type	Standard/Quad Release	Fork Stem	0-140 Lbs. User's Weight	141-220 Lbs. User's Weight	221-250 Lbs. User's Weight
Non Suspension	Standard	With 1" Extension	1101206-NLA	1101206-NLA	1101206-NLA
Non Suspension	Quick Release	Without 1" Extension	1098499-NLA	1098499-NLA	1098499-NLA
Non Suspension	Quick Release	With 1" Extension	1098500-NLA	1098500-NLA	1098500-NLA
Suspension	Standard	With 1" Extension	1098110-NLA	1098109-NLA	1098108-NLA
Suspension	Quick Release	Without 1' Extension	1098517-NLA	1098515-NLA	1098513-NLA
Suspension	Quick Release	With 1' Extension	1098518-NLA	1098516-NLA	1098514-NLA