

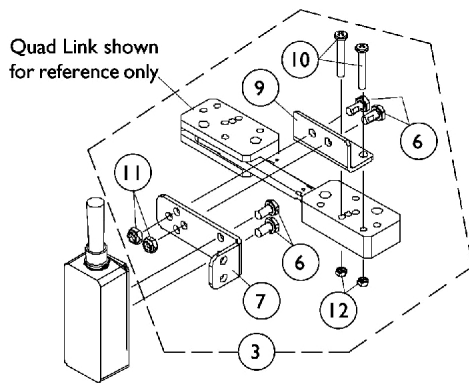
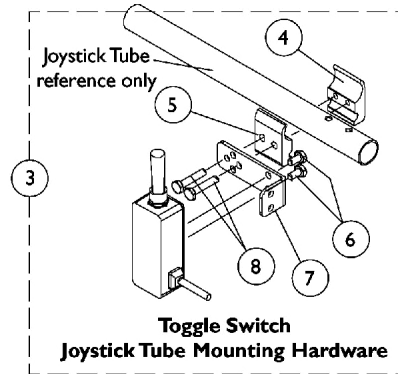


# Tilt Recline Single Switch (TRSS)


As of 7/1/2009, switches are mercury free. TRSS Tilt Recline Single Switch is not used with MK6i electronics.

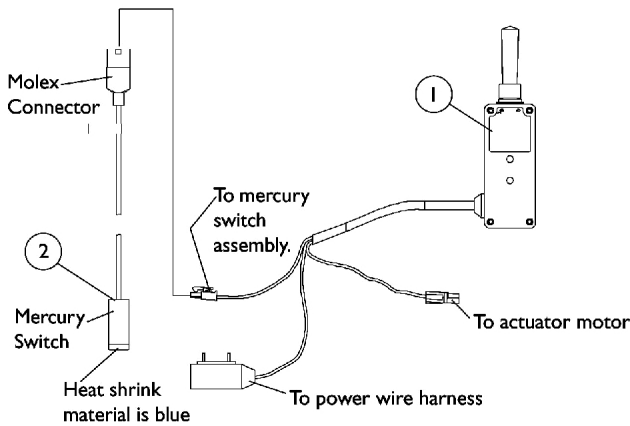


**Toggle Switch  
Quad Link Mounting Hardware  
Before 10/31/05**

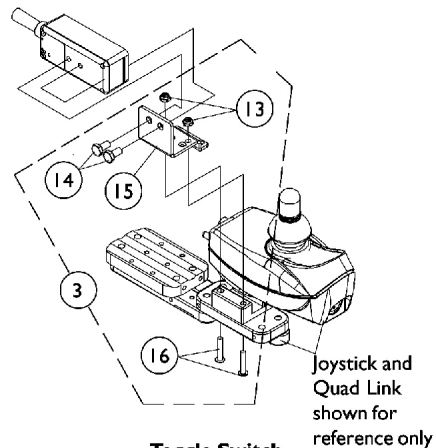

**Toggle Switch  
Quad Link Mounting Hardware  
After 10/30/05**

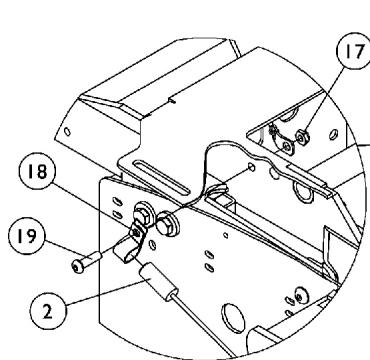

**Mercury Switch Mounting Hardware**

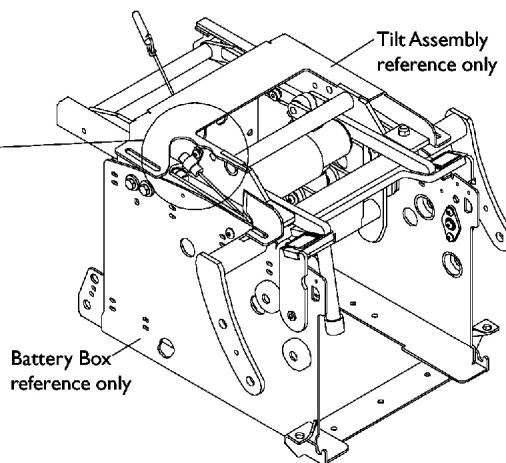

**Battery Box  
reference only**

**Tilt Assembly  
reference only**

# Tilt Recline Single Switch (TRSS)

As of 7/1/2009, switches are mercury free. TRSS Tilt Recline Single Switch is not used with MK6i electronics.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1089384	Switch, Toggle Tilt Recline Single (TRSS)	1	1
2		1080635-NLA	NLA - Harness, Recline/Tilt Limit Mercury Free Ball Switch	1	1
3	A	1091496	Kit, Joystick Tube Toggle Switch Mounting Hardware	1	1
4			NLA - Half Clamp, Threaded Holes	1	1
5			Half Clamp, Non-Threaded Holes	1	1
6			Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	2
7			NLA - Bracket, Toggle Switch Mounting, Black	1	1
8			NLA - Screw, Hex Head (1/4-20 x 1-1/8")	1	2
3	B	1091497-NLA	NLA - Kit, Quad Link Toggle Switch Mounting Hardware	1	1
6			Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	4
7			NLA - Bracket, Toggle Switch Mounting, Black	1	1
9			NLA - Bracket, Quad Link Toggle Switch Mounting, Black	1	1
10			Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
11			Locknut (1/4-20)	1	2
12			Locknut (#10-32)	1	2
3	C	1139684-NLA	NLA - Kit, Quad Link Toggle Switch Mounting Hardware	1	1
13			Locknut (#10-32)	1	2
14			Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	2
15			Bracket, Toggle Switch Mounting, Black	1	1
16			Screw, Phillips Pan Head w/Patch (#10-32 x 7/8")	1	2
17		1025195	Locknut (1/4-20)	1	1
18		0062800-NLA	NLA - Bracket, Cable/Wire Mounting, Black	1	1
19		1028410-NLA	NLA - Screw, Socket Button Head (1/4-20 x 7/8")	1	1

NOTE: A - Includes items 4-8  
 B - Before 10/31/05 and includes items 6, 7 and 9-12  
 C - After 10/30/05 and includes items 13-16