


# ADJASBA Seat Frame, Seat Pan and Hardware

TRBKTS Option - Introduced 12/19/05

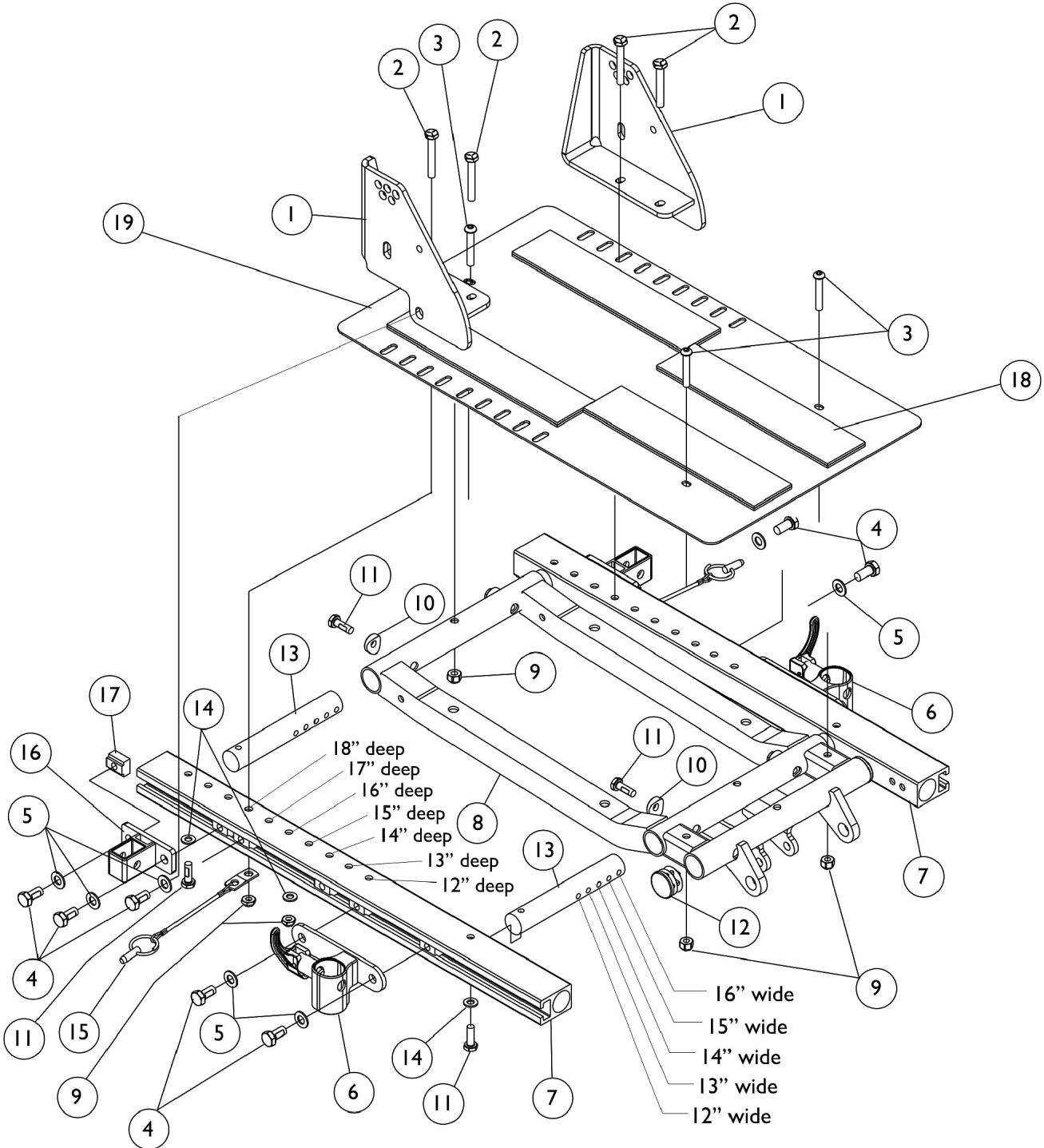

## ADJASBA Seat Frame, Seat Pan and Hardware

### TRBKTS Option - Introduced 12/19/05

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1133597-NLA	NLA - Bracket, Back Cane Mounting, Black - Right	1	1
1		1133598-NLA	NLA - Bracket, Back Cane Mounting, Black - Left	1	1
2		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	4
3		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	3
4		1122652	Screw, Hex Head w/ Patch (5/16-18 x 5/8")	1	10
5		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	10
6		1122619	Socket, Arm, Front - Right	1	1
6		1122620	Socket, Arm, Front - Left	1	1
7		1126127-NLA	NLA - Rail, Side (12 - 18" D)	1	2
8		1121982-NLA	NLA - Frame, Center Seat, Black	1	1
9		1025736	Locknut (1/4-20)	1	7
10		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
11		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	8
12		1029501	Plug Button, Black (1")	1	2
13		1126126-NLA	NLA - Crossbar, Seat Frame (12 - 16" W)	1	4
14		1026907	Washer (1/4 x 1/2 x 1/16")	1	6
15		1068039	Pin, Pull	1	2
16		1126142	Socket, Arm, Rear	1	2
17		1069724	T-Nut (5/16-18)	1	10
18		1039949-NLA	NLA - Fastener, Hook Adhesive Back (2" W x 16" L)	1	4
18		1044578-NLA	NLA - Fastener, Loop Adhesive Back (2" W x 16" L)	1	4
19	A	1126144	Pan, Seat, with Hook and Loop Fasteners (12 - 16" W x 12 - 18" D)	1	1

NOTE: A - Includes item 18