


# ADJASBA Seat Frame Mounting Hardware (12"-16" Wide) Low S-T-F TRRO/TRBKTS Option

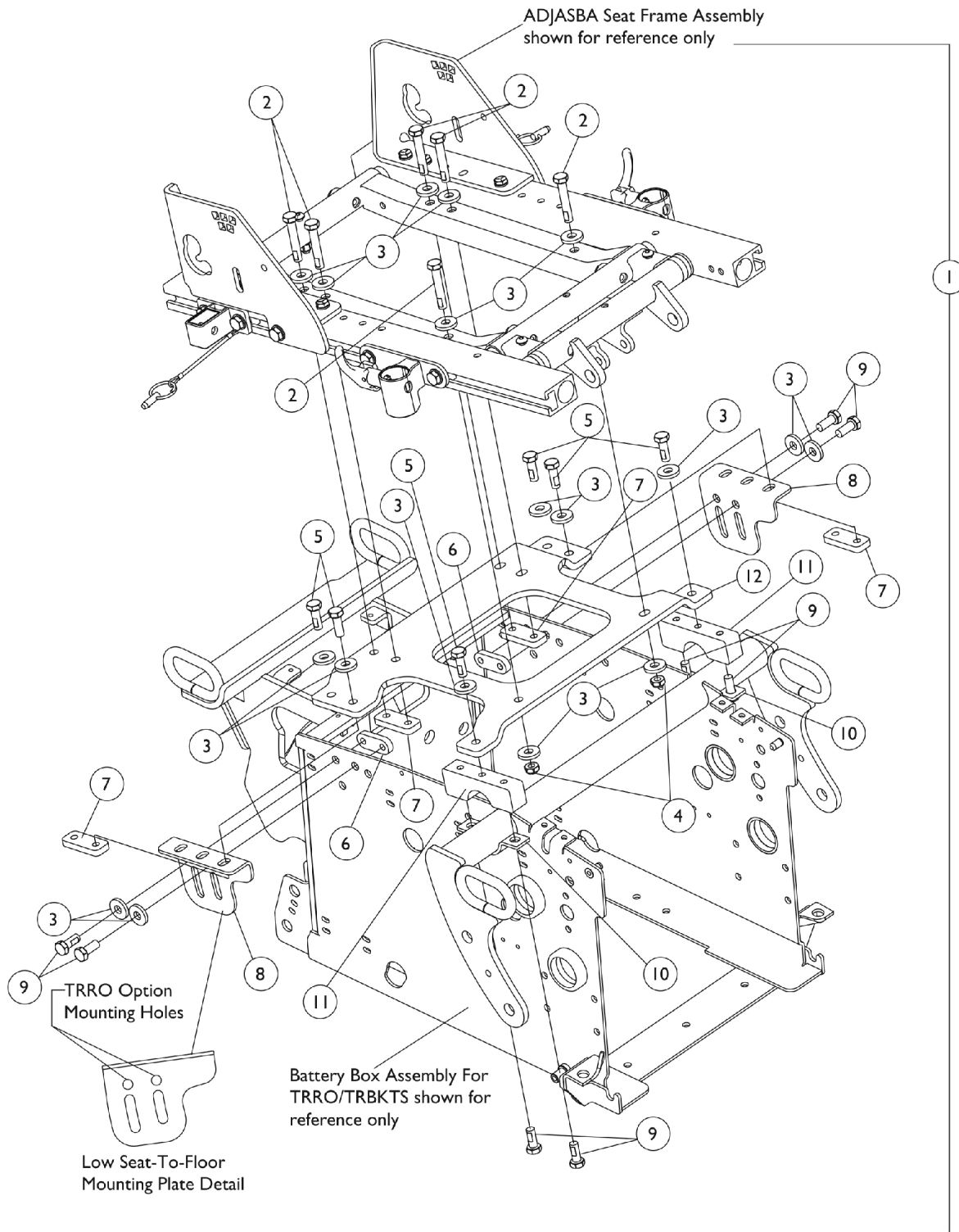

## ADJASBA Seat Frame Mounting Hardware (12"-16" Wide) Low S-T-F TRRO/TRBKTS Option

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1139868-NLA	NLA - Kit, Seat Frame Mtg. Hdwr. 12"-16" W, Low S-T-F with TRRO/TRBKTS Option	1	1
2			NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	6
3			Washer, Flat (21/64 x 53/64 x 7/64")	1	18
4			Locknut (5/16-18)	1	4
5			NLA - Screw, Hex Head w/ Patch (5/16-18 x 7/8")	1	6
6			NLA - Nut, Strap	1	2
7			Plate, Nut (5/16-18)	1	4
8			Plate, Low Seat-To-Floor Mounting, Rear TRRO/TRBKTS - Right	1	1
8			Plate, Low Seat-To-Floor Mounting, Rear TRRO/TRBKTS - Left	1	1
9			Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	8
10			NLA - Bracket, Lower Seat, Front, Black	1	2
11			NLA - Bracket, Upper Seat, Front, Black	1	2
12			Bracket, Seat Frame Mounting, Black (12 - 16" W)	1	1

NOTE: A - Includes items 2-12