
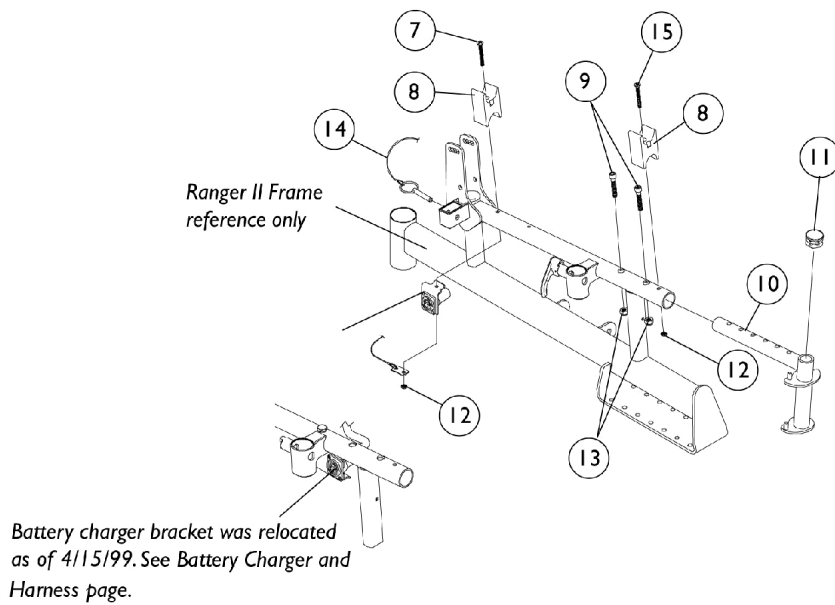


# Side Frames & Hardware

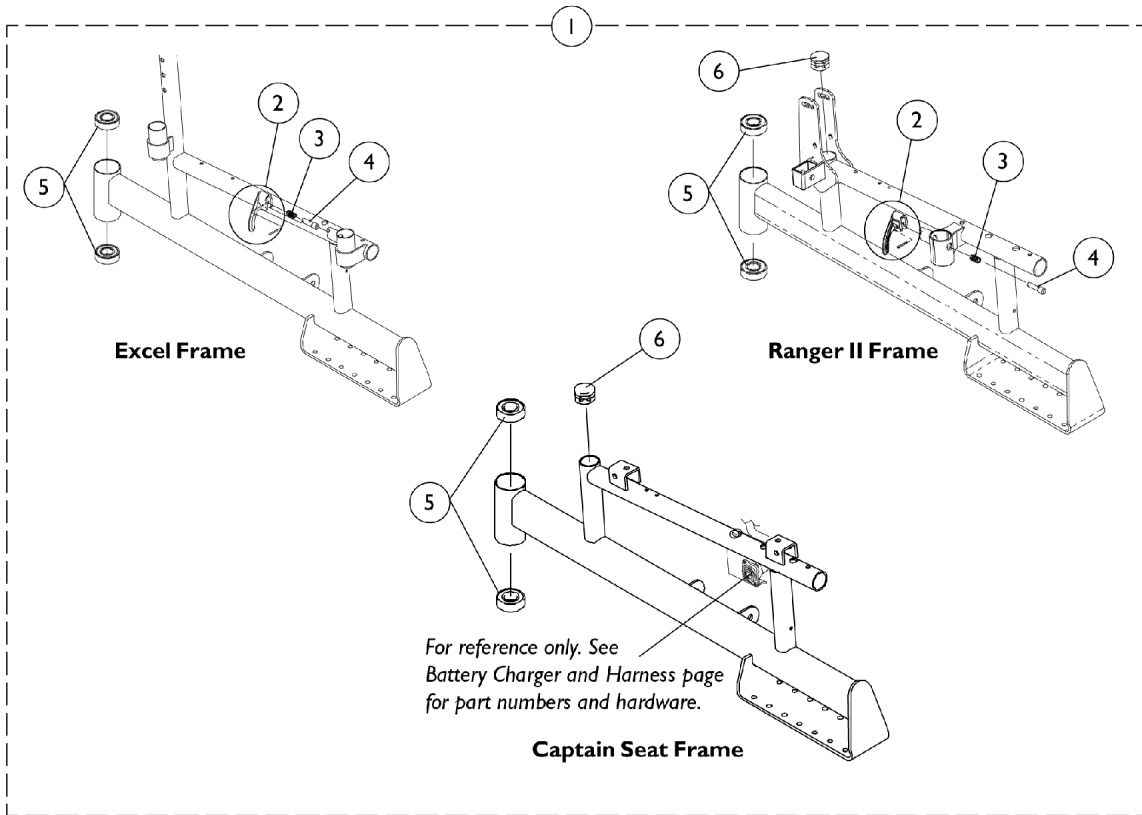

## Side Frames & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,C	8881080260-NL A	NLA - Frame, Side with Hardware, Color - Right	1	2
1	A,B,C	8881080261-NL A	NLA - Frame, Side with Hardware, Color - Left	1	2
1	A,B,D	8881080734-NL A	NLA - Frame, Side with Hardware, Color - Right	1	2
1	A,B,D	8881080735-NL A	NLA - Frame, Side with Hardware, Color - Left	1	2
1	A,B,E	8881085258-NL A	NLA - Frame, Side with Hardware, Color - Right	1	2
1	A,B,E	8881085259-NL A	NLA - Frame, Side with Hardware, Color - Left	1	2
2		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
3		47471X074	Spring, Arm Lock	1	2
4		1028594	Pin, Arm Lock, Black	1	2
5		40117X006	Bearing (5/8")	1	4
6		1029501	Plug Button, Black (1")	1	2
7		1085124-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1-3/4")	1	4
8		1007541	Clamp, Seat	1	4
9		7930304-NLA	NLA - Screw, Socket Head (1/4-20 x 1-1/4")	25	1
10		1060099-NLA	NLA - Hanger, Front Rigging, Black - Right	1	2
10		1060100-NLA	NLA - Hanger, Front Rigging, Black - Left	1	2
11		1027097	Plug Button, Black (3/4")	1	2
12		1025731	Locknut (#10-32)	1	3
13		7930333	Locknut (1/4-20)	25	1
14		1068039	Pin, Pull	1	2
15		1085134-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1-5/8")	1	2
<p>NOTE: A - Specify frame color when ordering. See page titled, "Frame Finishes, Touch-Up Paint and Upholstery Options."            B - Decals not included with frames. See page titled, "Packaging, Literature and Decals."            C - Ranger II.            D - Excel.            E - Captain Seat.</p>					