
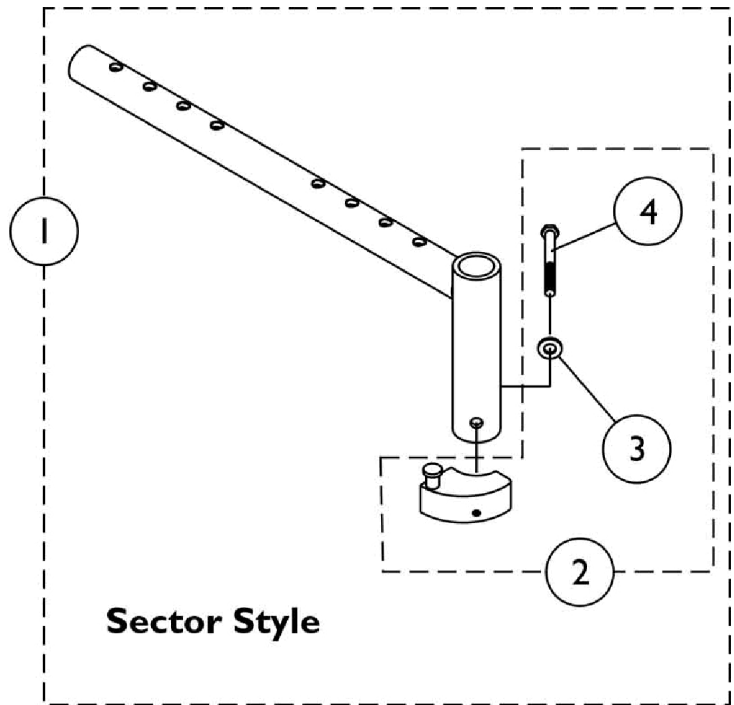


Front Rigging Hanger Assembly

Pin and Sector Style For Chairs with ASBA Seats (Before 7/5/05)

NOTE: Front Rigging Hangers and Seat Pans share the same mounting hardware, see Seat Pan and Mounting Hardware page for the illustration.

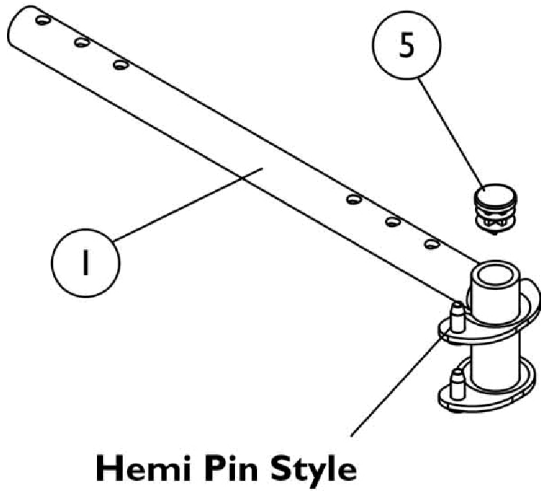

Front Rigging Hanger Assembly

Pin and Sector Style For Chairs with ASBA Seats (Before 7/5/05)

NOTE: Front Rigging Hangers and Seat Pans share the same mounting hardware, see Seat Pan and Mounting Hardware page for the illustration.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1068435	Front Rigging Hanger Assembly w/ Sector Block, Black - Right (12 - 21" D)	1	1
1	A	1068436	Front Rigging Hanger Assembly w/ Sector Block, Black - Left (12 - 21" D)	1	1
1	A	1068437-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Black - Right (22" D)	1	1
1	A	1068438-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Black - Left (22" D)	1	1
1		1114766	Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Right (12 - 21" D)	1	1
1		1114767	Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Left (12 - 21" D)	1	1
1		1132339-NLA	NLA - Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Right (22" D)	1	1
1		1132340-NLA	NLA - Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Hemi, Black - Left (22" D)	1	1
2	B	1054571	Kit, Sector Block - Right	1	1
2	B	1054572	Kit, Sector Block - Left	1	1
3		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
4		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	1
5		1027097	Plug Button, Black (3/4")	1	1
	C	1070459-NLA	NLA - Kit, Seat Pan Mounting Hardware	1	1
			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
			Locknut (1/4-20)	1	6

NOTE: A - Includes front rigging hanger and item 2
 B - Includes sector block assembly and items 3-4
 C - Not Shown. Front Rigging Hangers and Seat Pans share the same mounting hardware