


Crossbraces & Hardware Captain Seat

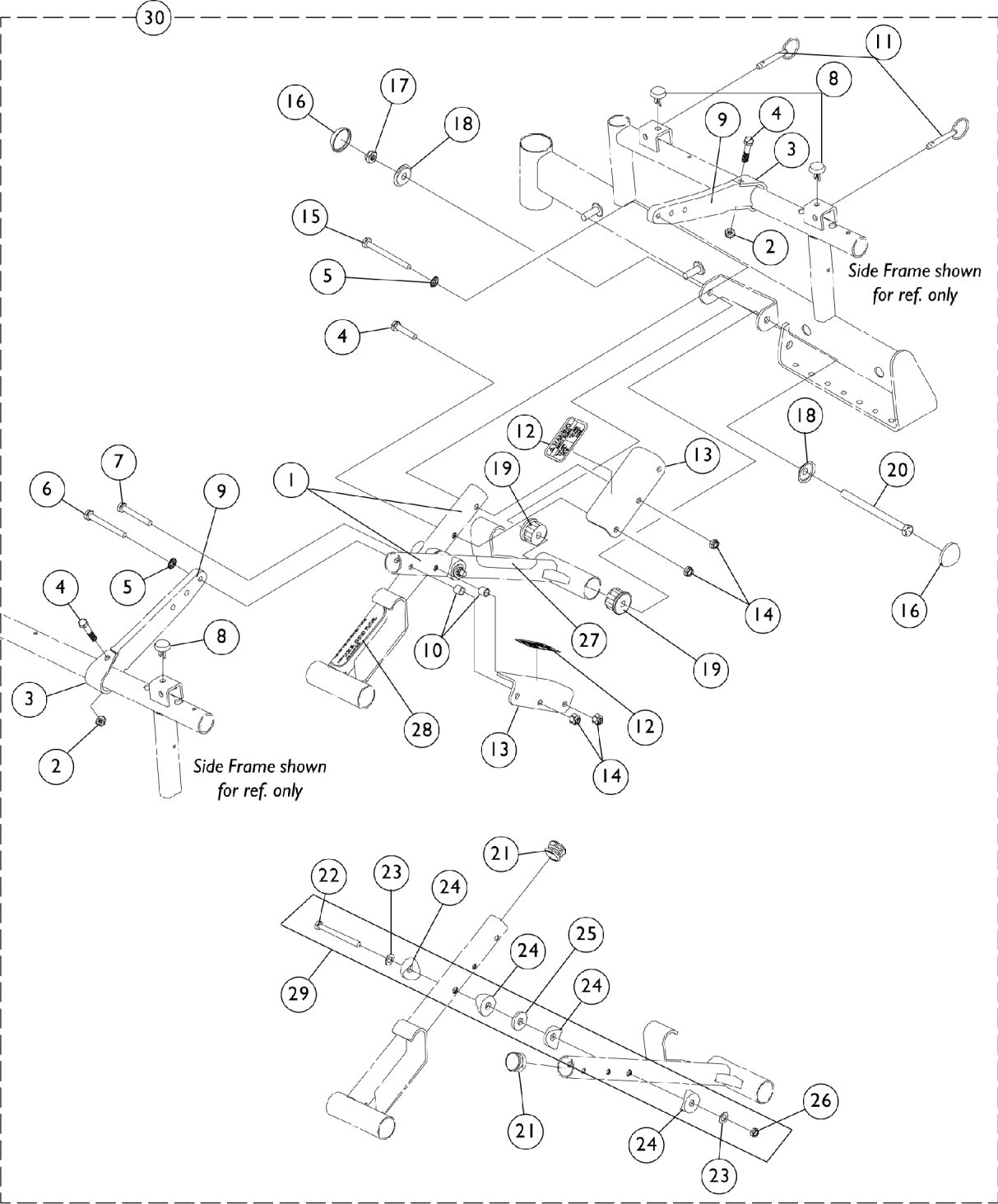

Crossbraces & Hardware Captain Seat

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|------------------------------------------------------------|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | 1086798-NLA | NLA - Crossbrace, Captain Seat, Black - Right (18" W) | 1 | 2 |
| 1 | A | 1086799-NLA | NLA - Crossbrace, Captain Seat, Black - Left (18" W) | 1 | 2 |
| 2 | | 7930323-NLA | NLA - Package, Locknut (1/4-20) | 25 | 1 |
| 3 | | 1057828-NLA | NLA - Strap, Pivot Link | 1 | 2 |
| 4 | | 1030949 | Screw, Hex Head (1/4-20 x 1-1/2") | 1 | 2 |
| 5 | | 1026900 | Washer (1/4 x 23/32 x 1/16") | 1 | 2 |
| 6 | | 1026154-NLA | NLA - Screw, Hex Head (1/4-20 x 2-1/4") | 1 | 1 |
| 7 | | 1024793-NLA | NLA - Screw, Hex Head (1/4-20 x 1-3/4") | 1 | 2 |
| 8 | | 1071604-NLA | NLA - Bumper, Nylon | 1 | 4 |
| 9 | | 1057382-NLA | NLA - Link, Pivot, Hemi, Black (16 - 18" W) | 1 | 2 |
| 10 | | 1031884-NLA | NLA - Bushing, Black or Color Chairs (9/32 x 3/8 x 11/32") | 1 | 2 |
| 11 | | 1071602-NLA | NLA - Pin, Anchor (Captain Seat) | 1 | 4 |
| 12 | | 1074142-NLA | NLA - Decal, Crossbrace | 1 | 2 |
| 13 | A | 1086800-NLA | NLA - Bracket, Push, Black | 1 | 2 |
| 14 | | 7930333 | Locknut (1/4-20) | 25 | 1 |
| 15 | | 1024796-NLA | NLA - Screw, Hex Head (1/4-20 x 2") | 1 | 1 |
| 16 | | 1028835-NLA | NLA - Cap, Hub, Black | 1 | 4 |
| 17 | | 7930363-NLA | NLA - Package, Locknut (5/16"-24) | 25 | 1 |
| 18 | | 1028491-NLA | NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.) | 1 | 4 |
| 19 | | 1056708-NLA | NLA - Plug, Pivot Tube | 1 | 4 |
| 20 | | 1038179-NLA | NLA - Screw, Hex Head (5/16-24 x 4-1/4") | 1 | 2 |
| 16-20 | B,C | 1057830-NLA | NLA - Hardware, Crossbrace Pivot Tube | 2 | 1 |
| 21 | | 1029501 | Plug Button, Black (1") | 1 | 2 |
| 22 | | 1025705-NLA | NLA - Screw, Hex Head (5/16-24 x 3-1/4") | 1 | 1 |
| 23 | | 1025704 | Washer, Flat (5/16 x 5/8 x 1/16") | 1 | 2 |
| 24 | | 1016263 | Spacer, Coved (3/4 x 1 x 1/2") | 1 | 4 |
| 25 | | 7930313 | Package, Washer, Nylon (5/16 x 7/8 x 1/8") | 25 | 1 |
| 26 | | 7930330-NLA | NLA - Locknut (5/16-24) | 25 | 1 |
| 27 | | 1035900 | Decal, Transportation | 1 | 1 |
| 28 | | 1119281-NLA | NLA - Plate, Name, Patent Numbers | 1 | 1 |
| 29 | D | 1086180-NLA | NLA - Hardware, Crossbrace Center Bolt Attaching | 1 | 1 |
| 30 | E | 1082683-NLA | NLA - Hardware, Crossbrace | 1 | 1 |

NOTE: A - Decals not included with frames. See page titled, "Packaging, Literature and Decals."

B - Not Shown.

C - Includes Items 16-20.

D - Includes Items 22-26.

E - Includes Items 2-8 and 10-28. Less items 1 and 9.