


S1263 One Piece Wooden Footboard and Hardware

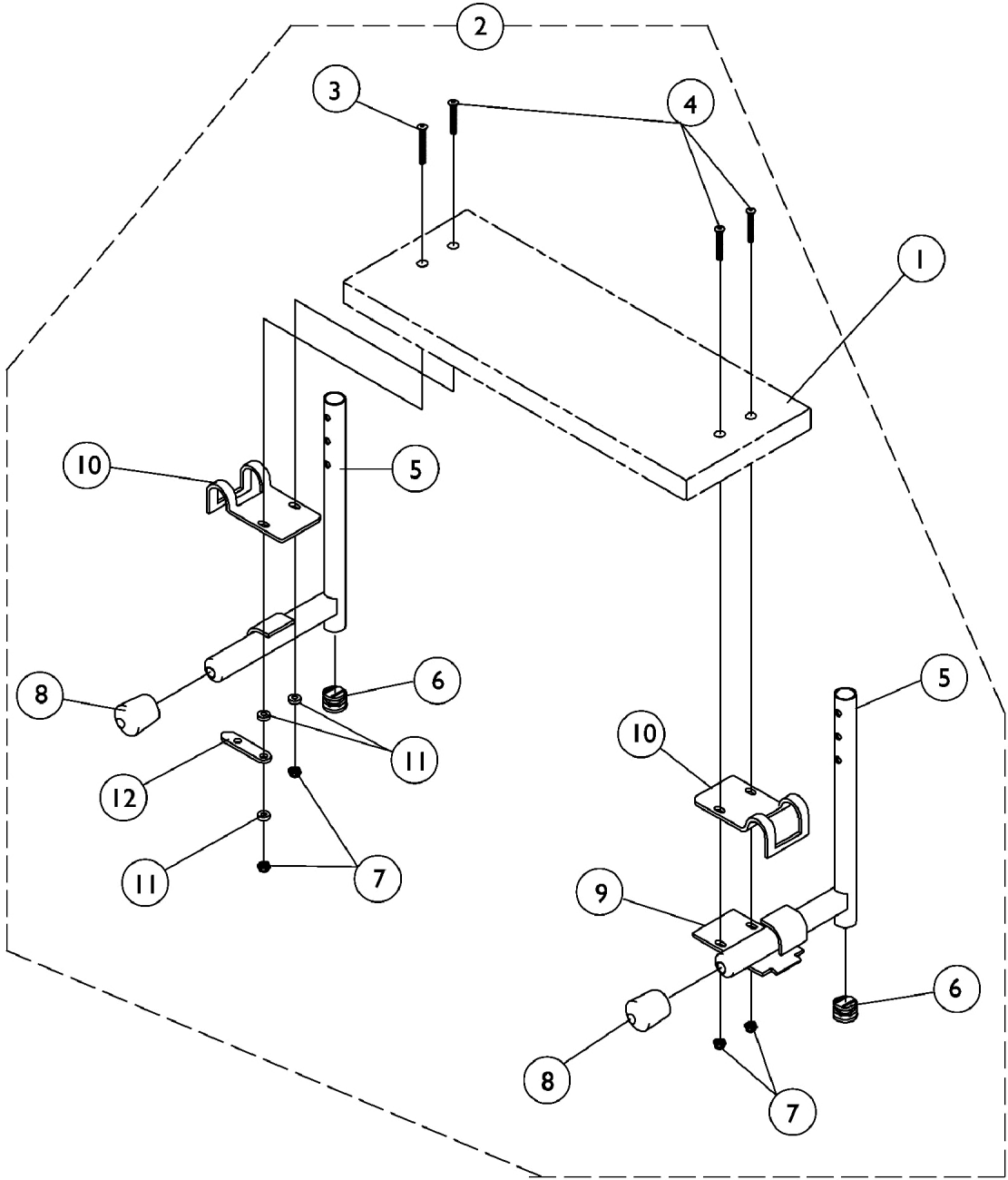

S1263 One Piece Wooden Footboard and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 One Piece Wooden Footboard without Hardware	1	1
2	B	See Chart	Heel Loop Hardware Packages (not shown)	1	1
3		1054573	Kit, Wooden Footboard Mounting Hardware	1	1
4			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/2")	1	1
5			NLA - Screw, Phillips Oval Head (#10-32 x 1-1/4")	1	3
6			Tube, Pivot/Slide, Chrome - Right (9-1/2" L)	1	1
7			Tube, Pivot/Slide, Chrome - Left (9-1/2" L)	1	1
8			Plug Button, Black (3/4")	1	2
9			Locknut (#10-32)	1	4
10			Cap, End, Tube, Black (7/8")	1	2
11			Lock, Pad Plate, Legrest, Black	1	1
12			Clamp, Black	1	2
			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
			NLA - Link, Locking, Chrome	1	1

NOTE: A - Not Shown.
B - Includes items 3-12

#1 One Piece Wooden Footboard without Hardware

NOTE: When selecting a wooden footboard for PW93ST or 70 Degree Tapered Footrest, order a 3" narrower footboard to accommodate the 1.5" footrest taper per side. Example: 17" wide chair will use a 14" wide chair footboard. S1263 option is not available for tapered footrest less than 17" wide.

Footrest-Footboard w/ Heel Loop Holes Part Number	Footrest-Footboard w/o Heel Loop Holes Part Number	Chair Width	Footboard Width	Footboard Depth
1034245-NLA	48247X003-NLA	14"	11-5/8"	6"
1034246-NLA	1031910-NLA	15"	12-1/2"	6"
1034247-NLA	48247X001-NLA	16"	13-3/4"	6"
1034248-NLA	1031911-NLA	17"	14-1/2"	6"
1034249-NLA	48247X000-NLA	18"	15-9/16"	6"
1034250-NLA	1031912-NLA	19"	16-1/2"	6"
1034251-NLA	1031913-NLA	20"	17-1/2"	6"
1034252-NLA	1031914-NLA	21"	18-1/2"	6"
1034253-NLA	1031915-NLA	22"	19-1/2"	6"
1034254-NLA	1031916-NLA	23"	20-1/2"	6"
1034255-NLA	1031917-NLA	24"	21-1/2"	6"

S1263 One Piece Wooden Footboard and Hardware

Heel Loop Hardware Packages (not shown)

(All hardware is included to attach heel loops to footboard.)

Chair Width	Part Number	Package Includes
14-15" Wide	1033904-NLA	2 each Heel Loops 4 each Screw, PPH (1/4-20 x 3") 4 each Spacer (1/4 x 11/32 x 1-3/4") 4 each Locknut (1/4-20)
16"-24" Wide	1033905-NLA	2 each Heel Loops 4 each Screw, PPH (1/4-20 x 3") 4 each Spacer (1/4 x 11/32 x 1-3/4") 4 each Locknut (1/4-20)