
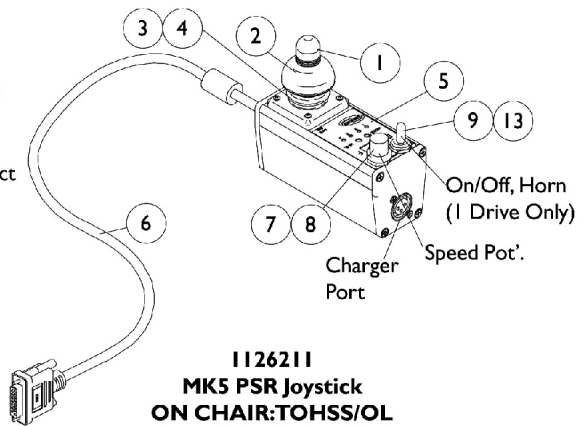
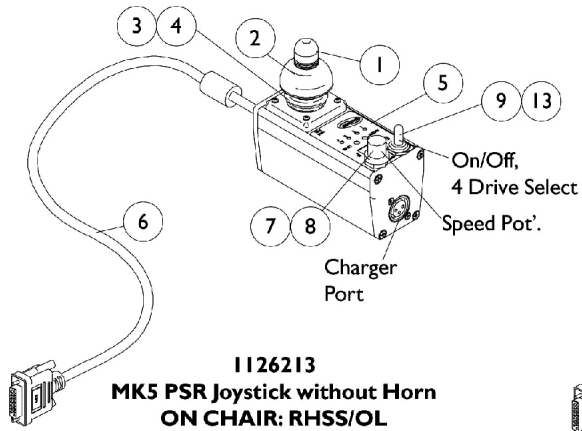
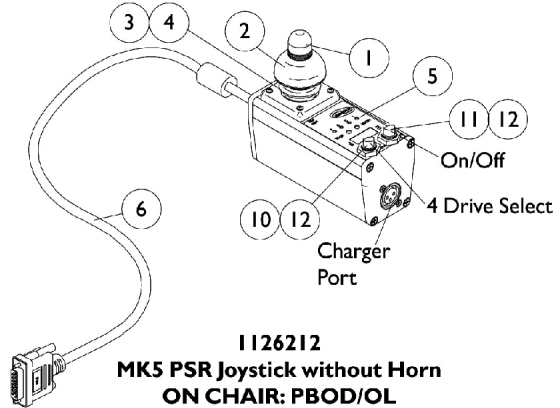
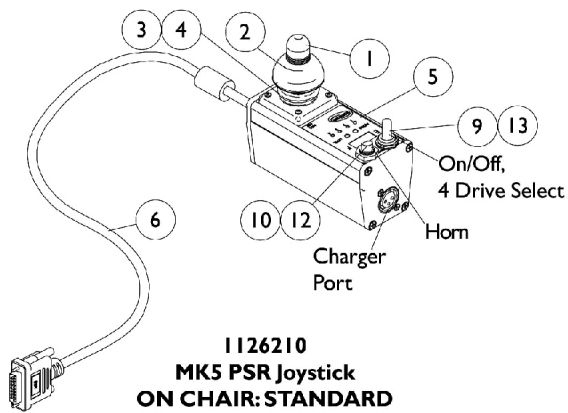


MK5 PSF/PSR Joysticks

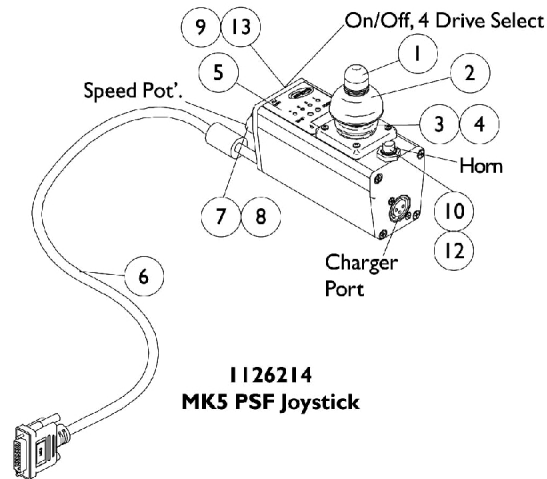

MK5 PSF/PSR Joysticks

Item Number	Description	PSF	PSR
See Pictorial	Joystick Assembly	1126214-NLA	1126210-NLA See Note(s) F, E, A
See Pictorial	Joystick Assembly	N/A	1126212-NLA See Note(s) F, E, B
See Pictorial	Joystick Assembly	N/A	1126213-NLA See Note(s) F, E, C
See Pictorial	Joystick Assembly	N/A	1126211-NLA See Note(s) F, E, D
1	Knob, Joystick	1040217	1040217
2	Skirt, Joystick	1040218	1040218
3	Inductive, Joystick	1036764-NLA	1036764-NLA
4	Boot, Joystick	50000X364-NLA	50000X364-NLA
5	Overlay, Joystick Decal	1126184-NLA See Note(s) G	1126184-NLA See Note(s) G
6	Cable Assembly	1126191-NLA	1126191-NLA
7	Potentiometer, Speed	1040225-NLA	1040225-NLA
8	Knob, Potentiometer	1025516-NLA	1025516-NLA
9	Switch, On/Off Momentary	1040185-NLA	1040185-NLA
10	Switch, Push Button SPST Momentary	1010047-NLA	1010047-NLA
11	Switch, Push Button SPST	N/A	1010048-NLA
12	Boot, Switch	1010049-NLA	1010049-NLA
13	Boot, Switch	1058464-NLA	1058464-NLA

NOTE: A - STANDARD: This PSR joystick includes a toggle on/off combined with a drive select and a push button horn. No speed selector.
B - PBOD: This PSR joystick includes push button on/off with a push button drive select. This gives you access to the four drives, but no horn and no speed select.
C - RHSS: This PSR joystick replaces the horn with a speed selector. This allows you to deviate from having a horn to having a speed selector dial, plus the drive selector to the four drives.
D - TOHSS: This PSR joystick includes a toggle on/off with horn and a speed select dial. You have access to only one drive.
E - OL: On/Off positioned on the left side of the PSR joystick and OR: On/Off positioned on right side of the PSR joystick.
F - PSR joystick configurations show the on/off switch on the left side. When the on/off switch is required on the right side, refer to part number 1126210 and order as a configured part.
G - Overlay is no longer available. The Joystick must be returned to Invacare for repair.