


Quad Link

Before 10/9/03

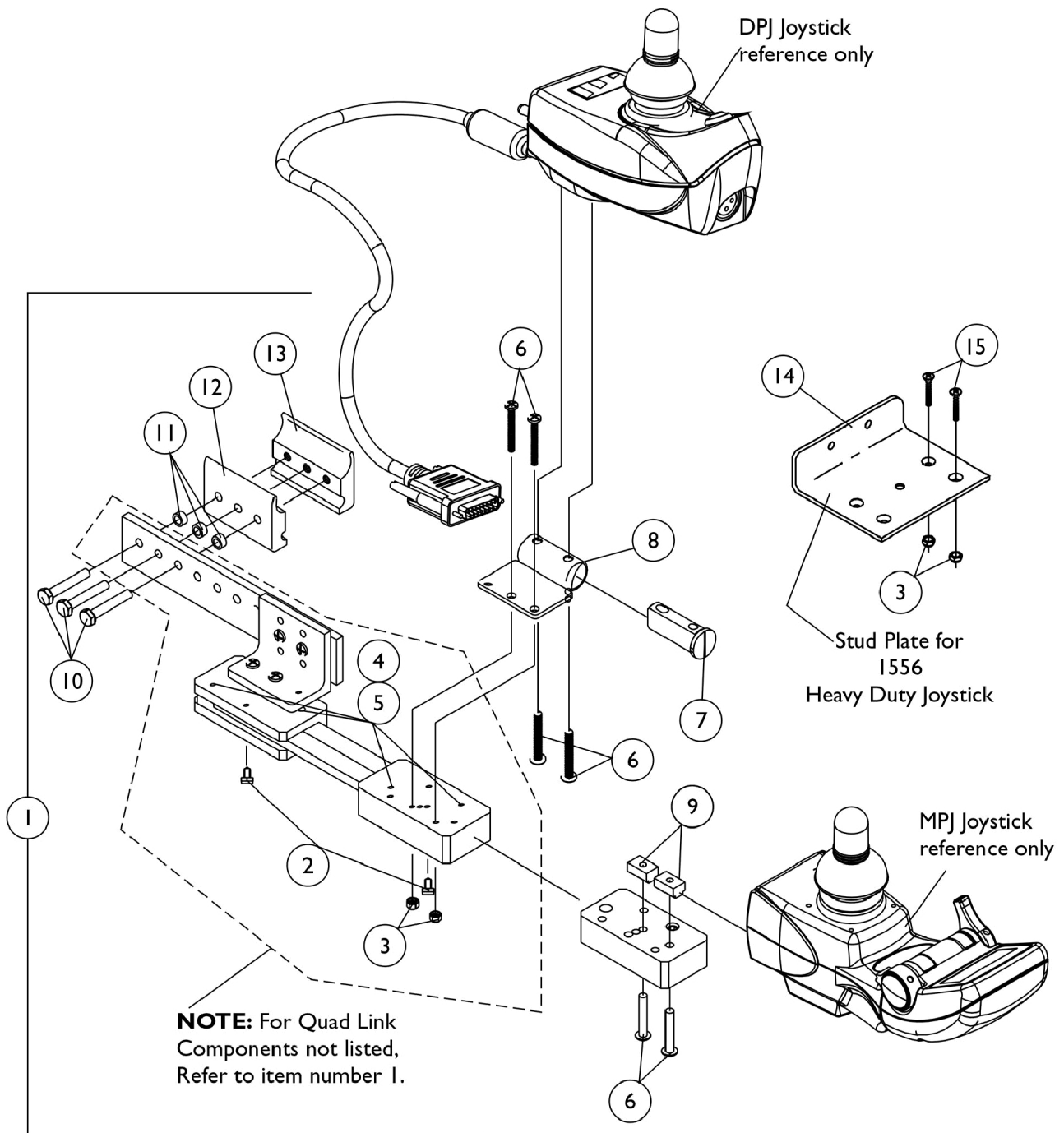

Quad Link Before 10/9/03

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1052402-NLA	NLA - Kit, Quad Link Assembly - Right	1	1
2			NLA - Ball Plunger Assembly with Lock Washer	2	1
3			Locknut (#10-32)	1	2
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
6			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	4
7			Spacer, Joystick Tube, Black	1	1
8			NLA - Stud Plate, Quad Link	1	1
10			Screw, Hex Head (1/4-20 x 1-1/2")	1	3
11			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3
12			Half Clamp, Non-Threaded Holes	1	1
13	A	1115372-NLA	NLA - Half Clamp, Threaded Holes	1	1
1			NLA - Kit, Quad Link Assembly - Left	1	1
2			NLA - Ball Plunger Assembly with Lock Washer	2	1
3			Locknut (#10-32)	1	2
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
6			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	4
7			Spacer, Joystick Tube, Black	1	1
8			NLA - Stud Plate, Quad Link	1	1
10			Screw, Hex Head (1/4-20 x 1-1/2")	1	3
11			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3
12	B	1117278-NLA	Half Clamp, Non-Threaded Holes	1	1
13			NLA - Half Clamp, Threaded Holes	1	1
1			NLA - Kit, Quad Link Assembly - Right	1	1
2			NLA - Ball Plunger Assembly with Lock Washer	2	1
3			Locknut (#10-32)	1	2
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
6			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
9			NLA - T-Nut (#10-32)	1	2
10			Screw, Hex Head (1/4-20 x 1-1/2")	1	3
11			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3
12	B	1117239-NLA	Half Clamp, Non-Threaded Holes	1	1
13			NLA - Half Clamp, Threaded Holes	1	1
1			NLA - Kit, Quad Link Assembly - Left	1	1
2			NLA - Ball Plunger Assembly with Lock Washer	2	1
3			Locknut (#10-32)	1	2
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
6			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
9			NLA - T-Nut (#10-32)	1	2
10			Screw, Hex Head (1/4-20 x 1-1/2")	1	3
11			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3
12			Half Clamp, Non-Threaded Holes	1	1
13			NLA - Half Clamp, Threaded Holes	1	1

Quad Link Before 10/9/03

Item Number	Note	Part Number	Description	Quantity Per			
				Package	Assembly		
1	C,D	1034703-NLA	NLA - Kit, Quad Link Assembly - Right	1	1		
2			NLA - Ball Plunger Assembly with Lock Washer	2	1		
3			Locknut (#10-32)	1	2		
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4		
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4		
10			Screw, Hex Head (1/4-20 x 1-1/2")	1	3		
11			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3		
12			Half Clamp, Non-Threaded Holes	1	1		
13			NLA - Half Clamp, Threaded Holes	1	1		
14			Plate, Stud, Quad Link For 1556 Heavy Duty Joystick	1	1		
15			Screw, Phillips Flat Head (#10-32 x 1-1/8")	1	2		
1			C	1114866-NLA	NLA - Kit, Quad Link Assembly - Left	1	1
2					NLA - Ball Plunger Assembly with Lock Washer	2	1
3					Locknut (#10-32)	1	2
4	NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1			4		
5	NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1			4		
10	Screw, Hex Head (1/4-20 x 1-1/2")	1			3		
11	NLA - Spacer (1/4 x 3/8 x 3/16")	1			3		
12	Half Clamp, Non-Threaded Holes	1			1		
13	NLA - Half Clamp, Threaded Holes	1			1		
14	Plate, Stud, Quad Link For 1556 Heavy Duty Joystick	1			1		
15	Screw, Phillips Flat Head (#10-32 x 1-1/8")	1	2				
2		1114863-NLA	NLA - Ball Plunger Assembly with Lock Washer	2	1		
NOTE: A - Includes the Quad Link, items 2-8, 10-13 and used with an SPJ-80 and DPJ joystick B - Includes the Quad Link, items 2-6, 9-13 and used with a MPJ joystick C - includes the Quad Link, items 2-5, 10-15 and used with a 1556 Heavy Duty joystick D - Quad Link Assembly is no longer available. Please see Quad Link Assembly (After 10/30/05) category and use new Quad Link design as a replacement.							