


MK5 TAC and Mounting Hardware

NOTE: MK5 TAC is used when ELRPW power articulating legrest are present

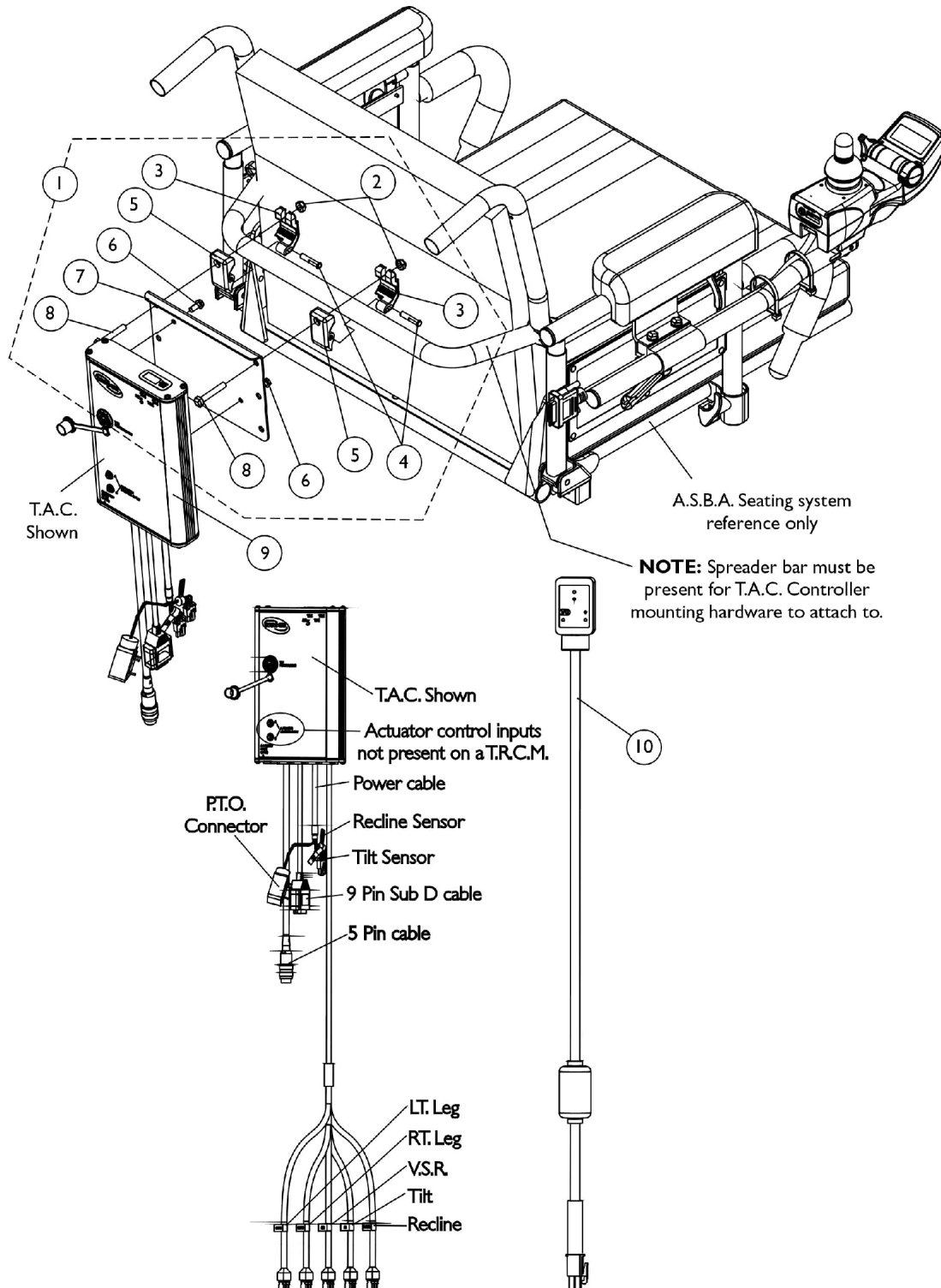

MK5 TAC and Mounting Hardware

NOTE: MK5 TAC is used when ELRPW power articulating legrest are present

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1121121-NLA	NLA - Hardware, MK5 T.A.C. Mounting, A.S.B.A. Seat	1	1
2		60275X000	Locknut (1/4-20)	1	2
3		1113407-NLA	NLA - Clamp, Outer	1	2
4		1113406-NLA	NLA - Pin, Hinge	1	2
5		1113408-NLA	NLA - Clamp, Inner	1	2
6		1079316-NLA	NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
7		1113601-NLA	NLA - Bracket, Controller Mounting, Black	1	1
8		1070759	Screw, Hex Head (1/4-20 x 1-1/2")	1	2
9	B	1118728-NLA	NLA - Controller, MK5 TAC	1	1
10		1119418	Cable, Auxiliary Power	1	1
	C	1118231-NLA	NLA - Cable, MK5 EX Controller Option	1	1

NOTE: A - Spreader bar must be present before attaching MK5 T.A.C. mounting hardware
 B - For MK5 EX controllers, option cable part number 1118231 must be present when using a MK5 T.A.C. controller
 C - Not Shown