


Front Rigging Non Hanger Hardware

When Front Rigging Is Excluded From Chair
Conventional and Contoura Seating

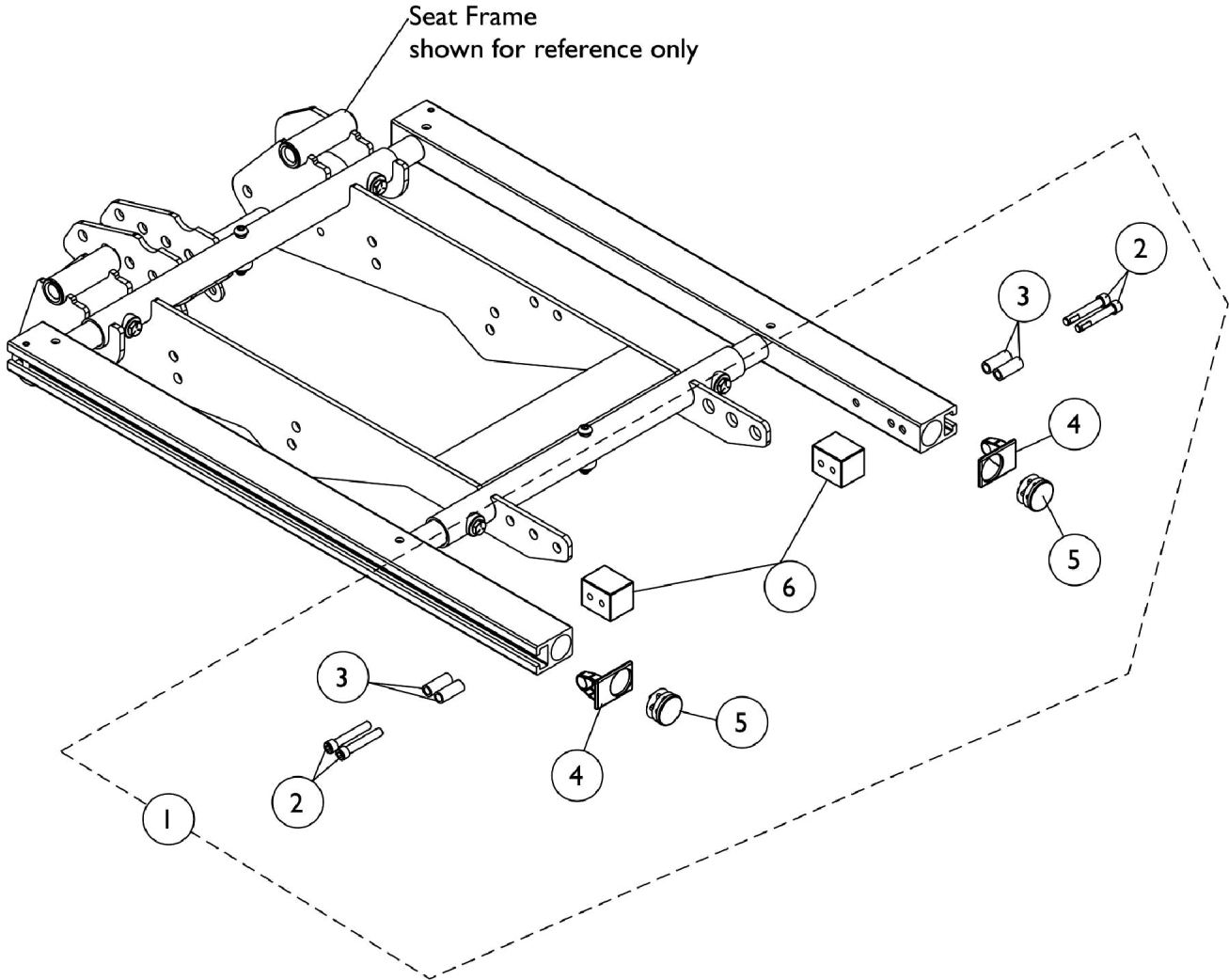

Front Rigging Non Hanger Hardware

When Front Rigging Is Excluded From Chair Conventional and Contoura Seating

| Item Number | Note | Part Number | Description | Quantity Per | |
|------------------------------|------|-------------|---------------------------------------------------|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | 1124723 | Kit, Side Rail Cap Hardware (Pair) | 2 | 1 |
| 2 | | | Screw, Socket Head Cap w/ Patch (1/4-20 x 1-1/2") | 1 | 4 |
| 3 | | | Spacer (1/4 x 3/8 x 29/32") | 1 | 4 |
| 4 | | | Cap, End, Side Rail, Black | 1 | 2 |
| 5 | | | Plug Button, Black (1") | 1 | 2 |
| 6 | | | Block, Threaded | 1 | 2 |
| NOTE: A - Includes items 2-6 | | | | | |