


MK5 TRECM, Shroud Cover, Mercury Free Switches and Mounting Hardware

Conventional Tilt & Tilt w/ BPO Option

(4/15/05-10/19/05)

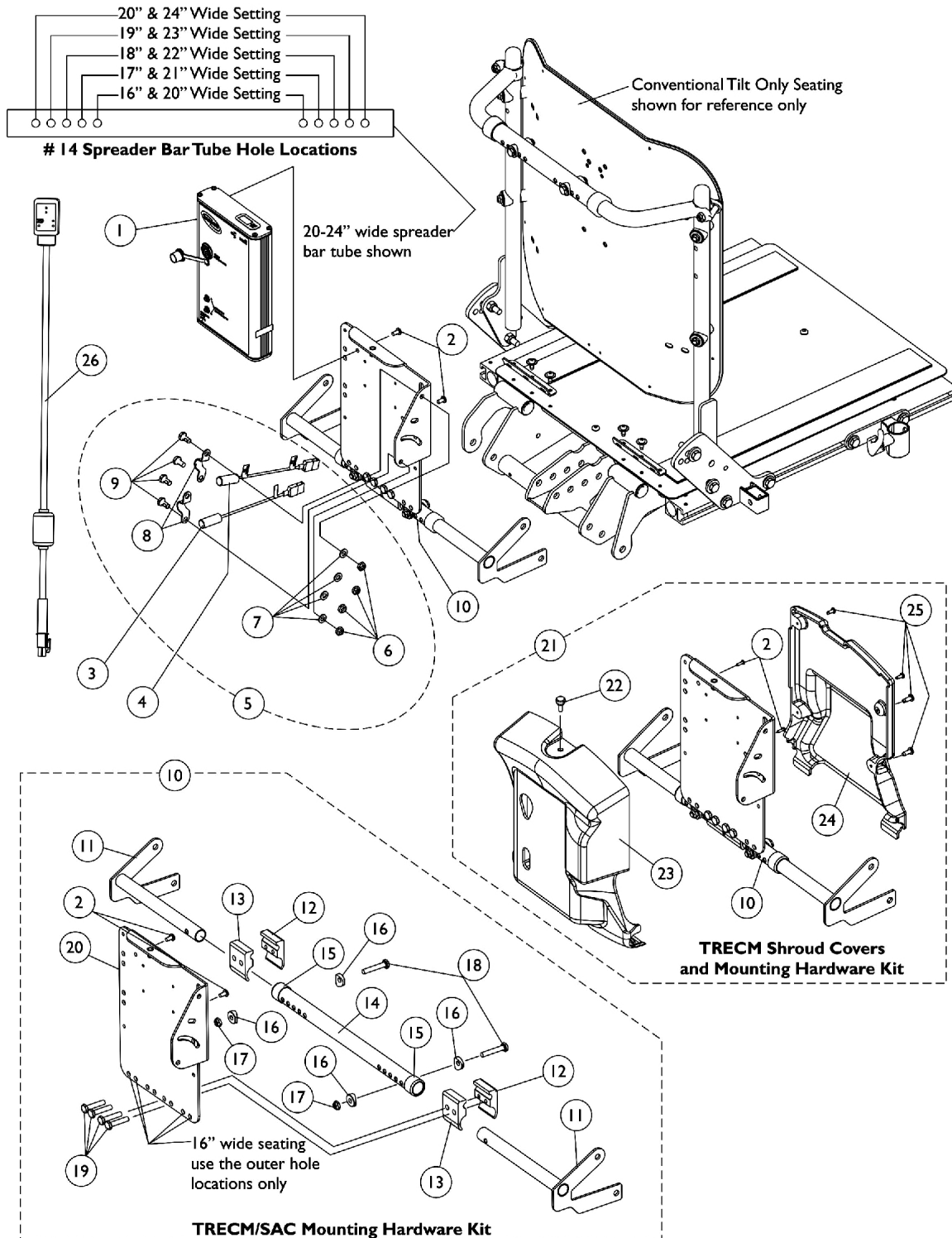

MK5 TRECM, Shroud Cover, Mercury Free Switches and Mounting Hardware

Conventional Tilt & Tilt w/ BPO Option (4/15/05-10/19/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1123263-NLA	NLA - Controller, MK5 TRECM	1	1
2		1079316-NLA	NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
3		1123268-NLA	NLA - Harness, Tilt Mercury Free Ball Switch	1	1
4		1080634-NLA	NLA - Harness, Drive Lockout Mercury Free Ball Switch	1	1
5	A	1125730-NLA	NLA - Kit, Drive Lockout/Recline Switch Mounting Hardware	2	1
6			Locknut (1/4-20)	1	4
7			Washer (1/4 x 1/2 x 1/16")	1	4
8			NLA - Bracket, Switch Mounting, Black	1	2
9			Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	4
10	B,D	1128362-NLA	NLA - Kit, TRECM/SAC Mounting Hardware 16-20" Wide For Conventional Tilt Only	1	1
2			NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
2			Screw, Phillips Pan Head (#4-40 x 3/8")	1	4
11			Bracket, Spreader Bar, Black - Right	1	1
11			Bracket, Spreader Bar, Black - Left	1	1
12			NLA - Half Clamp, Threaded Holes	1	2
13			Half Clamp, Non-Threaded Holes	1	2
14			Tube, Spreader Bar, Black (16 - 20" W)	1	1
15			Collar, Anti-Rattle (7/8")	1	2
16			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
17			Locknut (1/4-20)	1	2
18			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	2
19			Screw, Hex Head (1/4-20 x 1-1/4")	1	4
20			NLA - TRECM/SAC/TAC/MK6i Connector Shroud Mounting Tray Assembly	1	1
10	B,D	1128361-NLA	NLA - Kit, TRECM/SAC Mounting Hardware 20-24" Wide For Conventional Tilt Only	1	1
2			NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
2			Screw, Phillips Pan Head (#4-40 x 3/8")	1	4
11			Bracket, Spreader Bar, Black - Right	1	1
11			Bracket, Spreader Bar, Black - Left	1	1
12			NLA - Half Clamp, Threaded Holes	1	2
13			Half Clamp, Non-Threaded Holes	1	2
14			Tube, Spreader Bar, Black (20 - 24" W)	1	1
15			Collar, Anti-Rattle (7/8")	1	2
16			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
17			Locknut (1/4-20)	1	2
18			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	2
19			Screw, Hex Head (1/4-20 x 1-1/4")	1	4

MK5 TRECM, Shroud Cover, Mercury Free Switches and Mounting Hardware

Conventional Tilt & Tilt w/ BPO Option (4/15/05-10/19/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
20			NLA - TRECM/SAC/TAC/MK6i Connector Shroud Mounting Tray Assembly	1	1
21	C	1128364	Kit, TRECM Shrouds and Mounting Hardware 16-20" Wide For Conventional Tilt Only	1	1
2			NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
11			Bracket, Spreader Bar, Black - Right	1	1
11			Bracket, Spreader Bar, Black - Left	1	1
12			NLA - Half Clamp, Threaded Holes	1	2
13			Half Clamp, Non-Threaded Holes	1	2
14			Tube, Spreader Bar, Black (16 - 20" W)	1	1
15			Collar, Anti-Rattle (7/8")	1	2
16			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
17			Locknut (1/4-20)	1	2
18			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	2
19			Screw, Hex Head (1/4-20 x 1-1/4")	1	4
20			NLA - TRECM/SAC/TAC/MK6i Connector Shroud Mounting Tray Assembly	1	1
22			Screw, Thumb (#10-32 x 1/2")	1	1
23			NLA - Shroud, Cover	1	1
24			Shroud, Back	1	1
25			Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	4
21	C	1128363	Kit, TRECM Shrouds and Mounting Hardware 20-24" Wide For Conventional Tilt Only	1	1
2			NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
11			Bracket, Spreader Bar, Black - Right	1	1
11			Bracket, Spreader Bar, Black - Left	1	1
12			NLA - Half Clamp, Threaded Holes	1	2
13			Half Clamp, Non-Threaded Holes	1	2
14			Tube, Spreader Bar, Black (20 - 24" W)	1	1
15			Collar, Anti-Rattle (7/8")	1	2
16			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
17			Locknut (1/4-20)	1	2
18			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	2
19			Screw, Hex Head (1/4-20 x 1-1/4")	1	4
20			NLA - TRECM/SAC/TAC/MK6i Connector Shroud Mounting Tray Assembly	1	1
22			Screw, Thumb (#10-32 x 1/2")	1	1
23			NLA - Shroud, Cover	1	1
24			Shroud, Back	1	1
25			Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	4
26		1119418	Cable, Auxiliary Power	1	1

MK5 TRECM, Shroud Cover, Mercury Free Switches and Mounting Hardware Conventional Tilt & Tilt w/ BPO Option (4/15/05-10/19/05)

NOTE:	A - Includes items 6-10 B - Includes items 2 and 11-20 C - Includes items 2, 11-20 and 22-25 D - Item 2 includes both attaching hardware screws for the MK5 TRECM and the MK5 SAC controller. Part number 1052041 screw is used to assemble a MK5 SAC to item 20 Mounting Tray Assembly.
-------	---