


Arms - Excel

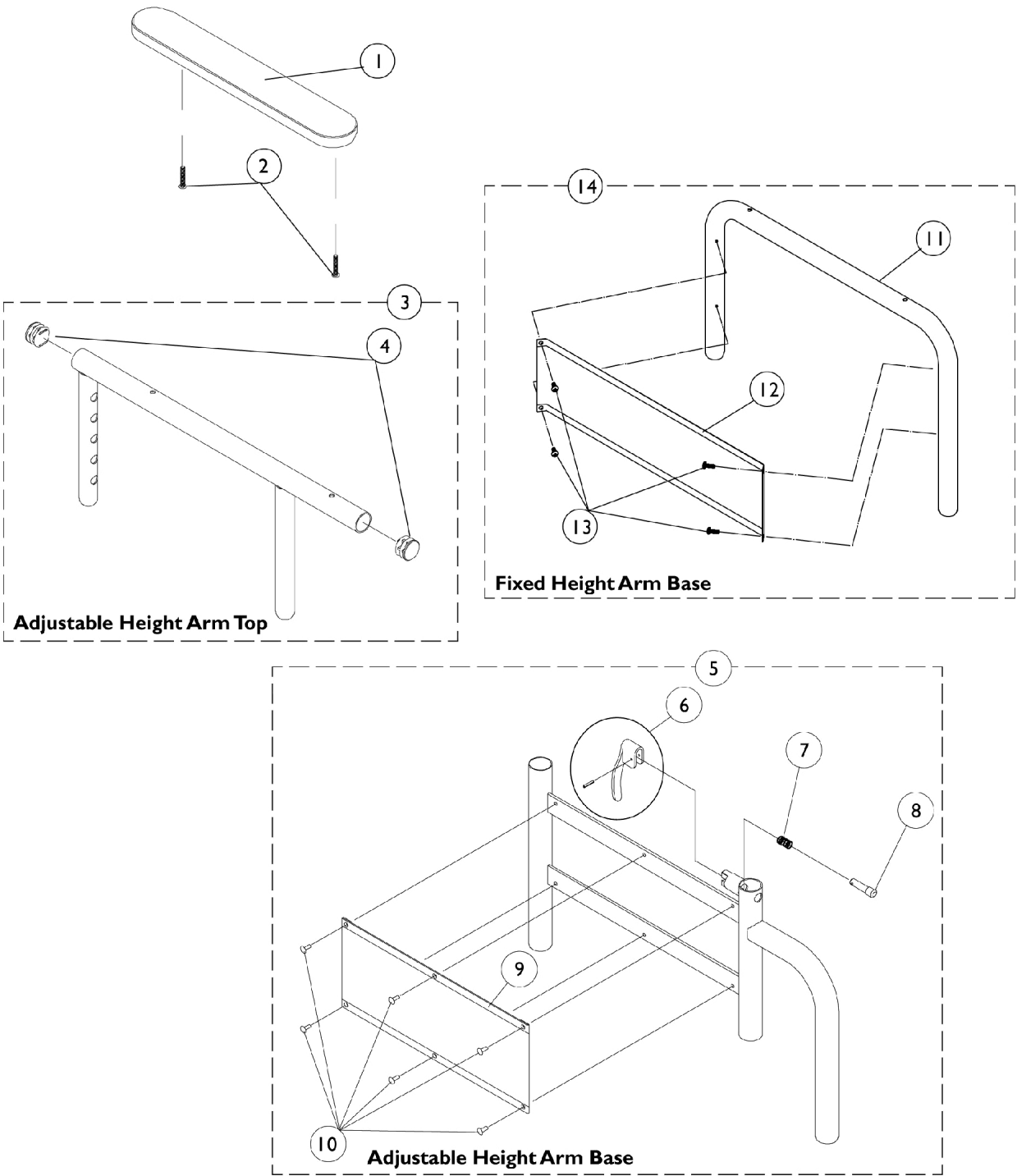

Arms - Excel

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881015932-NL A	NLA - Pad, Arm, Vinyl, Full Length - Right (14" L x 2" W)	1	1
1	A	8881015933-NL A	NLA - Pad, Arm, Vinyl, Full Length - Left (14" L x 2" W)	1	1
1	B,D	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
1		1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
1	A	8881011811-NL A	NLA - Pad, Arm, Vinyl, Desk Length (9" L x 2" W)	1	1
1	B,D	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
1		1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
2		1055949	Package, Screw, Phillips Pan Head (#10-32 x 1-3/8")	2	1
2		1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
3		1041522	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
3		1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5	C	8881057741-NL A	NLA - Arm Base, Adjustable Height, Color - Right	1	1
5	C	8881057742-NL A	NLA - Arm Base, Adjustable Height, Color - Left	1	1
6		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
7		47471X074	Spring, Arm Lock	1	1
8		1028594	Pin, Arm Lock, Black	1	1
9		1064130-NLA	NLA - Guard, Clothing	1	1
10		1028645	Rivet	1	6
11	C	8881057723	Desk Length Arm Tube (Color), Right	1	1
11	C	8881057724-NL A	NLA - Tube, Arm, Desk Length, Color - Left	1	1
11	C	8881057715-NL A	NLA - Tube, Arm, Full Length, Color - Right	1	1
11	C	8881057716-NL A	NLA - Tube, Arm, Full Length, Color - Left	1	1
12		1035472-NLA	NLA - Guard, Clothing, Desk Length	1	1
12		1044572-NLA	NLA - Guard, Clothing, Full Length (16 - 19" D)	1	1
13		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
14	C	8881057729-NL A	NLA - Arm Assembly, Desk Length, Color - Right	1	1
14	C	8881057730-NL A	NLA - Arm Assembly, Desk Length, Color - Left	1	1
14	C	8881057719-NL A	NLA - Arm, Full Length without Armpad, Color - Right	1	1

Arms - Excel

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
14	C	8881057720-NL A	NLA - Arm, Full Length without Armpad, Color - Left	1	1
NOTE: A - Specify upholstery color when ordering. See page titled, "Frame Finishes, Touch-Up Paint and Upholstery Options." B - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in parts with screws for chairs manufactured before 1/3/02. C - Specify frame finish when ordering. See page titled, "Frame Finishes, Touch-Up Paint and Upholstery Options." D - After 7/9/07 Invacare recommends ordering armpads in pairs to assure uniform appearance on chair					