



Drive Wheel and Hub Mount (14 x 3")

Optional on Chair
(As Of 10/15/99)

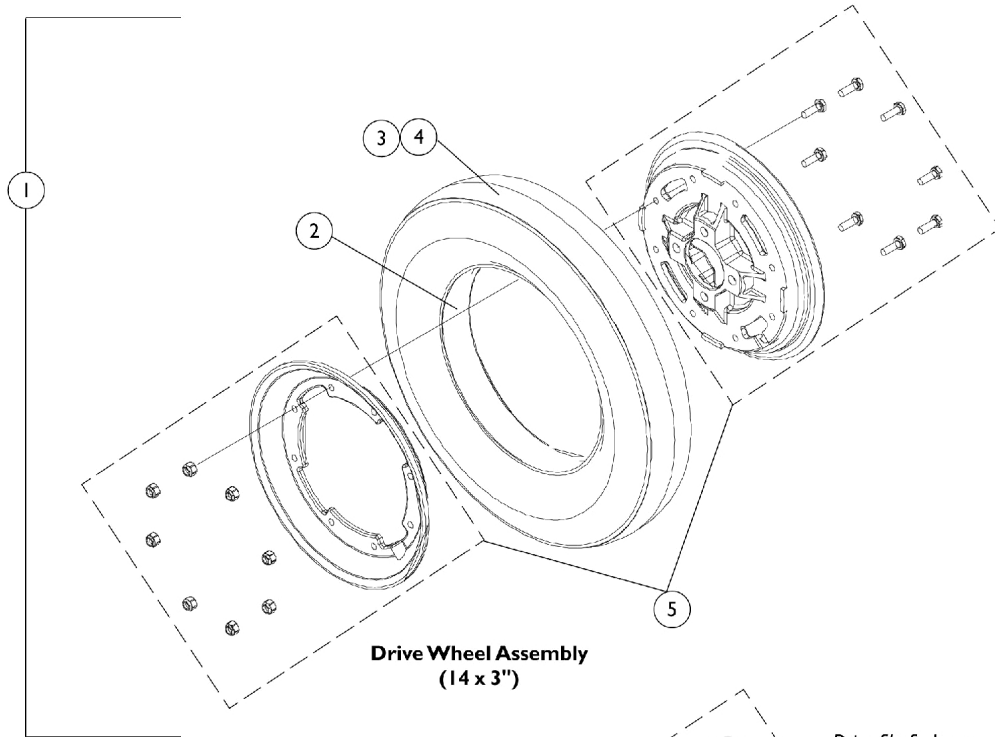

**Drive Wheel Assembly
(14 x 3")**

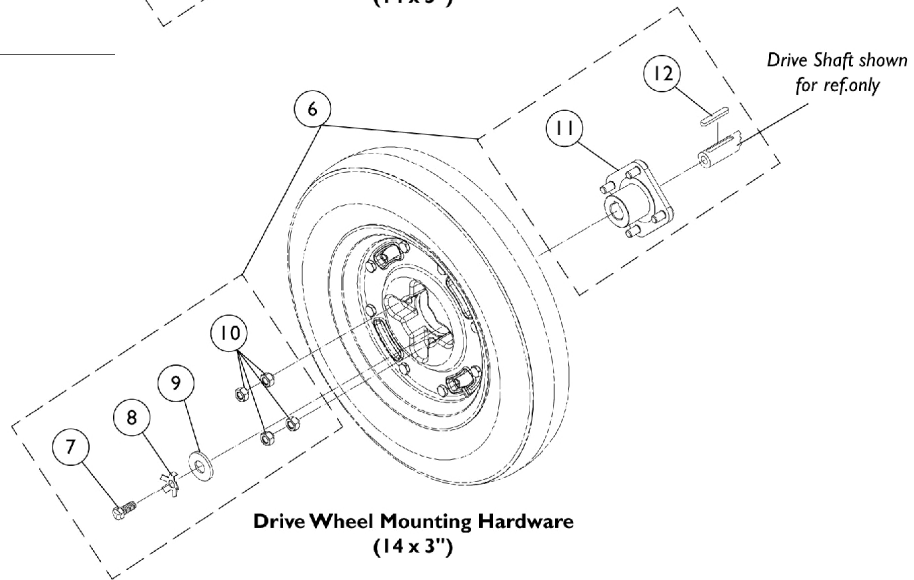

**Drive Wheel Mounting Hardware
(14 x 3")**

Drive Wheel and Hub Mount (14 x 3")

Optional on Chair (As Of 10/15/99)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1090649-NLA	NLA - Pneumatic Wheel Drive Assembly (14" x 3")	1	1
1		1090650-NLA	NLA - Drive Wheel Assembly w/ Flat Free Insert, Grey Tire/Black Rim (14 x 3")	1	1
2		1101754	Tire w/ Foam Fill Insert, Grey (14" x 3")	1	1
3		1039652	Tire, Pneumatic, Grey (14" x 3")	1	1
4		1039651	Tube, Inner, Black (14 x 3")	1	1
5	A	1090651-NLA	NLA - Kit, 8" Rim w/ Hardware (14" x 3" Wheel)	1	1
6	B	1090648-NLA	NLA - Hardware, Drive Wheel Mounting w/ Instruction Sheet (14 x 3")	1	1
7		7930322-NLA	NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm)	25	1
8		1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	2	1
9		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	1
10		1024444	Locknut (5/16-24)	1	4
11		1086681-NLA	NLA - Hub, Wheel (14" x 3")	1	1
12		1023264	Key (6mm x 6mm x 40mm)	1	1

NOTE: A - Hardware consists of: (8) ea. Screw, Hex Head Cap (1/4-20 x 3/4") and (8) ea. Locknut, High Profile (1/4-20").
B - Includes Items 7-12 and Instruction Sheet.