
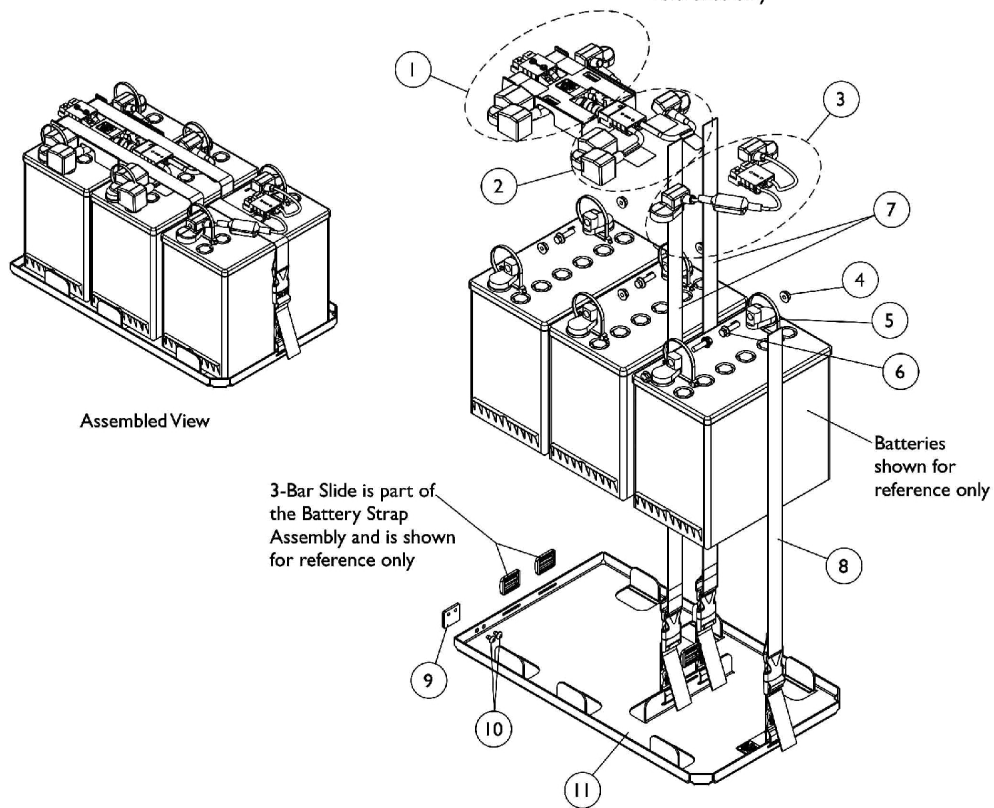


Battery Tray Assembly and Wire Harnesses For Vent Tray (3) 22NF Batteries

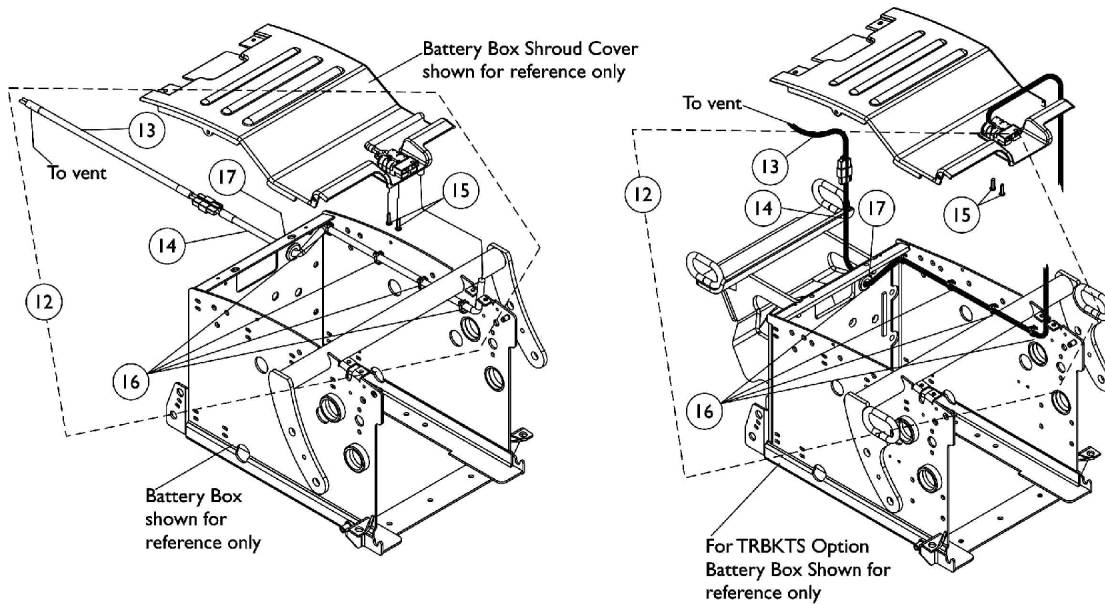

Battery Tray Assembly and Wire Harnesses For Vent Tray (3) 22NF Batteries

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|---------------|--|--------------|----------|
| | | | | Package | Assembly |
| 1 | A,B | 1120363 | Battery Wire Harness with Terminal Hardware, 22NF - Rear | 1 | 1 |
| 2 | A,B | 1120362 | Battery Wire Harness with Terminal Hardware, 22NF - Front | 1 | 1 |
| 3 | A,B | 1121143 | Harness, Battery Wire with Terminal Hardware, 22NF Vent | 1 | 1 |
| 4 | | 16721X000 | Locknut (1/4-20) | 1 | 6 |
| 5 | | 1095502-NLA | NLA - Package, Tie Wrap (11-1/2" L) | 10 | 1 |
| 6 | | 1096799-NLA | NLA - Screw, Hex Flange Whiz Lock (1/4-20 x 7/8") | 1 | 4 |
| 7 | | 1121124-NLA | NLA - Battery Strap Assembly, 22NF and GP24 | 1 | 2 |
| 8 | | 1121125-NLA | NLA - Battery Strap Assembly For Vent, 22NF | 1 | 1 |
| 9 | | 1117186-NLA | NLA - Bracket, Battery Tray Stop, Black | 1 | 1 |
| 10 | | 90017X000-NLA | NLA - Screw, Phillips Pan Head (#10-32 x 5/16") | 1 | 2 |
| 11 | C | 1121142-NLA | NLA - Battery Tray Assembly, TDX 22NF For Vent | 1 | 1 |
| 12 | D | 1121029-NLA | NLA - Kit, Vent Battery Harness Assembly with Hardware | 1 | 1 |
| 13 | | | Harness, Vent Battery | 1 | 1 |
| 14 | | | NLA - Harness, Vent Battery Extension | 1 | 1 |
| 15 | | | Screw, Phillips Pan Head (#8-16 x 5/8") | 1 | 2 |
| 16 | | | Tie Wrap (5-5/8" L) | 1 | 4 |
| 17 | | | NLA - Grommet, Rubber | 1 | 1 |
| 12 | D | 1139531-NLA | NLA - Kit, Vent Battery Harness Assembly with Hardware For TRBKTS Option | 1 | 1 |
| 13 | | | Harness, Vent Battery | 1 | 1 |
| 14 | | | Harness, Vent Battery Extension, TRBKTS Option | 1 | 1 |
| 15 | | | Screw, Phillips Pan Head (#8-16 x 5/8") | 1 | 2 |
| 16 | | | Tie Wrap (5-5/8" L) | 1 | 4 |
| 17 | | | NLA - Grommet, Rubber For TRBKTS Option | 1 | 1 |
| 16 | | 1095503-NLA | NLA - Package, Tie Wrap (5-5/8" L) | 10 | 1 |
| 17 | | 1055747-NLA | NLA - Grommet, Rubber | 1 | 1 |
| 17 | | 1137236-NLA | NLA - Grommet, Rubber For TRBKTS Option | 1 | 1 |

NOTE: A - Individual components are not available for the battery wire harnesses
B - Includes items 4-6
C - Includes items 7-10
D - Includes items 13-17