



Arms - Van Seat and Hardware

16-20" Wide Models (After 8/31/02)

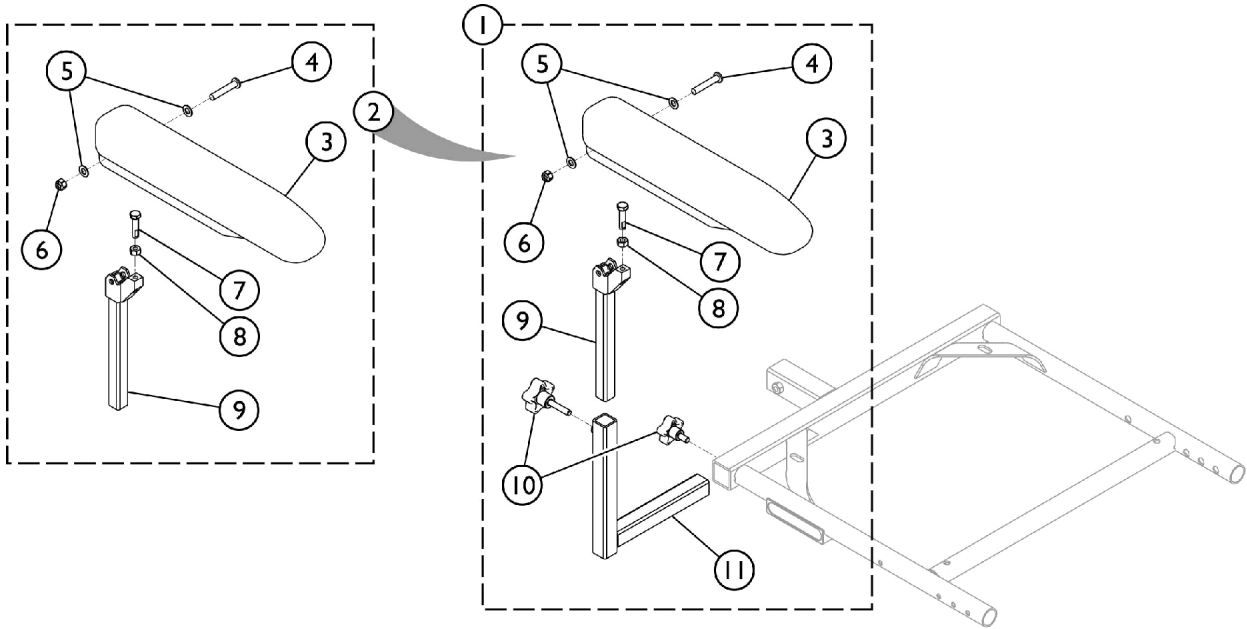

Adjustable Height/Adjustable Width Arm Assembly

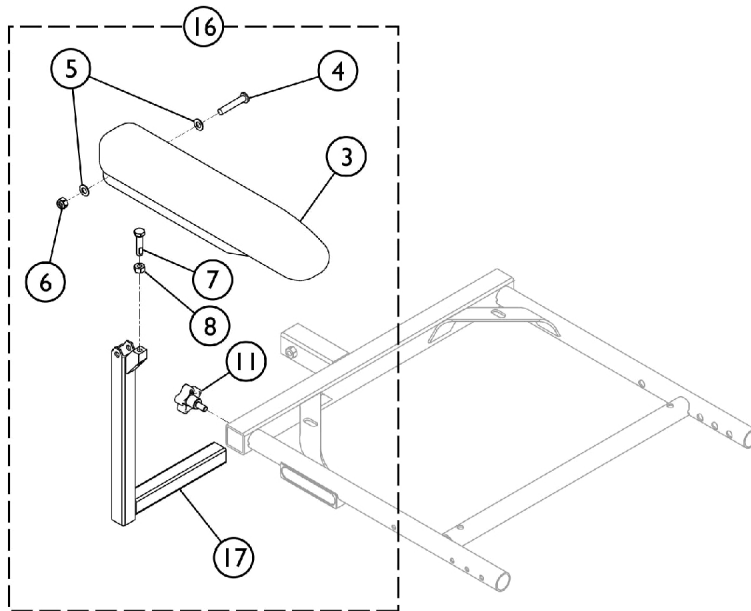

Fixed Height/Adjustable Width Arm Assembly

Arms - Van Seat and Hardware

16-20" Wide Models (After 8/31/02)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1142474	Arm Assembly, Adj. Height, Full Length, Black - Right	1	1
1	A	1142475	Arm Assembly, Adj. Height, Full Length, Black - Left	1	1
1	A	1142492	Arm Assembly, Adj. Height, Desk Length, Black - Left	1	1
1	A	1142493	Arm Assembly, Adj. Height, Desk Length, Black - Right	1	1
2	B	1142487	Upper Arm Assembly, Desk Length (Right/Left)	1	1
3			Arm Pad, Desk Length, Black Urethane (12-1/4" L x 2-5/8" W)	1	1
4			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	1
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Locknut (5/16-18)	1	1
7			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	1
8			NLA - Nut, Hex (5/16-18)	1	1
9			Tube, Arm, Upper Adjustable Height, Black	1	1
2	B	1140426	Upper Arm Assembly, Full Length (Right/Left)	1	1
3			Arm Pad, Full Length, Urethane, Black (15" L x 2-5/8" W)	1	1
4			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	1
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Locknut (5/16-18)	1	1
7			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	1
8			NLA - Nut, Hex (5/16-18)	1	1
9			Tube, Arm, Upper Adjustable Height, Black	1	1
3		1139923	Arm Pad, Desk Length, Black Urethane (12-1/4" L x 2-5/8" W)	1	1
3		1139924	Arm Pad, Full Length, Urethane, Black (15" L x 2-5/8" W)	1	1
10		1114910	Knob, Adjustment, Black (5/16-18 x 3/4")	1	2
11		1113646	Tube, Arm, Lower Adjustable Width, Black - Right	1	1
11		1113647	Tube, Arm, Lower Adjustable Width, Black - Left	1	1
16	C	1142488	Arm Assembly, Desk Length Fixed Height - Right	1	1
3			Arm Pad, Desk Length, Black Urethane (12-1/4" L x 2-5/8" W)	1	1
4			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	1
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Locknut (5/16-18)	1	1
7			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	1
8			NLA - Nut, Hex (5/16-18)	1	1
10			Knob, Adjustment, Black (5/16-18 x 3/4")	1	1
17			NLA - Tube, Arm, Fixed Height, Black - Right	1	1
16	C	1142489	Arm Assembly, Desk Length Fixed Height - Left	1	1
2			Arm Pad, Desk Length, Black Urethane (12-1/4" L x 2-5/8" W)	1	1
3			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	1
4			Washer, Flat (5/16 x 5/8 x 1/16")	1	2

Arms - Van Seat and Hardware 16-20" Wide Models (After 8/31/02)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
5			Locknut (5/16-18)	1	1
6			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	1
7			NLA - Nut, Hex (5/16-18)	1	1
10			Knob, Adjustment, Black (5/16-18 x 3/4")	1	1
17			NLA - Tube, Arm, Fixed Height, Black - Left	1	1
16	C	1142483	Arm Assembly, Full Length Fixed Height - Right	1	1
3			Arm Pad, Full Length, Urethane, Black (15" L x 2-5/8" W)	1	1
4			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	1
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Locknut (5/16-18)	1	1
7			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	1
8			NLA - Nut, Hex (5/16-18)	1	1
10			Knob, Adjustment, Black (5/16-18 x 3/4")	1	1
17			NLA - Tube, Arm, Fixed Height, Black - Right	1	1
16	C	1142486	Arm Assembly, Full Length Fixed Height - Left	1	1
3			Arm Pad, Full Length, Urethane, Black (15" L x 2-5/8" W)	1	1
4			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	1
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Locknut (5/16-18)	1	1
7			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	1
8			NLA - Nut, Hex (5/16-18)	1	1
10			Knob, Adjustment, Black (5/16-18 x 3/4")	1	1
17			NLA - Tube, Arm, Fixed Height, Black - Left	1	1
NOTE: A - Includes items 2 - 11. B - Includes items 3 - 9. C - Includes items 3 - 8, 10, and item 17.					