


S.F.T. Switch w/ Drive Lockout Built In (After 1/17/99) Basic Recline 4XP

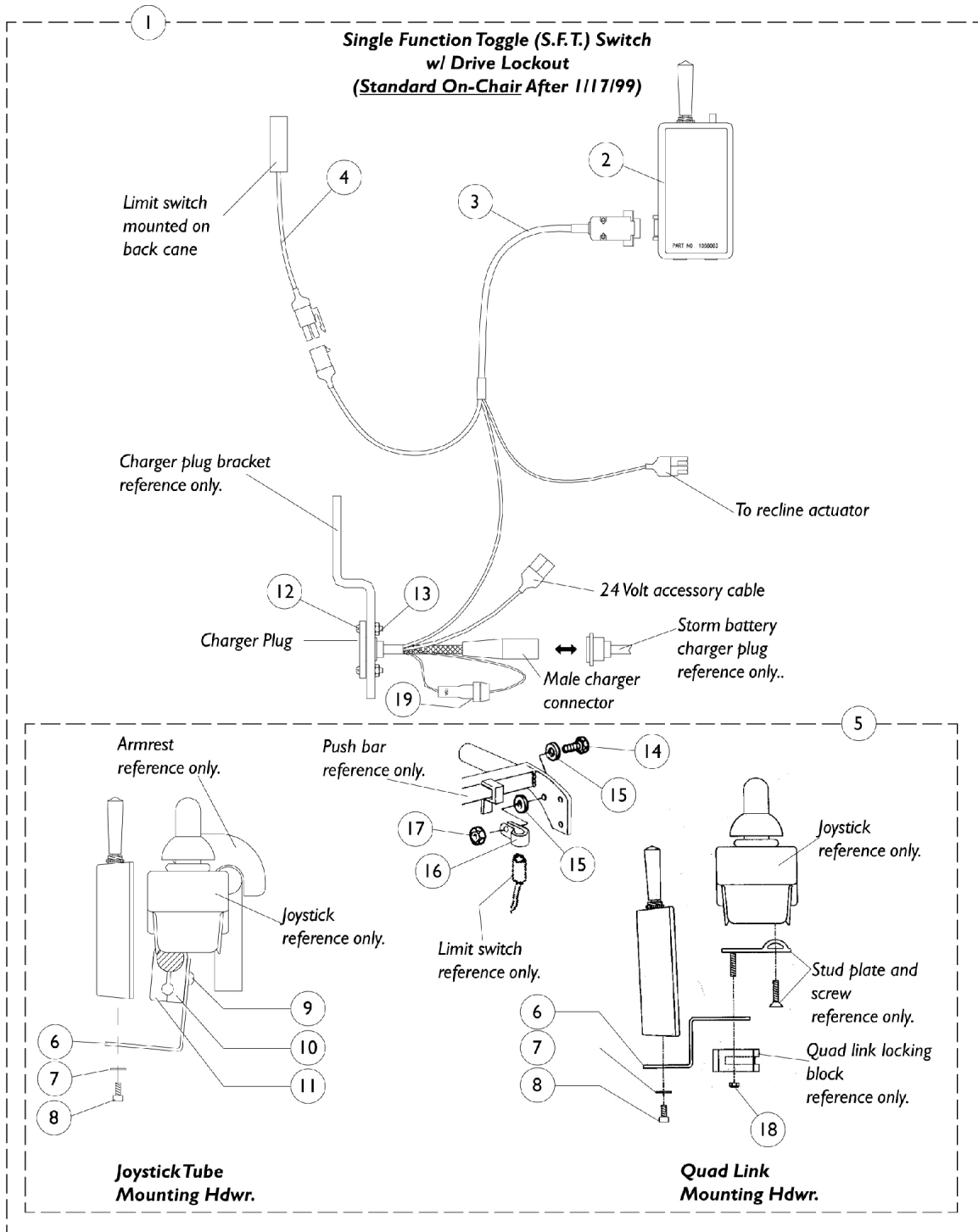

S.F.T. Switch w/ Drive Lockout Built In (After 1/17/99) Basic Recline 4XP

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1068182-NLA	NLA - Switch, S.F.T. with Drive Lockout Retrofit Kit	1	1
2	A,C	1068083-NLA	NLA - Switch, S.F.T. w/ Drive Lockout Assembly	1	1
3		1068178-NLA	NLA - Harness, Wiring, S.F.T. Switch w/ Drive Lockout	1	1
4	D	0018600-NLA	NLA - Switch, Limit - Molex Style	1	1
5		1068177-NLA	NLA - Kit, Hardware, Attaching, S.F.T. Switch w/ Drive Lockout Retrofit	1	1
	B	1077196-NLA	NLA - Instruction Sheet	1	1
6	F	1068171-NLA	NLA - Bracket, Joystick Mounting, Black	1	1
6	E	1068087-NLA	NLA - Bracket, Quad Link Mounting, Black	1	1
7		0224806-NLA	NLA - Washer (#10)	1	2
8		0067006-NLA	NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	2
9		0051206-NLA	NLA - Screw, Machine, Hex Socket, Button Head (1/4-20 x 1-1/4)	1	2
10		1049586	Half Clamp, Non-Threaded Holes	1	1
11		1049584-NLA	NLA - Half Clamp, Threaded Holes	1	1
12		313329-NLA	NLA - Screw, Phillips Pan Head (#6-32 x 1/2")	1	2
13		0143205-NLA	NLA - Locknut, Low Profile (#6-32)	1	2
14		0139306-NLA	NLA - Screw, Socket Button Head (#10-32 x 3/4")	1	1
15		1077143	Lockwasher, External Tooth (#10)	1	2
16		0062800-NLA	NLA - Bracket, Cable/Wire Mounting, Black	1	1
17		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
18		317601-NLA	NLA - Locknut, Low Profile (#10-32)	1	2
19		98000X055	Fuse, Automotive (15 AMP)	1	1
	B	0154500-NLA	NLA - Cable Tie, Black (4")	1	6
	B	0152400-NLA	NLA - Tie Wrap, Black (8" L)	1	6
NOTE: A - New style S.F.T. switch has the drive lockout feature built in. B - Not Shown. C - After 1/17/99 D - Molex Style E - Use with Quad Link before 4/07/06 F - Use with Joystick Tube and Quad Link (QLA) after 4/06/06					