


MAX Controller 5.7 S/N start at TE4000 & 4-Way Switch Tilt/Recline & Recline Only

NOTE: MKIVT.R.C.M. replaced the 5.7 MAX controller 6/15/98.

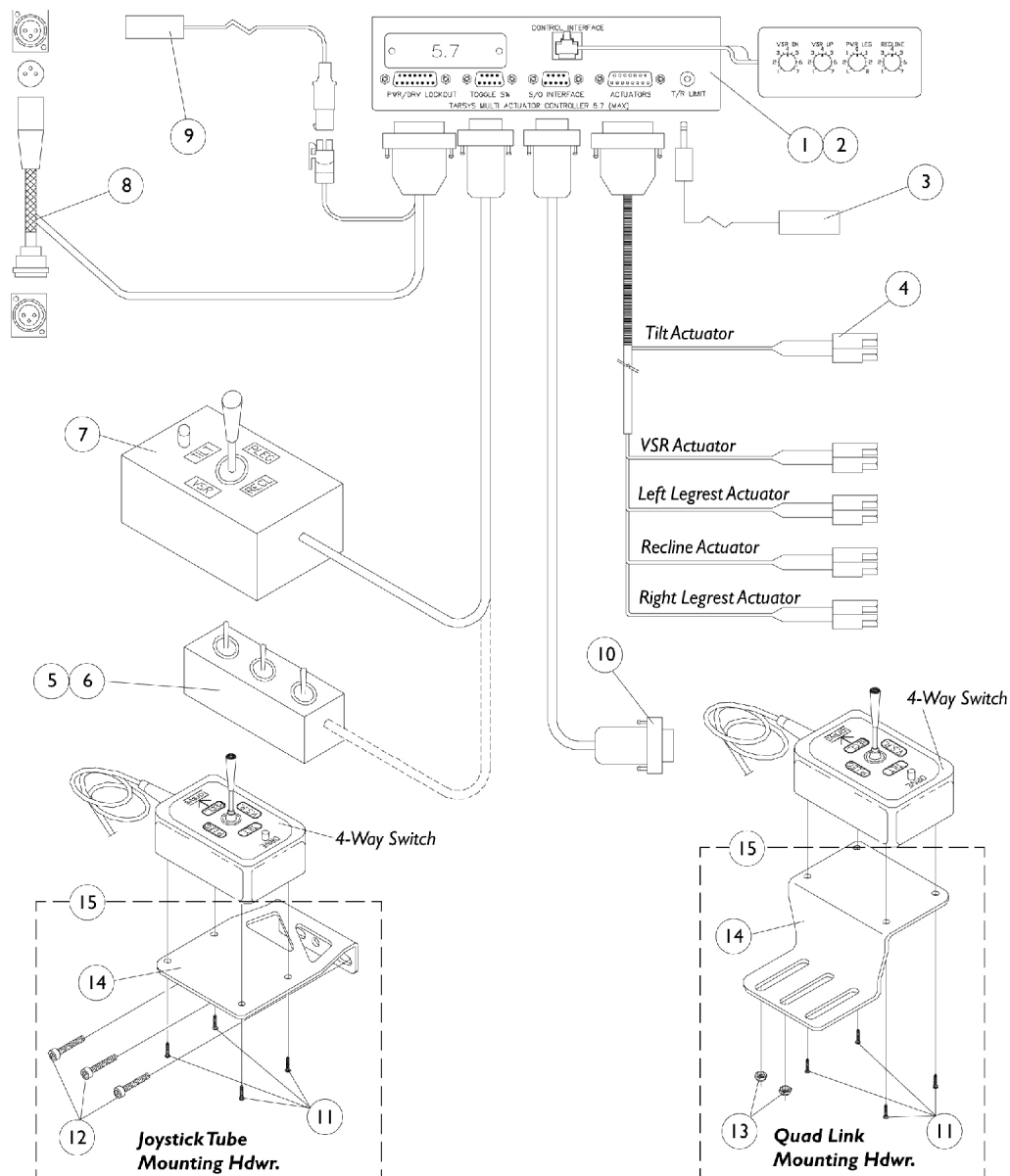

MAX Controller 5.7 S/N start at TE4000 & 4-Way Switch Tilt/Recline & Recline Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,D	0282700-NLA	NLA - Controller, MAX (5.7)	1	1
		1081573	Controller, MAX 5.7 (For Upgrade Kit Only)	1	1
2		0139306-NLA	NLA - Screw, Socket Button Head (#10-32 x 3/4")	1	2
2		0281300	Rivnut (#10-32 x 19/32")	1	2
2	A	0224806-NLA	NLA - Washer (#10)	1	2
3		0018500-NLA	NLA - Switch, Limit	1	1
4		0278600-NLA	NLA - Harness, Actuator	1	1
5		0038100-NLA	NLA - Box, Switch, Triple	1	1
5		0037900-NLA	NLA - Box, Switch, Double	1	1
5		0038000	Box, Switch, Single	1	1
6		1077304	Dual Lock, Male/ Female Adhesive Back (1" x 3")	1	1
7		0279800-NLA	NLA - Box, Four Way Switch	1	1
8	B	0278900-NLA	NLA - Harness, Power, Storm, Round Plug	1	1
9		0018600-NLA	NLA - Switch, Limit - Molex Style	1	1
10		0200300-NLA	NLA - Cable, Interface, Power Legs Combined	1	1
10		0015000	Cable, Interface, Power Legs Independent (1 or 2 Functions)	1	1
10		0014800	Cable, Interface, Invacare R.I.M. Control	1	1
10		0015400-NLA	NLA - Cable, Interface, Peachtree Power Legs Combined	1	1
11		1072695-NLA	NLA - Screw, Phillips Pan Head Tap (#6 x 1/2")	1	4
12		7930304-NLA	NLA - Screw, Socket Head (1/4-20 x 1-1/4")	25	3
13	F	317601-NLA	NLA - Locknut, Low Profile (#10-32)	1	2
14		0194602-NLA	NLA - Plate, Joystick Tube Mounting	1	1
14		0212002-NLA	NLA - Plate, Quad Link Mounting	1	1
15		1077266-NLA	NLA - Mounting Plate Assembly, Joystick Tube	1	1
15		1077267	Mounting Plate Assembly, Quad Link	1	1
NOTE: A - Tilt and Recline B - Drive Lockout C - Max Controller 5.7 is used only when the power and actuator "Adaptor Cables" are present. D - Not Shown E - Use with Quad Link before 4/07/06 F - Use with Joystick Tube and Quad Link (QLA) after 4/06/06					