

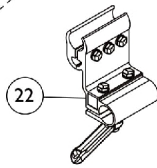


MK5 MPJ Joystick and Mounting Hardware


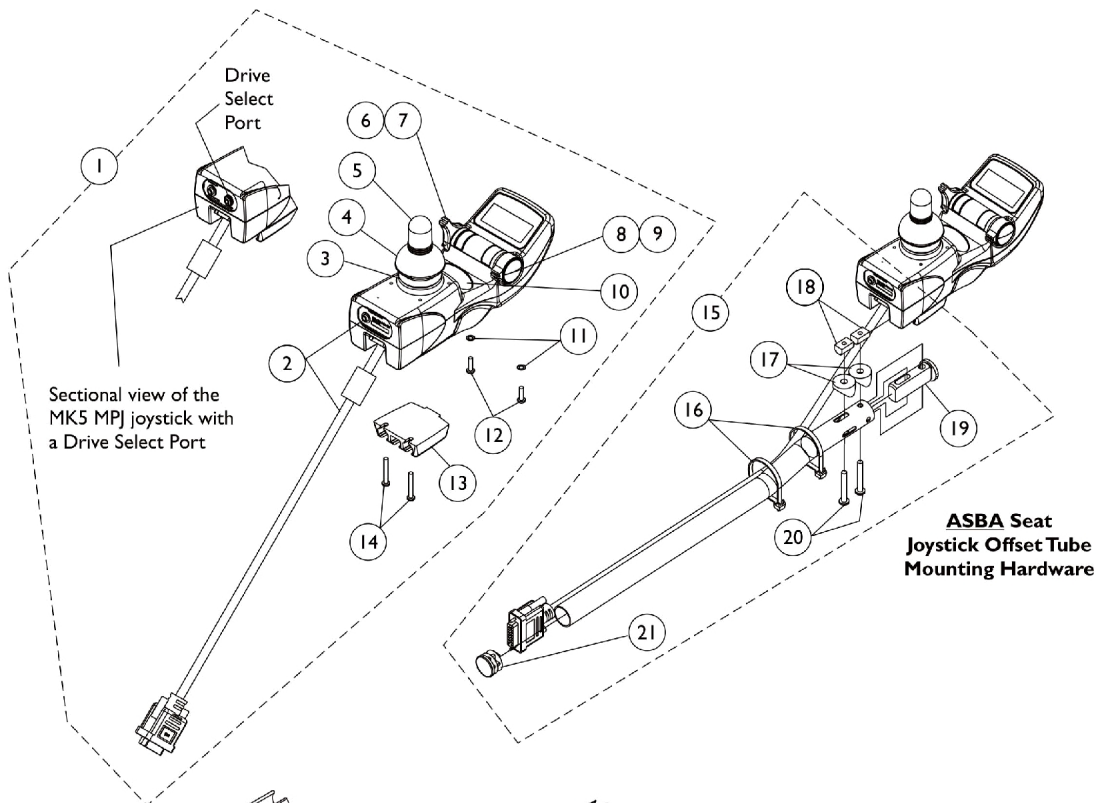


ASBA Seat, Fixed Height Joystick Offset Tube Clamp Assembly

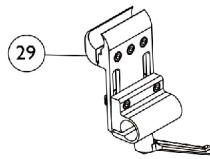

ASBA Seat, Adjustable Height Joystick Offset Tube Clamp Assembly

Individual components are not available

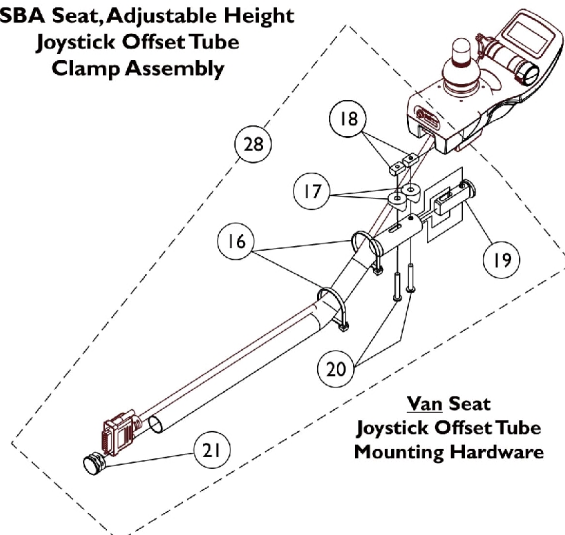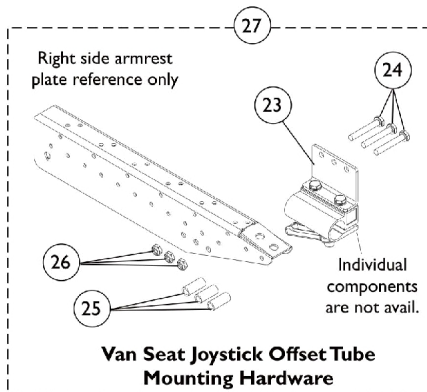

MK5 MPJ Joystick and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1115740-NLA	NLA - Joystick, MK5 MPJ	1	1
1	A	1115740E-NLA	NLA - WARR RPLMT - Joystick, MK5 MPJ	1	1
1	A	1122214-NLA	NLA - Joystick, MK5 MPJ with Drive Select Port	1	1
2		1117038-NLA	NLA - Joystick Bottom and Cable Assembly, MK5 MPJ	1	1
3		1115235-NLA	NLA - Inductive, MK5 MPJ/ DPJ/ SPJ-80 Joystick	1	1
4		1040218	Skirt, Joystick Rubber, Black	1	1
5		1040217	Knob, Joystick Rubber, Black	1	1
6		1115237-NLA	NLA - Switch Assembly, Power/ Drive Select MK5 MPJ	1	1
7		1115214-NLA	NLA - Knob, Power/Drive Select - MK5/MK6i MPJ	1	1
8		1115239-NLA	NLA - Potentiometer Assembly, Speed MK5 MPJ	1	1
9		1115233-NLA	NLA - Knob, Speed Potentiometer - MK5 MPJ	1	1
9		1139998-NLA	NLA - Knob, Speed Adjustment, Black - MK6i	1	1
10		1095497	Decal, Invacare Logo (1.6)	1	1
11		1008834-NLA	NLA - Lockwasher, Internal Tooth (#6)	1	2
12		1115242-NLA	NLA - Screw, Torx Thread Forming (#6-32 x 1/2")	1	2
13		1115503-NLA	NLA - Bracket, T-Nut Mounting	1	1
14		1115244	Screw, Torx Thread Forming (#6-32 x 1")	1	2
15	B	1115860-NLA	NLA - Joystick Offset Tube w/ Hardware, Black (1/2")	1	1
16		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	2
17		40157X019-NLA	NLA - Spacer, Coved (1/4 x 7/8 x 7/16")	1	2
18		1117216-NLA	NLA - T-Nut (#10-32)	1	2
19		1093063	Spacer, Joystick Tube, Black	1	1
20		1026107	Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
21		1027095	Plug Button, Black (7/8")	1	1
22		1093071	Clamp, Fixed Height Joystick Tube	1	1
23		1105049	Clamp Assembly, Fixed Height, Captain/ Van Seat	1	1
24		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	3
25		1087219	Spacer (1/4 x 3/8 x 1")	1	3
26		1025195	Locknut (1/4-20)	1	3
27	C	1115000	Kit, Joystick Tube Mounting Hardware - Captain Seat	1	1
28	B	1123922-NLA	NLA Joystick Offset Tube w/ Hardware, Black (2")	1	1
29	E	1139422	Clamp, Height Adjustable Joystick Tube (ARM250)	1	1
	D	1006270	Extension, Joystick, "T" Style (1560) (4" W x 5-1/2" L)	1	1
	D	1006271	Extension, Joystick, Straight Style (1561) (1/2" W x 5" L)	1	1

NOTE: A - Includes items 2-14
 B - Includes Joystick Tube plus items 16-21
 C - Includes items 23-26
 D - Not Shown
 E - Introduced as an option (ARM250) and can be used to replace a Fixed Height Joystick Clamp.