



# Electronics - Quad Link Assembly

## Before 4/07/06

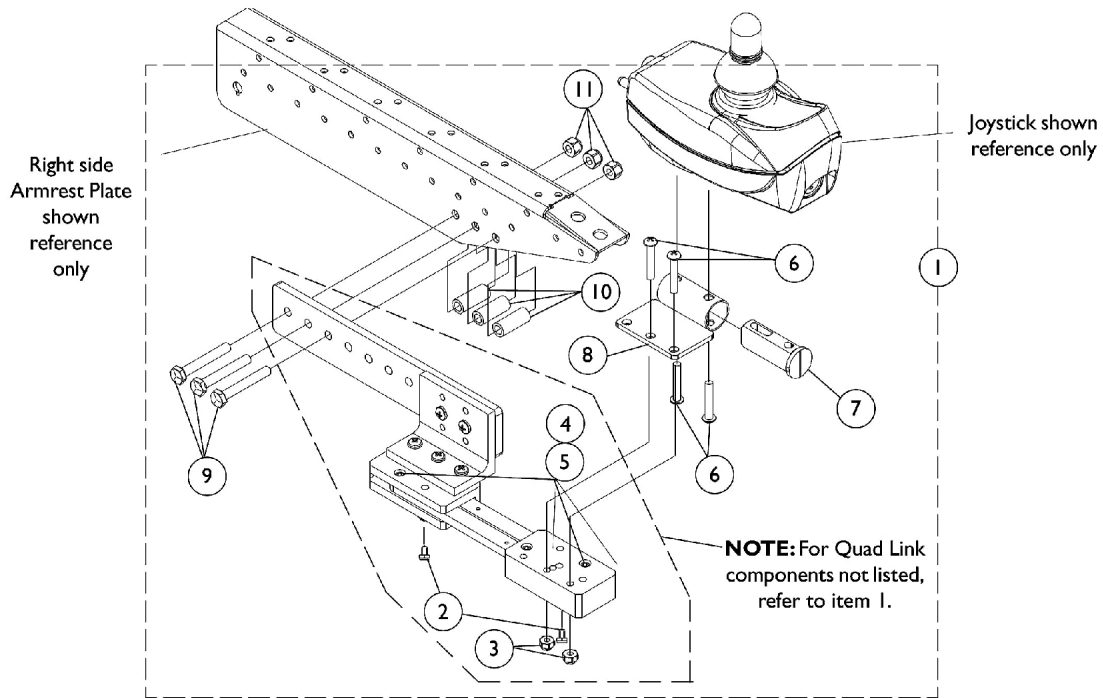

**Semi Recline Van Seat and Office Style Van Seat**

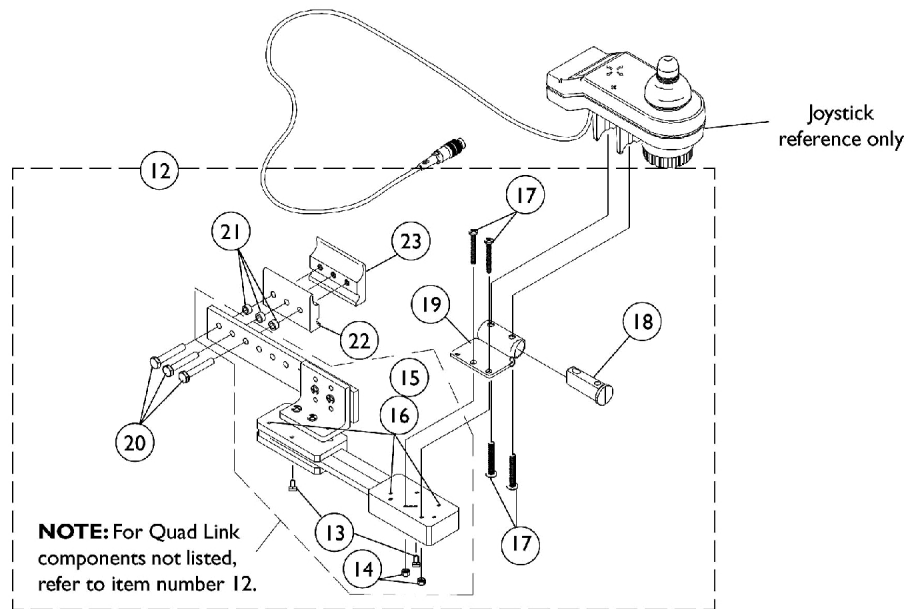

**ASBA Seat**

## Electronics - Quad Link Assembly Before 4/07/06

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1119215-NLA	NLA - Kit, Quad Link Assembly - Right For Van Seats	1	1
2			NLA - Ball Plunger Assembly with Lock Washer	2	1
3			Locknut (#10-32)	1	2
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
6			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	4
7			Spacer, Joystick Tube, Black	1	1
8			NLA - Stud Plate, Quad Link	1	1
9			NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	3
10			Spacer (1/4 x 3/8 x 1")	1	3
11			Locknut (1/4-20)	1	3
1	A	1119216-NLA	NLA - Kit, Quad Link Assembly - Left For Van Seats	1	1
2			NLA - Ball Plunger Assembly with Lock Washer	2	1
3			Locknut (#10-32)	1	2
4			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
5			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
6			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	4
7			Spacer, Joystick Tube, Black	1	1
8			NLA - Stud Plate, Quad Link	1	1
9			NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	3
10			Spacer (1/4 x 3/8 x 1")	1	3
11			Locknut (1/4-20)	1	3
12	B	1052402-NLA	NLA - Kit, Quad Link Assembly - Right	1	1
13			NLA - Ball Plunger Assembly with Lock Washer	2	1
14			Locknut (#10-32)	1	2
15			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
16			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
17			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	4
18			Spacer, Joystick Tube, Black	1	1
19			NLA - Stud Plate, Quad Link	1	1
20			Screw, Hex Head (1/4-20 x 1-1/2")	1	3
21			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3
22			Half Clamp, Non-Threaded Holes	1	1
23			NLA - Half Clamp, Threaded Holes	1	1
12	B	1115372-NLA	NLA - Kit, Quad Link Assembly - Left	1	1
13			NLA - Ball Plunger Assembly with Lock Washer	2	1
14			Locknut (#10-32)	1	2
15			NLA - Screw, Cap, Socket Head (#10-32 x 1/2")	1	4
16			NLA - Bushing, Bronze (3/16 x 1/4 x 1/4")	1	4
17			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	4
18			Spacer, Joystick Tube, Black	1	1
19			NLA - Stud Plate, Quad Link	1	1
20			Screw, Hex Head (1/4-20 x 1-1/2")	1	3
21			NLA - Spacer (1/4 x 3/8 x 3/16")	1	3

## Electronics - Quad Link Assembly Before 4/07/06

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
22			Half Clamp, Non-Threaded Holes	1	1
23			NLA - Half Clamp, Threaded Holes	1	1
NOTE: A - Used on Semi Recline and Office Style Van Seats. Includes items 2-11. B - Used on ASBA Seats. Includes items 13-23.					