


Captain Seat Arms and Hardware

18" Wide Models

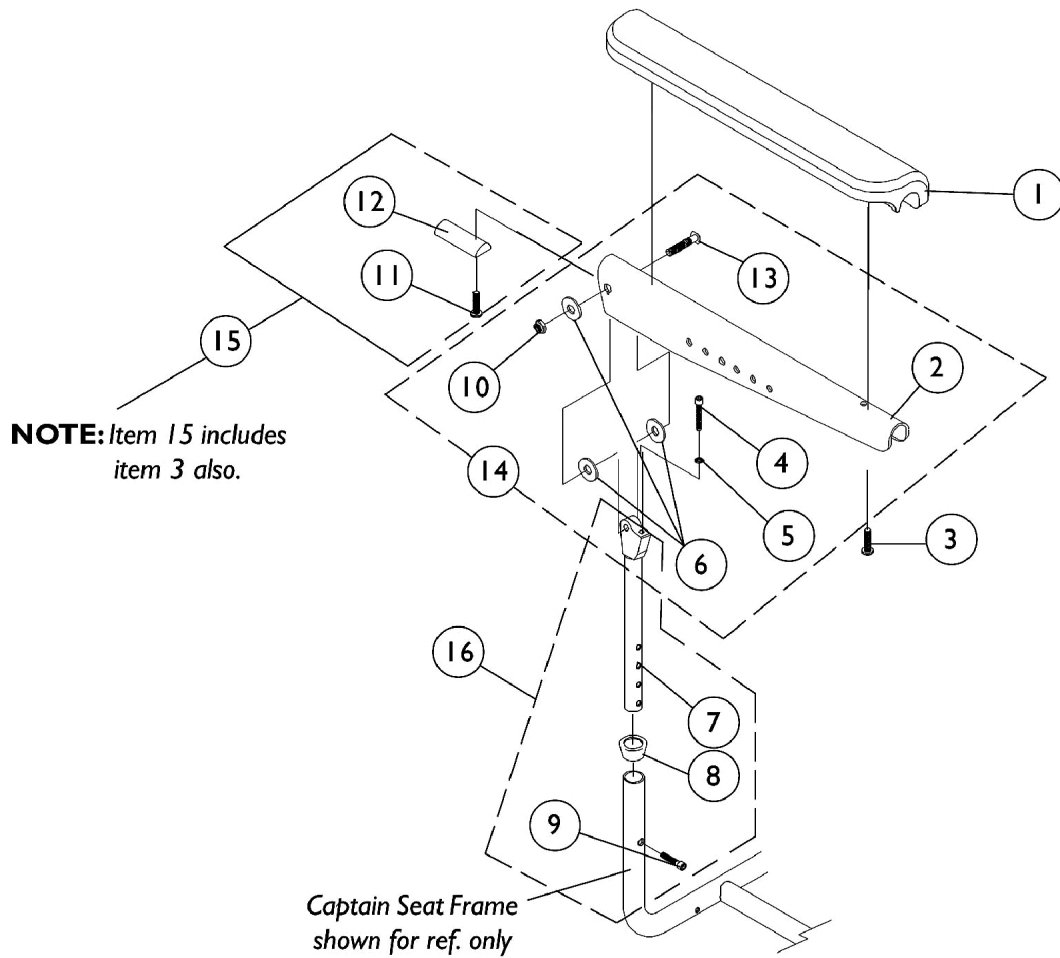

Captain Seat Arms and Hardware 18" Wide Models

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|--------------------------------------------------------------------------------------|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | 1117328 | Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) | 1 | 2 |
| 2 | | 1071359-NLA | NLA - Plate, Armrest (Black) | 1 | 2 |
| 3 | | 1028755 | Screw, Phillips Pan Head (#10-32 x 5/8") | 1 | 4 |
| 4 | | 1009837 | Screw, Socket Button Head (1/4-20 x 1") | 1 | 2 |
| 5 | | 1025711-NLA | NLA - Nut, Hex (1/4-20) | 1 | 2 |
| 6 | | 1025704 | Washer, Flat (5/16 x 5/8 x 1/16") | 1 | 6 |
| 7 | | 1070928 | Tube, Arm, Upper Height and Width Adjustable, Black | 1 | 2 |
| 8 | | 15680X013 | Anti-Rattle (1") | 1 | 2 |
| 9 | | 7930304-NLA | NLA - Screw, Socket Head (1/4-20 x 1-1/4") | 25 | 1 |
| 10 | | 1026906-NLA | NLA - Locknut (5/16-24) | 1 | 2 |
| 11 | | 1102546-NLA | NLA - Screw, Phillips Pan Head (#10-32 x 7/8") | 1 | 2 |
| 12 | | 1070106-NLA | NLA - Insert, Armrest, Black | 1 | 2 |
| 13 | | 1031026 | Screw w/ Lug (5/16-24 x 1-3/8") | 1 | 2 |
| 14 | B | 1074120-NLA | NLA - Kit, Armrest Attaching Hardware (Pair) | 2 | 1 |
| 15 | C | 1073378-NLA | NLA - Kit, Arm Pad Attaching Hardware | 2 | 1 |
| 16 | D | 1070756-NLA | NLA - Arm Tube Assembly, Upper Adjustable (Pair) | 2 | 1 |

NOTE: A - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs for chairs manufactured before 1/3/02.
B - Includes items 2, 4, 5, 6, 10, and 13.
C - Includes items 3, 11, and 12.
D - Includes 7, 8, and 9.