


Swingarm and Caster with Hardware - Invacare 400

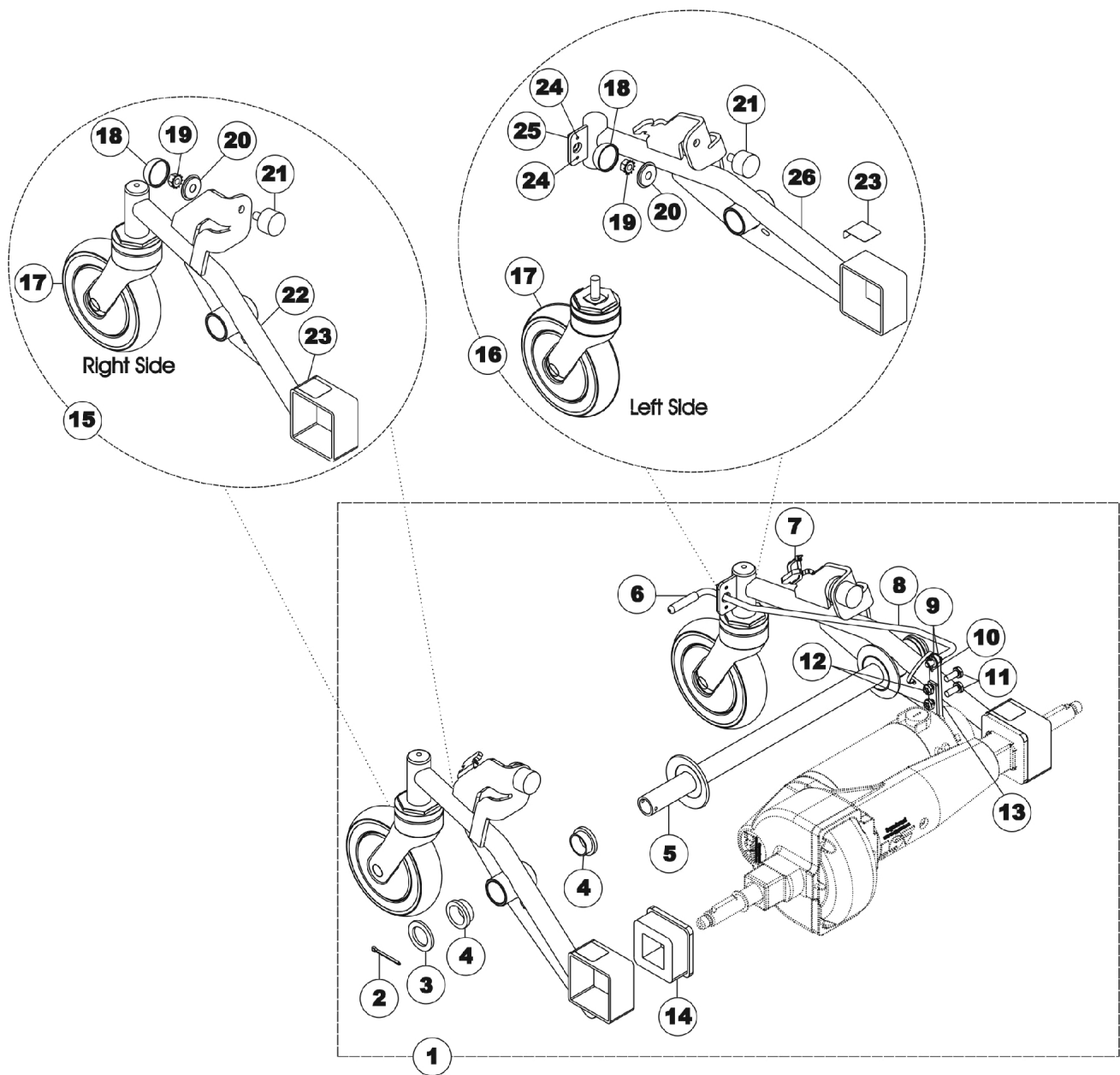

Swingarm and Caster with Hardware - Invacare 400

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1124906-NLA	NLA - Frame, Swingarm Assembly (Invacare 400 Scooter)	1	1
2			NLA - Pin, Cotter (1/8 x 1-1/4")	1	2
3			NLA - Washer, Steel (29/32 x 1-3/8 x 1/8")	1	2
4			NLA - Washer, Shoulder	1	4
5			NLA - Tube, Pivot, Black	1	1
6			Tip, Plastic, Black	1	1
7			Tie Wrap (5-5/8" L)	1	2
8			NLA - Rod, Release Brake	1	1
9			Washer (1/4 x 1/2 x 1/16")	1	2
10			NLA - Pin, Cotter (5/64 x 1/2")	1	1
11			Screw, Hex Head (1/4-20 x 5/8")	1	2
12			NLA - Locknut (1/4-20)	1	2
13			NLA - Bracket, Extension Brake, Black	1	1
14			NLA - Element, Viscous	1	2
17			NLA - Caster, Swivel (5-1/2") (Invacare 400 Scooter)	1	2
18			NLA - Cap, Hub, Black	1	2
19			NLA - Nut, Keps (5/16-18)	1	2
20			NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	2
21			NLA - Bumper, Rubber (1" Diameter x 5/8")	1	2
23			Decal, Warning Pinch Point	1	2
24			NLA - Rivet (1/8 x 1")	1	2
25			NLA - Plate, Nylon	1	1
26			NLA - Frame, Swingarm, Black - Left	1	1
22			NLA - Frame, Swingarm, Black - Right	1	1
16	B,C	1122050-NLA	NLA - Swingarm with Caster - Left (Invacare 400 Scooter)	1	1
17			NLA - Caster, Swivel (5-1/2") (Invacare 400 Scooter)	1	2
18			NLA - Cap, Hub, Black	1	2
19			NLA - Nut, Keps (5/16-18)	1	2
20			NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	2
21			NLA - Bumper, Rubber (1" Diameter x 5/8")	1	2
23			Decal, Warning Pinch Point	1	2
26			NLA - Frame, Swingarm, Black - Left	1	1
15	B,D	1122049-NLA	NLA - Swingarm with Caster - Right (Invacare 400 Scooter)	1	1
17			NLA - Caster, Swivel (5-1/2") (Invacare 400 Scooter)	1	2
18			NLA - Cap, Hub, Black	1	2
19			NLA - Nut, Keps (5/16-18)	1	2
20			NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	2
21			NLA - Bumper, Rubber (1" Diameter x 5/8")	1	2
22			NLA - Frame, Swingarm, Black - Right	1	1
23			Decal, Warning Pinch Point	1	2

Swingarm and Caster with Hardware - Invacare 400

NOTE: A - Includes: Items 2-26; Assembly Includes both Right and Left sides.
B - The Swingarm assembly includes the caster.
C - Includes: Items 17-21 & 23-26.
D - Includes: Items 17-23.