


Front Rigging Hanger Assembly

After 5/21/06

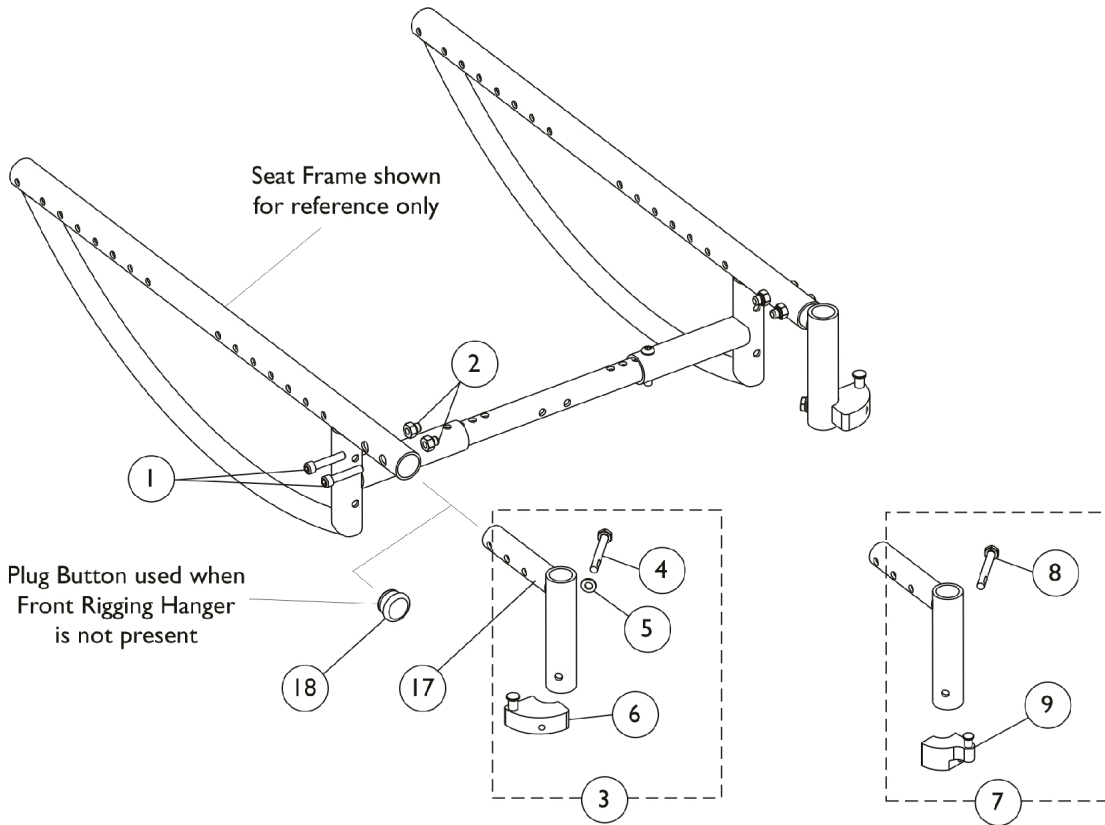

Hanger Brackets for **Swingaway Front Riggings** on chairs with a **15-19" or 18-22" Depth Range**

Hanger Brackets for **Swingaway Front Riggings** on chairs with a **12-16" Depth Range**

Hanger Brackets for **Lift Off Front Riggings** on chairs with a **15-19" or 18-22" Depth Range**

Hanger Brackets for **Lift Off Front Riggings** on chairs with a **12-16" Depth Range**

Front Rigging Hanger Assembly After 5/21/06

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1074640	Screw, Allen Head (1/4-28 x 1-1/4")	1	2
2		1085111	Nut, Hex (1/4-28)	1	2
3	A	1054866	Kit, Sector Block Hardware (Pair)	1	1
4			Screw, Hex Head w/ Patch (1/4-20 x 2")	1	2
5			Washer (1/4 x 1/2 x 1/16")	1	2
6			NLA - Block, Sector - Right	1	1
6			NLA - Block, Sector - Left	1	1
7	B	1137404	Kit, Sector Block Hardware	1	1
8			Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
9			Block, Sector - Right	1	1
9			Block, Sector - Left	1	1
10	C	1142975	Kit, Sector Block Hardware, Lift Off (Pair)	1	1
11			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
12			Washer (1/4 x 1/2 x 1/16")	1	2
13			Block, Sector - Right	1	1
13			Block, Sector - Left	1	1
14	D	1142950-NLA	NLA - Kit, Sector Block Hardware	1	1
15			Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
16			NLA - Block, Sector - Right	1	1
16			NLA - Block, Sector - Left	1	1
17		1086389	Tube, Support, Front Rigging Hanger, Black - Right	1	1
17		1086390	Tube, Support, Front Rigging Hanger, Black - Left	1	1
18		1127676	Package, Plug Button, Black (1")	10	1

NOTE: A - Chairs with Swingaway Front Riggings that have a depth range of 15-19" or 18-22". Includes items 4-6.
 B - Chairs with Swingaway Front Riggings that have a depth range of 12-16". Includes items 8-9.
 C - Chairs with Lift Off Front Riggings that have a depth range of 15-19" or 18-22". Includes items 11-13.
 D - Chairs with Lift Off Front Riggings that have a depth range of 12-16". Includes items 15-16.