
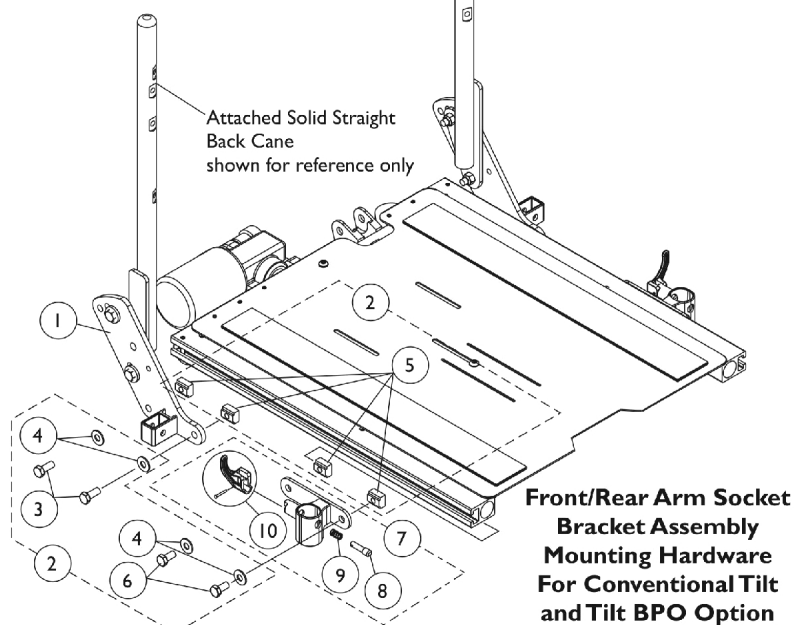


Arm Mounting Hardware

Conv. (CGT) (12-16" W x 16-18" D), Conv., Contoura, ABS & Metal Curved Back (CGT) (16-24" W) & (CGE/CGTE) (16-22" W)

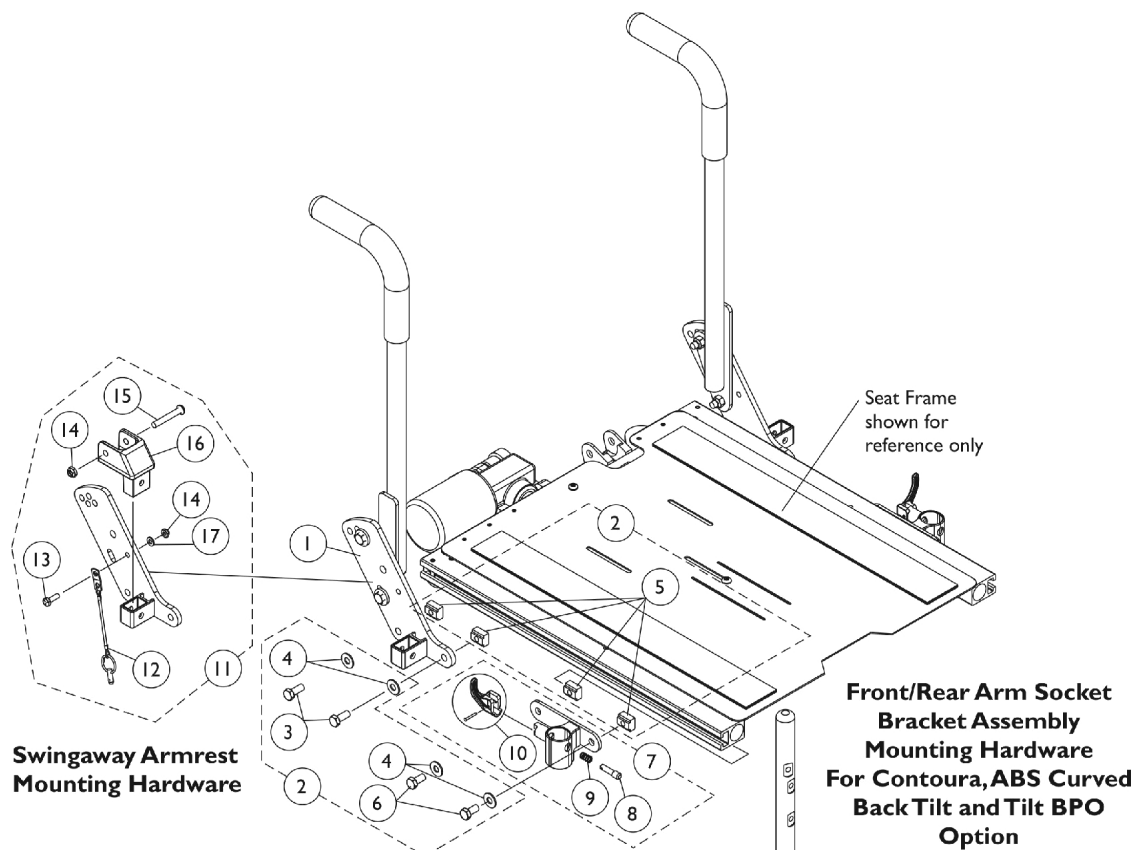

Arm Mounting Hardware

**Conv. (CGT) (12-16" W x 16-18" D), Conv., Contoura, ABS & Metal Curved
Back (CGT) (16-24" W) & (CGE/CGTE) (16-22" W)**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1125478	Bracket, Arm Socket/Cane, Rear - Right	1	1
1		1125479	Bracket, Arm Socket/Cane, Rear - Left	1	1
2		1125734	Kit, Front/Rear Arm Socket Assembly Mounting Hardware	1	1
3			Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	2
4			Washer, Steel (5/16 x 3/4 x 1/16")	1	4
5			T-Nut (5/16-18)	1	4
6	B		Screw, Hex Head w/ Patch (5/16-18 x 5/8")	1	2
7		1122619	Socket, Arm, Front - Right	1	1
7		1122620	Socket, Arm, Front - Left	1	1
8	C	1028594	Pin, Arm Lock, Black	1	1
9		47471X074	Spring, Arm Lock	1	1
10		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
11		1125736	Kit, Swingaway Armrest Mounting Hardware	1	1
12			Pin, Pull	1	1
13			Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	1
14			Locknut (1/4-20)	1	2
15			Screw, Socket Button Head (1/4-20 x 1-5/8")	1	1
16			Clevis, Pivot	1	1
17			Washer (1/4 x 1/2 x 1/16")	1	1
NOTE: A - Includes items 3-6 B - Includes Front Arm Socket and items 8-10 C - Includes items 12-17					