


Swingarm Assembly and Hardware - SureStep

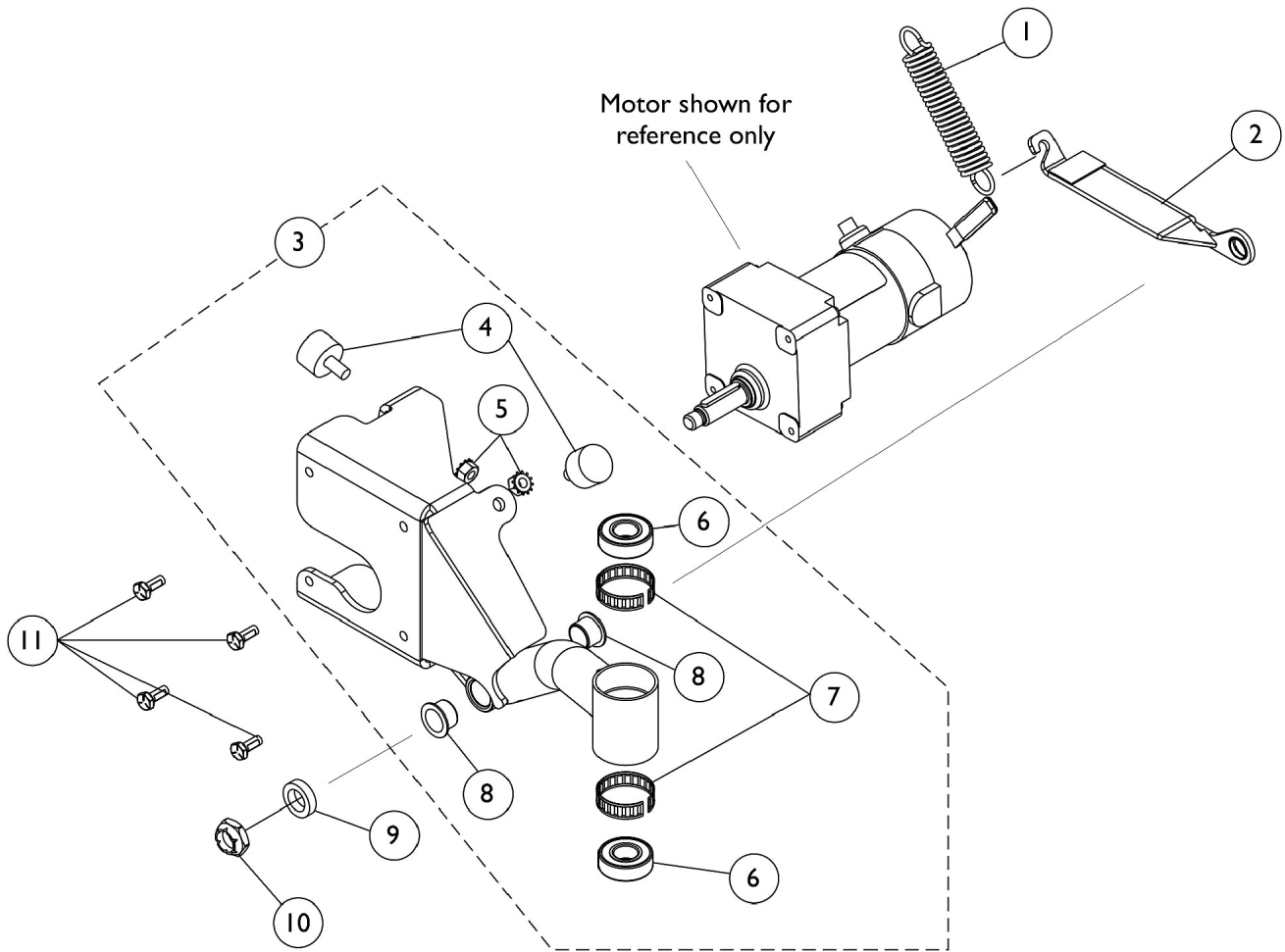

Swingarm Assembly and Hardware - SureStep

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1127400	Spring, Extension (1 x 4-1/2" L)	1	1
2	A	1143829	Bracket, Stop Spring with Flanged Bearing and Hook Fastener, Black - Right	1	1
2	A	1143830	Bracket, Stop Spring with Flanged Bearing and Hook Fastener, Black - Left	1	1
3	B	1143678	Swingarm Assembly - Right	1	1
3	B	1143679-NLA	NLA - Swingarm Assembly - Left	1	1
4		1108437-NLA	NLA - Bumper, Rubber (1" Diameter x 5/8")	1	2
5		000037-NLA	NLA - Nut, Keps (5/16-18)	1	2
6		40117X006	Bearing (5/8")	1	2
7		1110717	Ring, Tolerance	1	2
8		1140994	Bearing, Flange (5/8 x 23/32 x 1/2")	1	2
9		1110685-NLA	NLA - Spacer, Bushing	1	1
10		1028550	Locknut (5/8-18)	1	1
11		1121024	Screw, Hex Head w/ Patch (1/4-20 x 5/8")	1	4

NOTE: A - Stop Spring Bracket includes flanged bearing and hook fastener.
 B - Includes Swingarm plus item 4-8.