
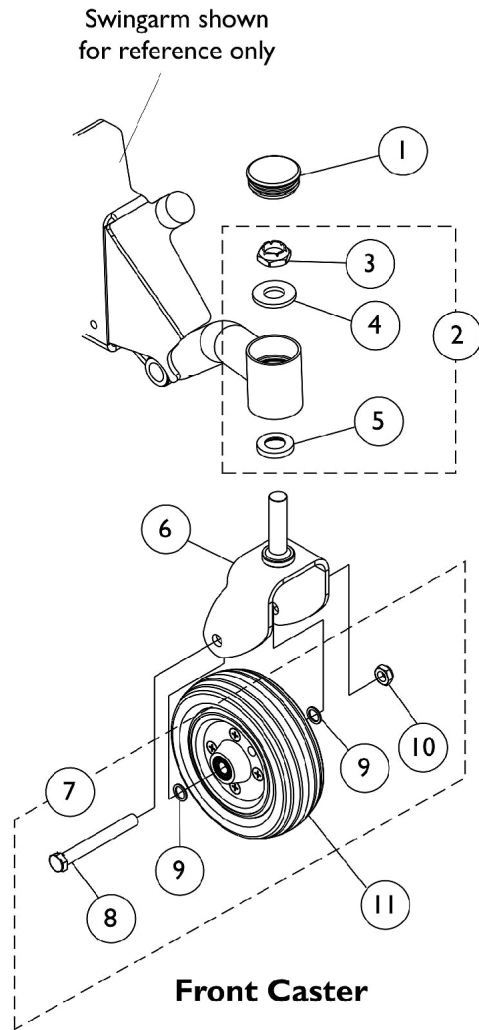


Casters and Forks

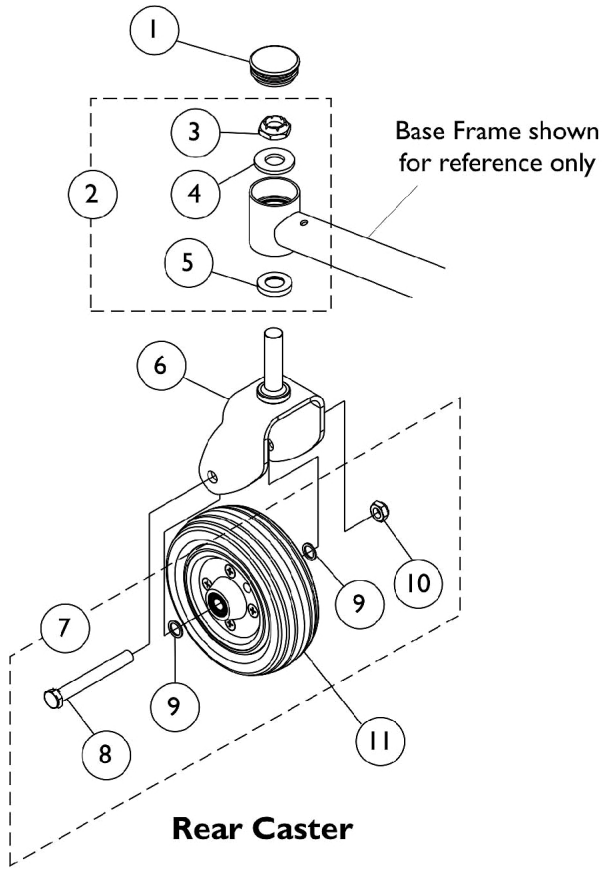

Casters and Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1110709	Cap, Head Tube, Black	1	1
2	A	1106226	Kit, Fork Mounting Hardware	1	1
3		1028550	Locknut (5/8-18)	1	1
4		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
5		1106068	Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
6		1140984-NLA	NLA - Fork and Stem Assembly	1	1
7	B	1110708	Kit, Caster Mounting Hardware	1	1
8		1025726	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
9		1025724	Washer, Steel (15/32 x 5/8 x 3/64")	1	2
10		1025734	Locknut (7/16-20)	1	1
11		1115179	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2")	1	1
NOTE: A - Includes Items 3-5. B - Includes Items 8-10.					