
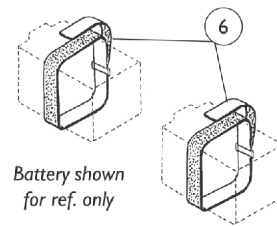
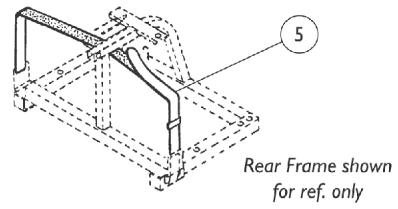


# Battery Harnesses & Chargers

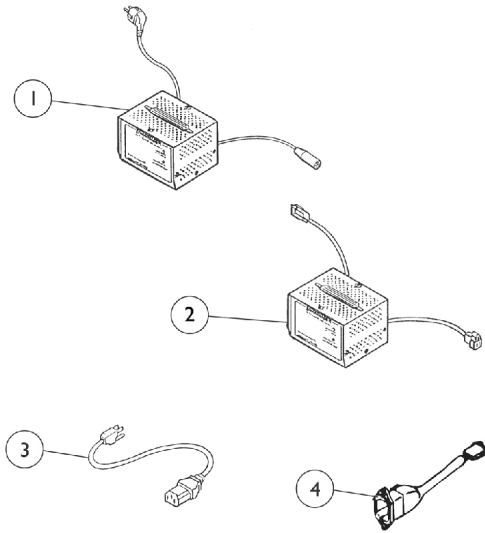

## Battery Harnesses & Chargers

| Item Number         | Note | Part Number | Description                                                                                                                                                                                                                                                                                  | Quantity Per |          |
|---------------------|------|-------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------|----------|
|                     |      |             |                                                                                                                                                                                                                                                                                              | Package      | Assembly |
| 1                   |      | 1105925-NLA | NLA - Charger, Battery, Off Board, Zoom-3, 2 Amp                                                                                                                                                                                                                                             | 1            | 1        |
| 1                   |      | 1088743-NLA | NLA - Charger, Battery, Off Board, 3 Amp, Lynx SX3P and Panther LX4                                                                                                                                                                                                                          | 1            | 1        |
| 2                   |      | 1088744-NLA | NLA - Charger, Battery, On Board, Lynx SX3/ LX3/ LX3 Plus and Panther MX4                                                                                                                                                                                                                    | 1            | 1        |
| 3                   |      | 1088745-NLA | NLA - Cord, Charger, AC                                                                                                                                                                                                                                                                      | 1            | 1        |
| 4                   |      | 1099754-NLA | NLA - Charging Cable, On Board with Hardware (Lynx SX3/ LX3/ LX3 Plus and Panther MX4)<br>Includes: (1) Charging Cable, (2) Screw, Phillips Pan Head (M3 x 0.5 x 16)<br>2 ea. Washer (3.2 x 10 x 0.8)<br>2 ea. Locknut (M3 x 0.5 x 6)<br>2 ea. Cable Tie<br>1 ea. Tube, PVC (19 x 20 x 260L) | 1            | 1        |
| 5                   |      | 1088741-NLA | NLA - Strap, Battery Fastening (All Models)                                                                                                                                                                                                                                                  | 1            | 1        |
| 6                   |      | 1088742-NLA | NLA - Strap, Carrying, Battery                                                                                                                                                                                                                                                               | 2            | 1        |
|                     | A    | 1088750-NLA | NLA - Harness, Battery Cable (Positive/ Negative) - Lynx SX3P/ SX3/ LX3/ LX3 Plus and Panther LX4                                                                                                                                                                                            | 1            | 1        |
|                     | A    | 1105926-NLA | NLA - Harness, Battery Cable (Positive/ Negative) - Zoom-3                                                                                                                                                                                                                                   | 1            | 1        |
|                     | A    | 1088751-NLA | NLA - Harness, Battery Cable (Positive/ Negative) - Panther MX4                                                                                                                                                                                                                              | 1            | 1        |
| NOTE: A - Not Shown |      |             |                                                                                                                                                                                                                                                                                              |              |          |