

Invacare® microAIR® (MA Series) Therapeutic Support Surfaces

MA500

MA600

MA800

MA900

MA1000



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 19_057. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.


Name	Phone Numbers

© 2021 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design, MicroAIR and Yes, you can. are registered trademarks of Invacare Corporation.

TABLE of CONTENTS

Description	Page Number
Accessories, Miscellaneous	23
MicroAIR 1000 Mattress	21
MicroAIR 500 Mattress	3
MicroAIR 600 Mattress	5
MicroAIR 800 Mattress	7
MicroAIR 800M42 Mattress	9
MicroAIR 800M4284 Mattress	11
MicroAIR 800M48 Mattress	13
MicroAIR 900 Mattress	15
MicroAIR 900M42 Mattress	17
MicroAIR 900M48 Mattress	19

MicroAIR 500 Mattress

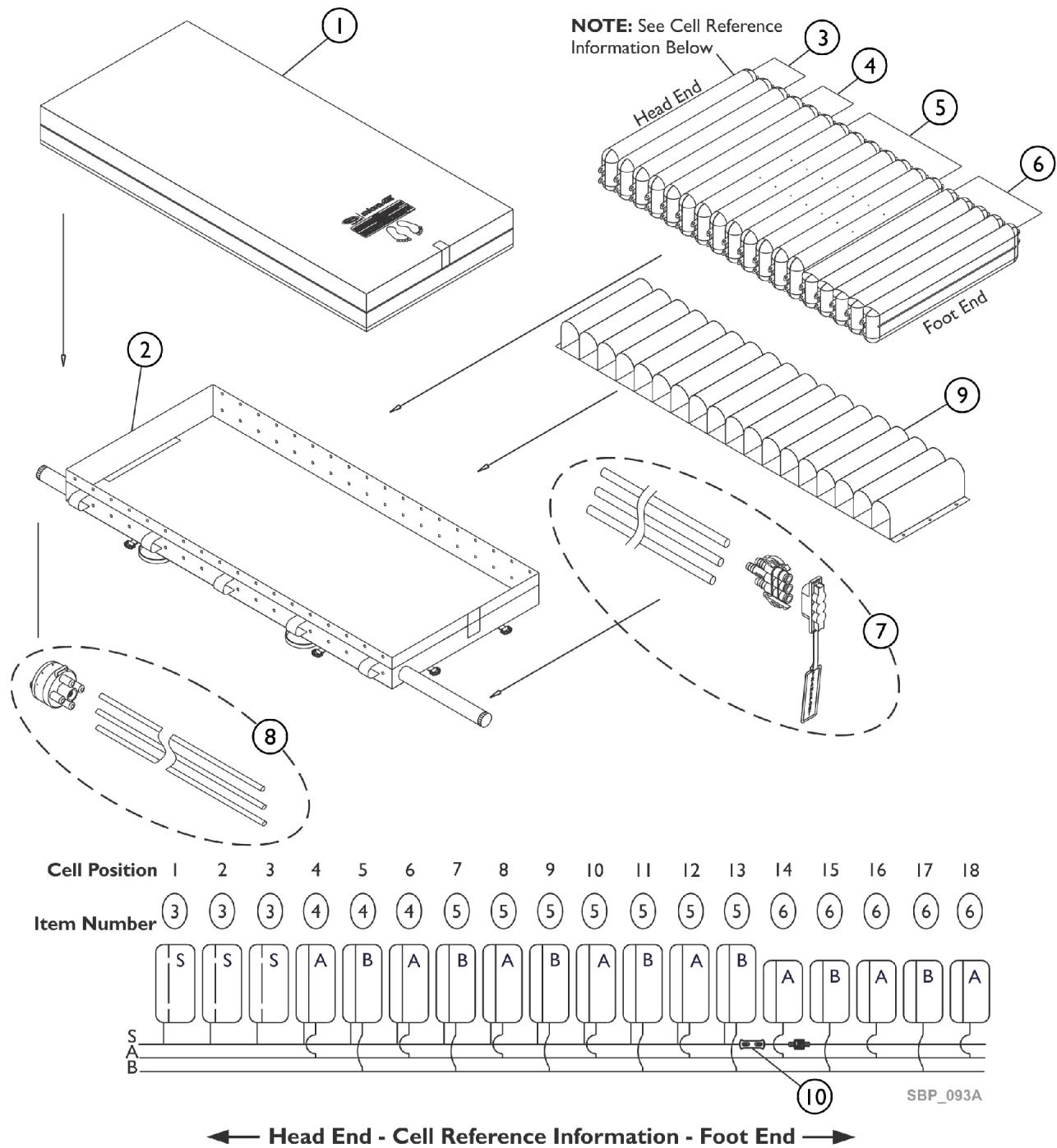

NOTE:


Letter S = Static Cells. Remain Inflated

Letters A and B = Alternating Sequence of the Cells.

MicroAIR 500 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124603	Top Cover	1	1
2		60124604	Bottom Cover	1	1
3		60124607	Head Cell for MA500 Mattress(No LAL)	1	3
4		60124608	Body Cell for MA500 Mattress(No LAL)	1	3
5		60124258	MA600 Vent Cells	1	7
6		60124609	Foot Cell for MA500(No LAL)	1	5
7		60124605	Connecting Tube Set for MA500	1	1
8		60124612	CPR Set for MA500	1	1
9		60124606	Belt Set for MA500 Mattress	1	1
10		60124610	Mattress Muffler for MA500	1	1
	A	60124173	Filter - MA500 / MA600	1	1
	A	60124175	Filter Cover - MA500 / MA600	1	1
	A	60124176	Hook Set, MA500	1	1
	A	60124611	CPR Valve, High Sensitivity Quick Inflate (Shoer Version)	1	1
	A	60124613	Trio Coupling Connector, MA500	1	1
NOTE: A - Not Shown					

MicroAIR 600 Mattress

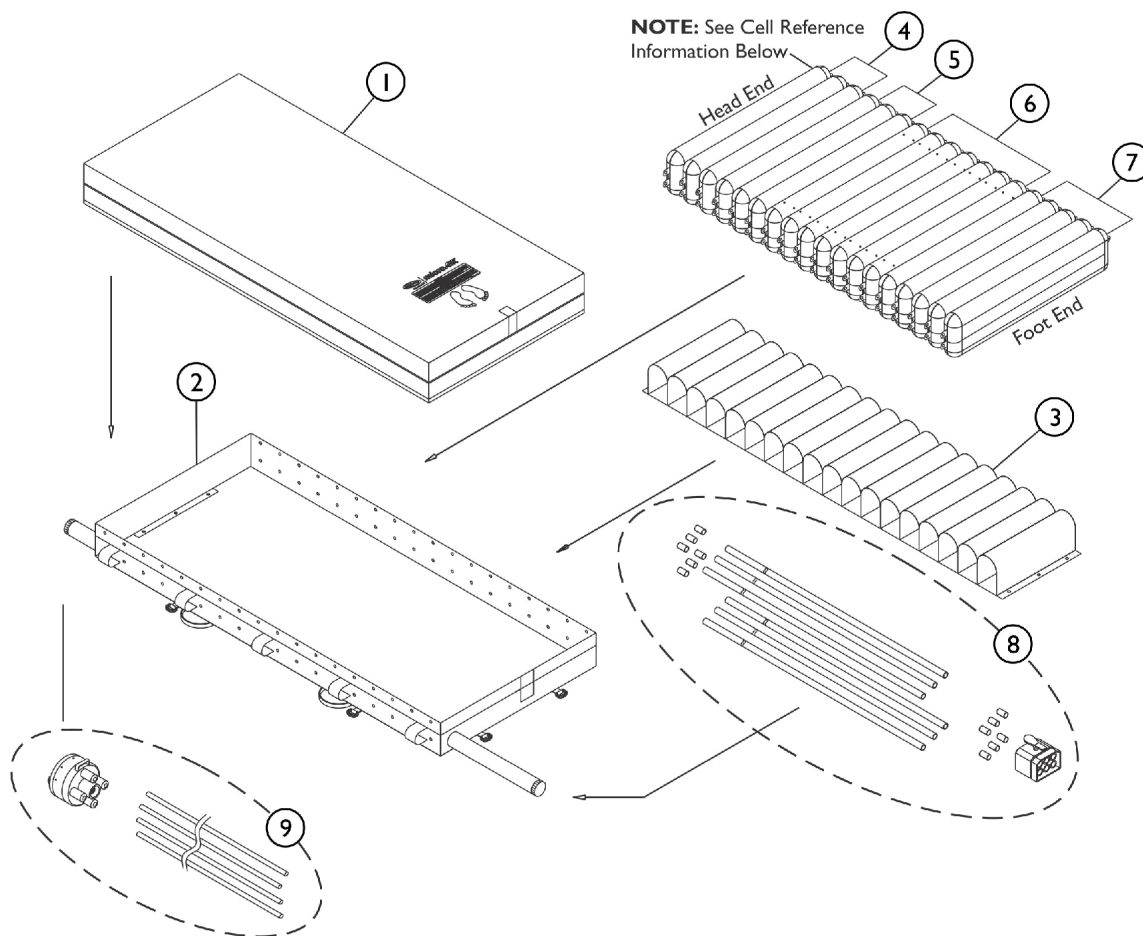

Cell Position	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18
Item Number	4	4	4	5	5	5	6	6	6	6	6	6	6	7	7	7	7	7
	S	S	S	A	B	C	A	B	C	A	B	C	A	B	C	A	B	C

S
C
B
A

← Head End - Cell Reference Information - Foot End →

NOTE:

Letter S = Static Cells. Remain Inflated.

Letters A, B and C = Alternating Sequence of the Cells.

Example:

When mattress is in 3-I alternating mode, cells marked "A", "B", and "C" will be in deflation mode in rotation.

1st cycle = A and B cells inflate, C cells deflate.


2nd cycle = A and C cells inflate, B cells deflate.

SBP_094A

MicroAIR 600 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124618	Top Cover	1	1
2		60124619	Bottom Cover	1	1
3		60124255	MA600 Belt Set	1	1
4		60124256	MA600 Head Cell	1	3
5		60124257	MA600 Non-Vented Air Cell	1	3
6		60124258	MA600 Vent Cells	1	7
7		60124259	MA600 Foot Cells	1	5
8		60124616	MA600 Connecting Tube Set	1	1
9		60124617	MA600 CPR Set	1	1
	A	60124614	Tee Connector, Barbed, for Tubing, (3/8"x3/8"x3/8")	1	1
	A	60124238	Tee Connector, Barbed, for Tubing, (1/2"x1/2"x1/2")	1	1
	A	60124240	Filter - MA500 / MA600	1	1
	A	60124175	Filter Cover - MA500 / MA600	1	1
	A	60124246	Rubber Foot Pad	1	1
	A	60124247	Fuse	1	1
	A	60124248	Spring Hook Set, M18 Series	1	1
	A	60124250	Connector to Pump, 6 Po	1	1
	A	60124615	Gray Tubing (Per Meter) (3/8")	1	1
	A	60124611	CPR Valve, High Sensitivity Quick Inflate (Shoer Version)	1	1
NOTE: A - Not Shown					


MicroAIR 800 Mattress

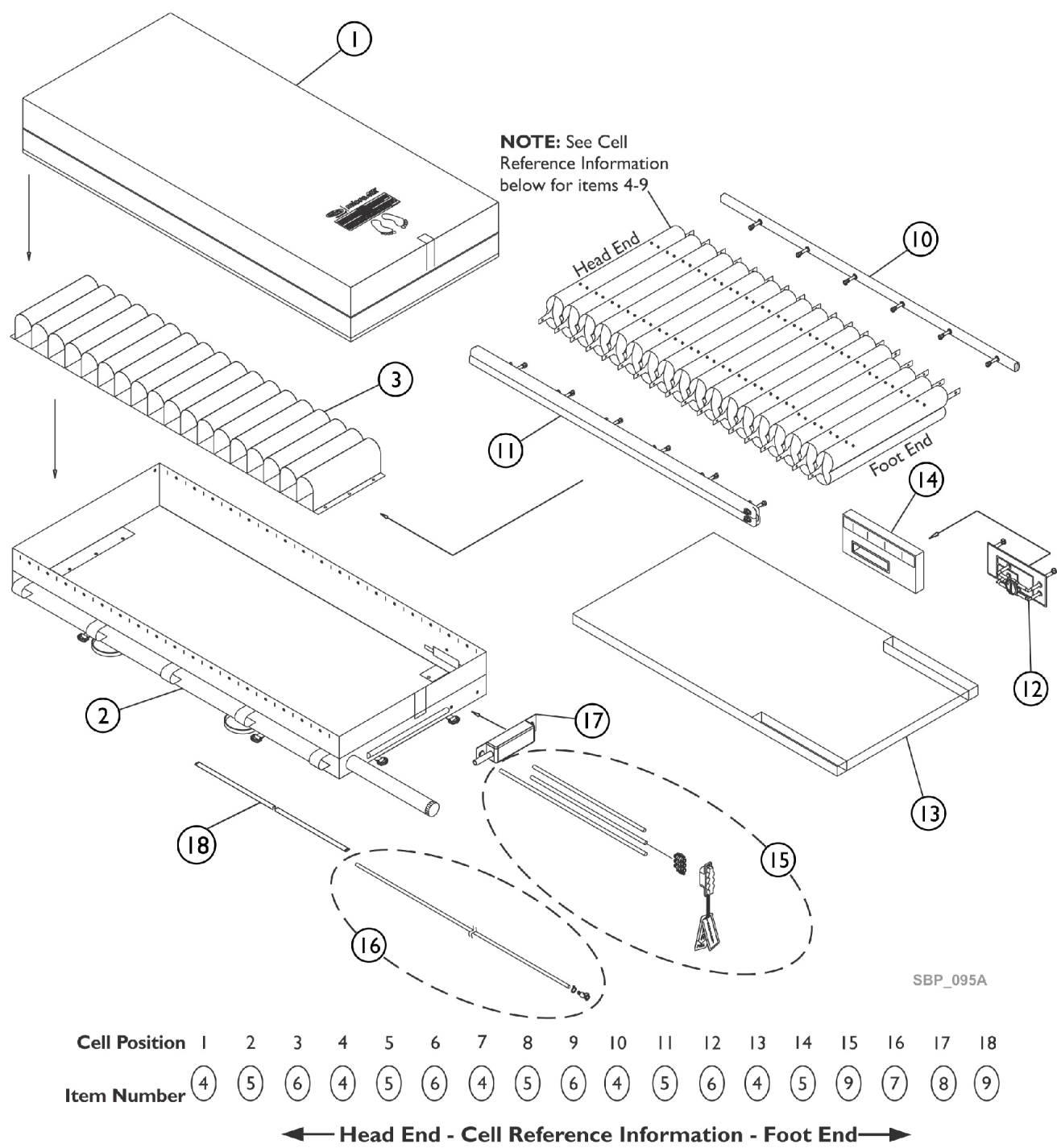

MicroAIR 800 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124641	Top Cover	1	1
2		60124642	Bottom Cover	1	1
3		60124622	Belt Set	1	1
4		60124624	8" Vent Cell (Blue)	1	5
5		60124625	8" Vent Cell (White)	1	5
6		60124626	8" Vent Cell (Green)	1	4
7		60124627	8" Foot Vent Cell (Blue)	1	1
8		60124628	8" Foot Vent Cell (White)	1	1
9		60124629	8" Foot Vent Cell (Green)	1	2
10		60124633	Left Side Air Manifold	1	1
11		60124634	Right Side Air Manifold	1	1
12		60124635	Foot Side Rail For (14-1/2" x 7")	1	1
13		60124630	Foam Cover	1	1
14		60124631	Foot Foam Cover	1	1
15		60124671	Connecting Tube Set (Alternating Pipe Set)	1	1
16		60124621	Connecting Tube Set (Exhausting Pipe)	1	1
17		60124632	Vent/ Muffler Exhaust	1	1
18		60124623	Hose Sleeve	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124184	Exhaust O-Ring	1	1
	A	60124187	CPC Exhaust Connector (MA800)	1	1
	A	60124190	Heel Sense valve sticker	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124638	CPC Connector, 3/8" Male	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1
NOTE: A - Not Shown					

MicroAIR 800M42 Mattress

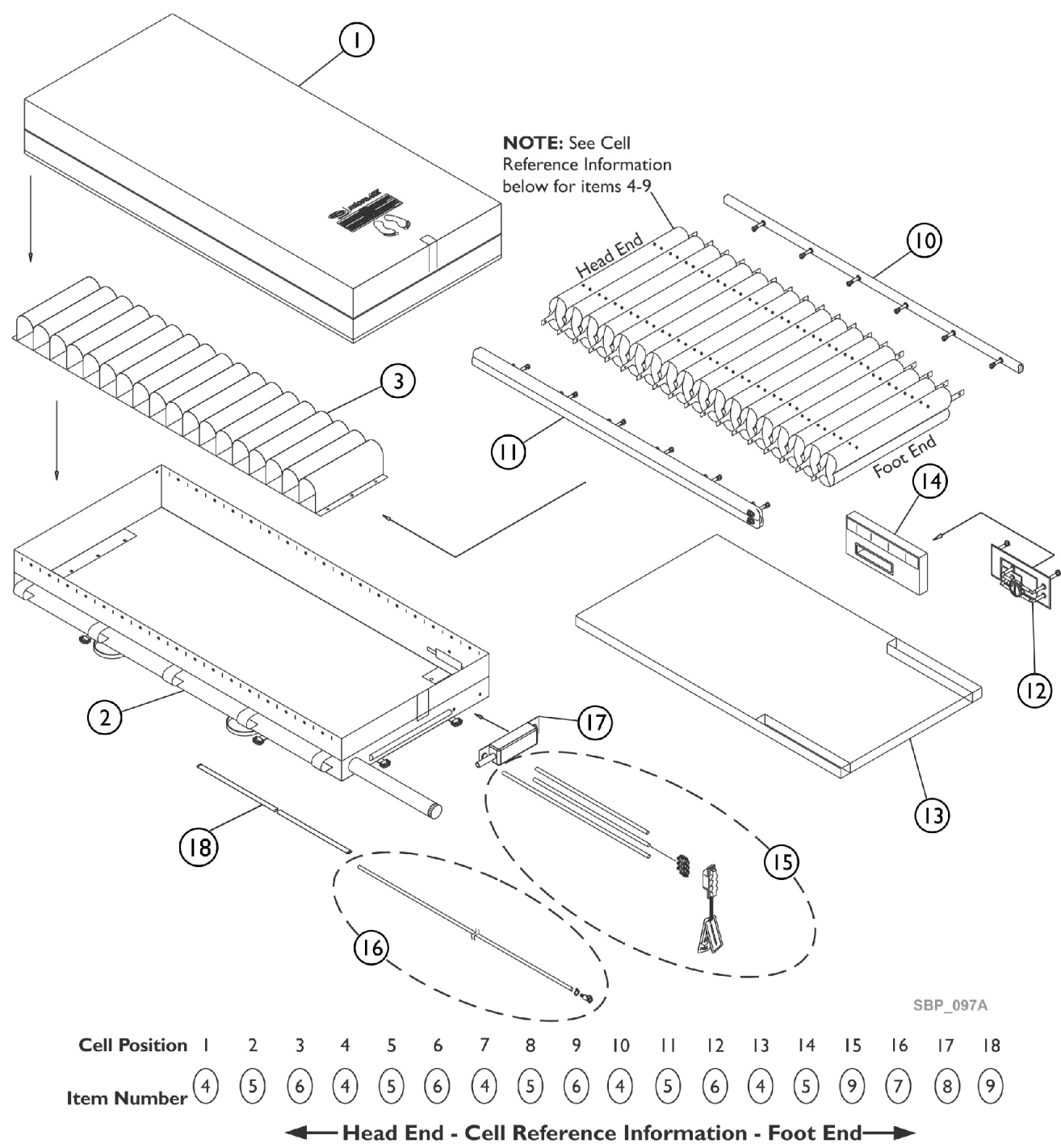


SBP_097A

MicroAIR 800M42 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124687	Top Cover	1	1
2		60124688	Bottom Cover	1	1
3		60124622	Belt Set	1	1
4		60124674	8" Vent Cell for 42" Mattress (Blue)	1	5
5		60124675	8" Vent Cell for 42" Mattress (Clear)	1	5
6		60124676	8" Vent Cell for 42" Mattress (Green)	1	4
7		60124677	8" Foot Vent Cell for 42" Mattress (Blue)	1	1
8		60124678	8" Foot Vent Cell for 42" Mattress (Clear)	1	1
9		60124679	8" Foot Vent Cell for 42" Mattress (Green)	1	2
10		60124633	Left Side Air Manifold	1	1
11		60124634	Right Side Air Manifold	1	1
12		60124635	Foot Side Rail For (14-1/2" x 7")	1	1
13		60124630	Foam Cover	1	1
14		60124631	Foot Foam Cover	1	1
15		60124620	Connecting Tube Set (Alternating Pipe)	1	1
16		60124621	Connecting Tube Set (Exhausting Pipe)	1	1
17		60124632	Vent/ Muffler Exhaust	1	1
18		60124623	Hose Sleeve	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124184	Exhaust O-Ring	1	1
	A	60124187	CPC Exhaust Connector (MA800)	1	1
	A	60124190	Heel Sense valve sticker	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124638	CPC Connector, 3/8" Male	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1
NOTE: A - Not Shown					

MicroAIR 800M4284 Mattress

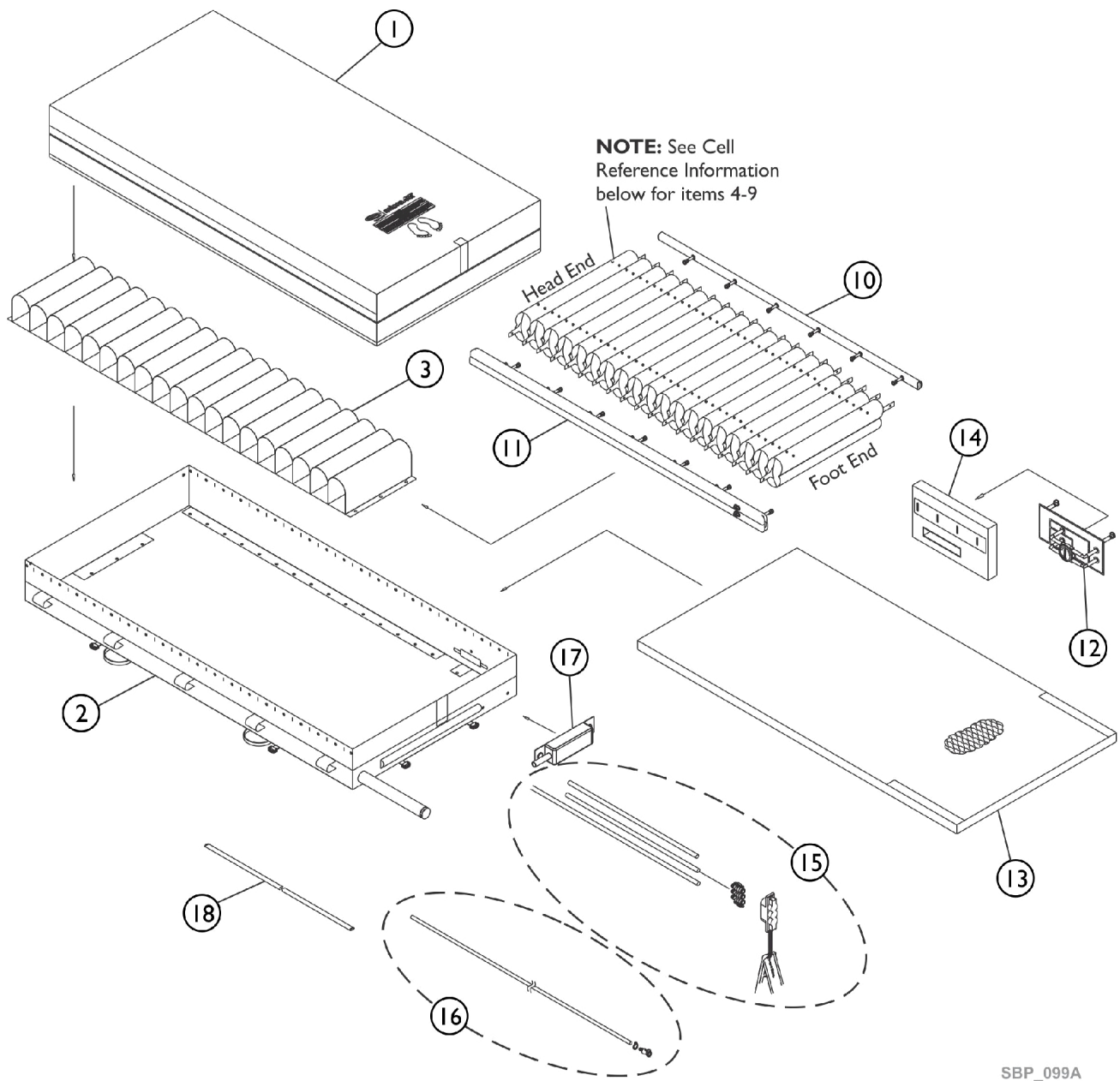

Cell Position	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19
Item Number	4	5	6	4	5	6	4	5	6	4	5	6	4	5	6	7	8	9	7

← Head End - Cell Reference Information - Foot End →


MicroAIR 800M4284 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124772	Top Cover	1	1
3		60124622	Belt Set	1	1
4		60124674	8" Vent Cell for 42" Mattress (Blue)	1	5
5		60124675	8" Vent Cell for 42" Mattress (Clear)	1	5
6		60124676	8" Vent Cell for 42" Mattress (Green)	1	5
7		60124677	8" Foot Vent Cell for 42" Mattress (Blue)	1	2
8		60124678	8" Foot Vent Cell for 42" Mattress (Clear)	1	1
9		60124679	8" Foot Vent Cell for 42" Mattress (Green)	1	1
10		60124749	Left Side Air Manifold	1	1
11		60124750	Right Side Air Manifold	1	1
12		60124635	Foot Side Rail For (14-1/2" x 7")	1	1
13		60124680	Foam Cover	1	1
14		60124631	Foot Foam Cover	1	1
15		60124671	Connecting Tube Set (Alternating Pipe Set)	1	1
16		60124621	Connecting Tube Set (Exhausting Pipe)	1	1
17		60124632	Vent/ Muffler Exhaust	1	1
18		60124623	Hose Sleeve	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124184	Exhaust O-Ring	1	1
	A	60124187	CPC Exhaust Connector (MA800)	1	1
	A	60124190	Heel Sense valve sticker	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124638	CPC Connector, 3/8" Male	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1

NOTE: A - Not Shown

Item #2 Bottom Cover is non-serviceable.


MicroAIR 800M48 Mattress

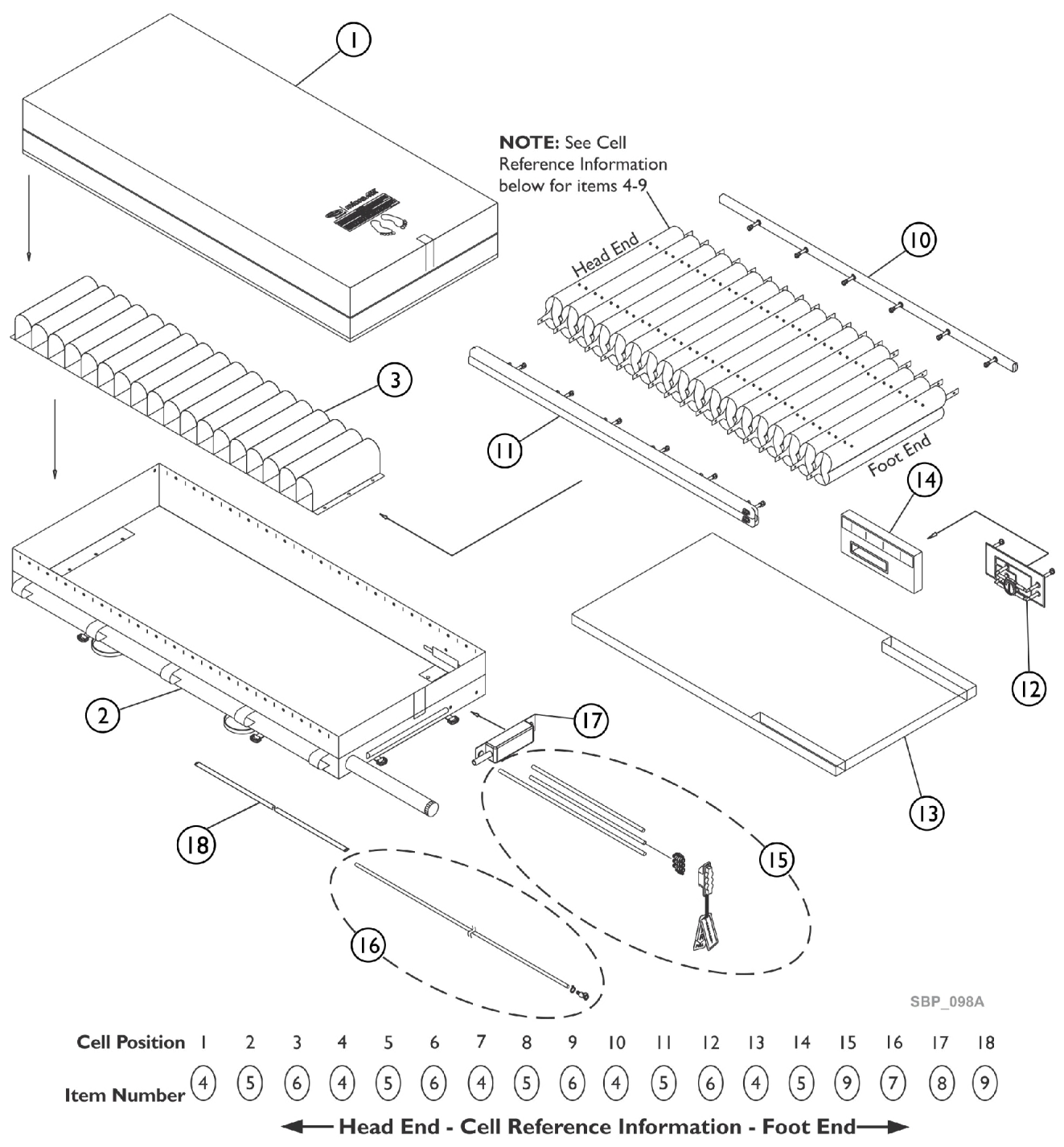

MicroAIR 800M48 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124753	Top Cover	1	1
2		60124754	Bottom Cover	1	1
3		60124622	Belt Set	1	1
4		60124693	8" Vent Cell for 48" Mattress (Blue)	1	5
5		60124694	8" Vent Cell for 48" Mattress (Clear)	1	5
6		60124695	8" Vent Cell for 48" Mattress (Green)	1	4
7		60124696	8" Foot Vent Cell for 48" Mattress (Blue)	1	1
8		60124697	8" Foot Vent Cell for 48" Mattress (Clear)	1	1
9		60124698	8" Foot Vent Cell for 48" Mattress (Green)	1	2
10		60124749	Left Side Air Manifold	1	1
11		60124750	Right Side Air Manifold	1	1
12		60124635	Foot Side Rail For (14-1/2" x 7")	1	1
13		60124746	Foam Cover	1	1
14		60124631	Foot Foam Cover	1	1
15		60124690	Connecting Tube Set (Alternating Pipe Set)	1	1
16		60124621	Connecting Tube Set (Exhausting Pipe)	1	1
17		60124632	Vent/ Muffler Exhaust	1	1
18		60124623	Hose Sleeve	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124184	Exhaust O-Ring	1	1
	A	60124187	CPC Exhaust Connector (MA800)	1	1
	A	60124190	Heel Sense valve sticker	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124638	CPC Connector, 3/8" Male	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1
NOTE: A - Not Shown					


MicroAIR 900 Mattress

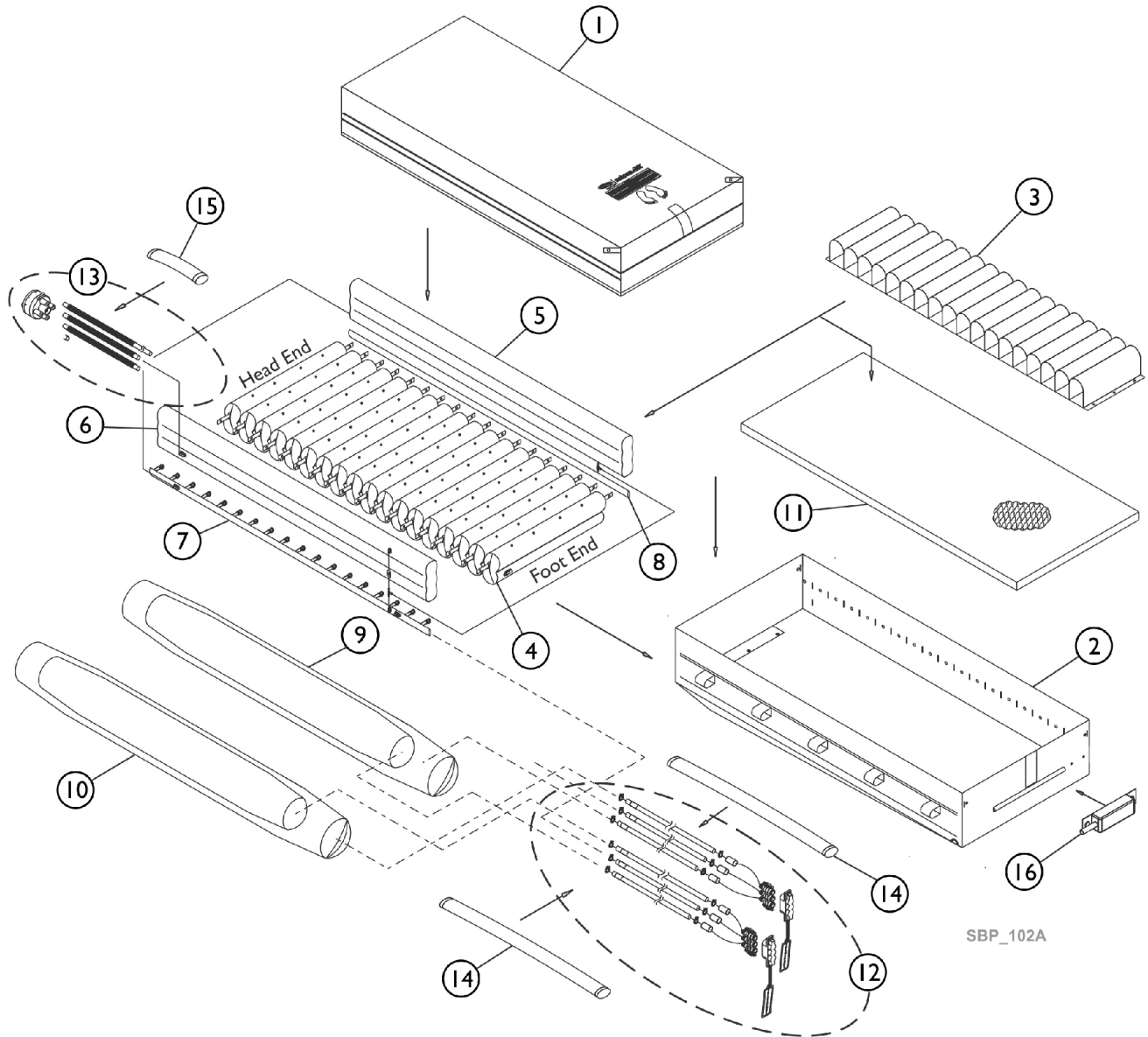

MicroAIR 900 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124653	Top Cover	1	1
2		60124654	Bottom Cover	1	1
3		60124218	MA900 36" & 42" Belt Set	1	1
4		60124652	MA900 36" Vent Cell	1	18
5		60124214	MA900 36" & 42" Left Side Bolster	1	1
6		60124215	MA900 36" & 42" Right Side Bolster	1	1
7		60124216	MA900 36" & 42" Right Side Rail	1	1
8		60124217	MA900 36" & 42" Left Side Rail	1	1
9		60124207	MA900 36" & 42" Left Turn Bladder	1	1
10		60124208	MA900 36" & 42" Right Turn Bladder	1	1
11		60124233	MA900 36" Foam Set	1	1
16		60124212	Mattress Muffler	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1
	A	60124213	Valve, One Way (SD / CRT)	1	1
NOTE: A - Not Shown					
Item #12 (Connecting Tube Set), #13 (CPR Set), #14 (Hose Sleeve, Fire Resistance CV 56" x 5") and #15 (Sleeve Cover) are not available to order					


MicroAIR 900M42 Mattress

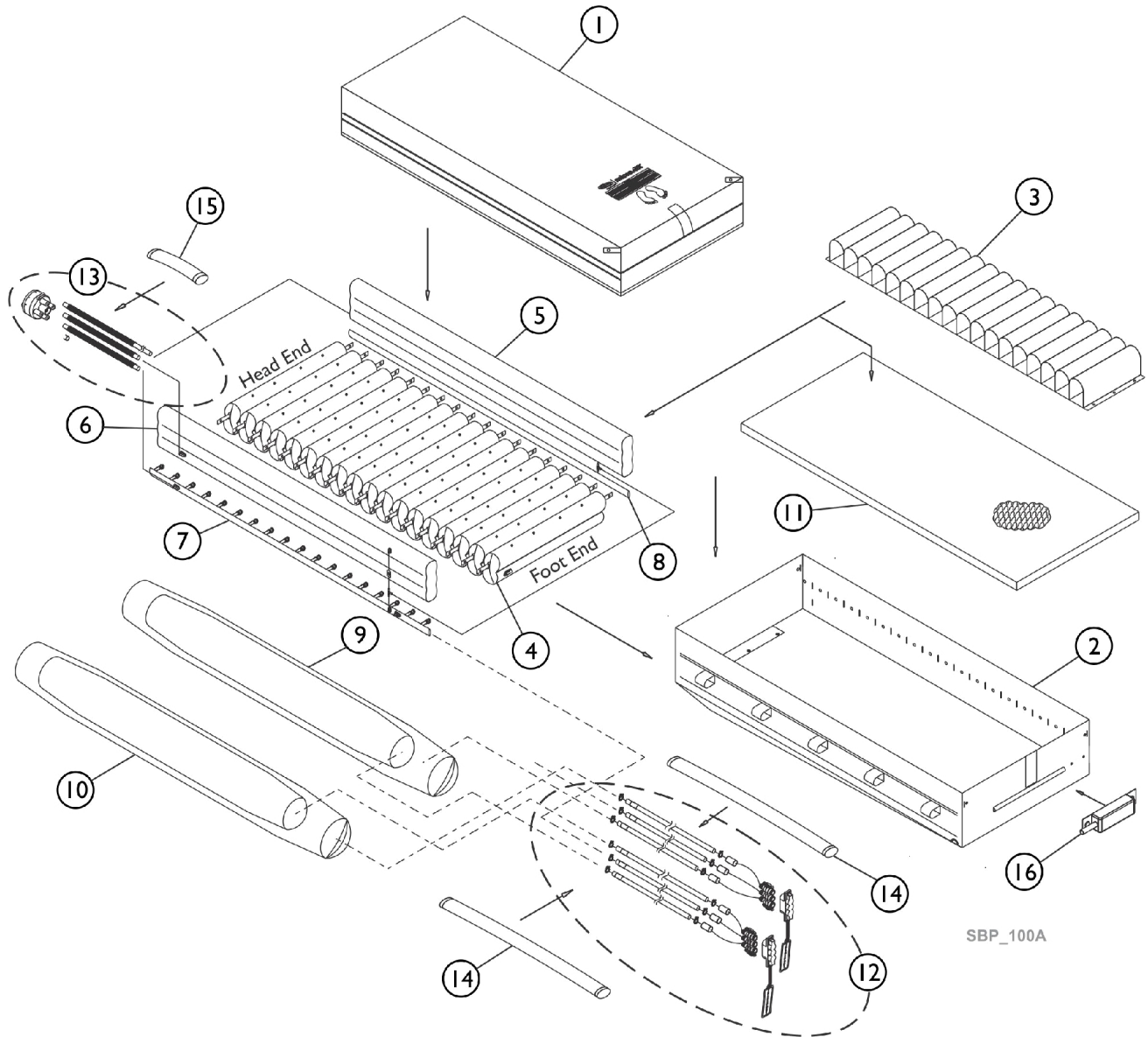

MicroAIR 900M42 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60126264	Top Cover	1	1
2		60124779	Bottom Cover	1	1
3		60124218	MA900 36" & 42" Belt Set	1	1
4		60128459	MA900 42" Cell	1	18
5		60124214	MA900 36" & 42" Left Side Bolster	1	1
6		60124215	MA900 36" & 42" Right Side Bolster	1	1
7		60124216	MA900 36" & 42" Right Side Rail	1	1
8		60124217	MA900 36" & 42" Left Side Rail	1	1
9		60124207	MA900 36" & 42" Left Turn Bladder	1	1
10		60124208	MA900 36" & 42" Right Turn Bladder	1	1
11		60124233	MA900 36" Foam Set	1	1
16		60124212	Mattress Muffler	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1
	A	60124213	Valve, One Way (SD / CRT)	1	1
NOTE: A - Not Shown					
Item #12 (Connecting Tube Set), #13 (CPR Set), #14 (Hose Sleeve, Fire Resistance CV) and #15 (Sleeve Cover) are not available to order					

MicroAIR 900M48 Mattress

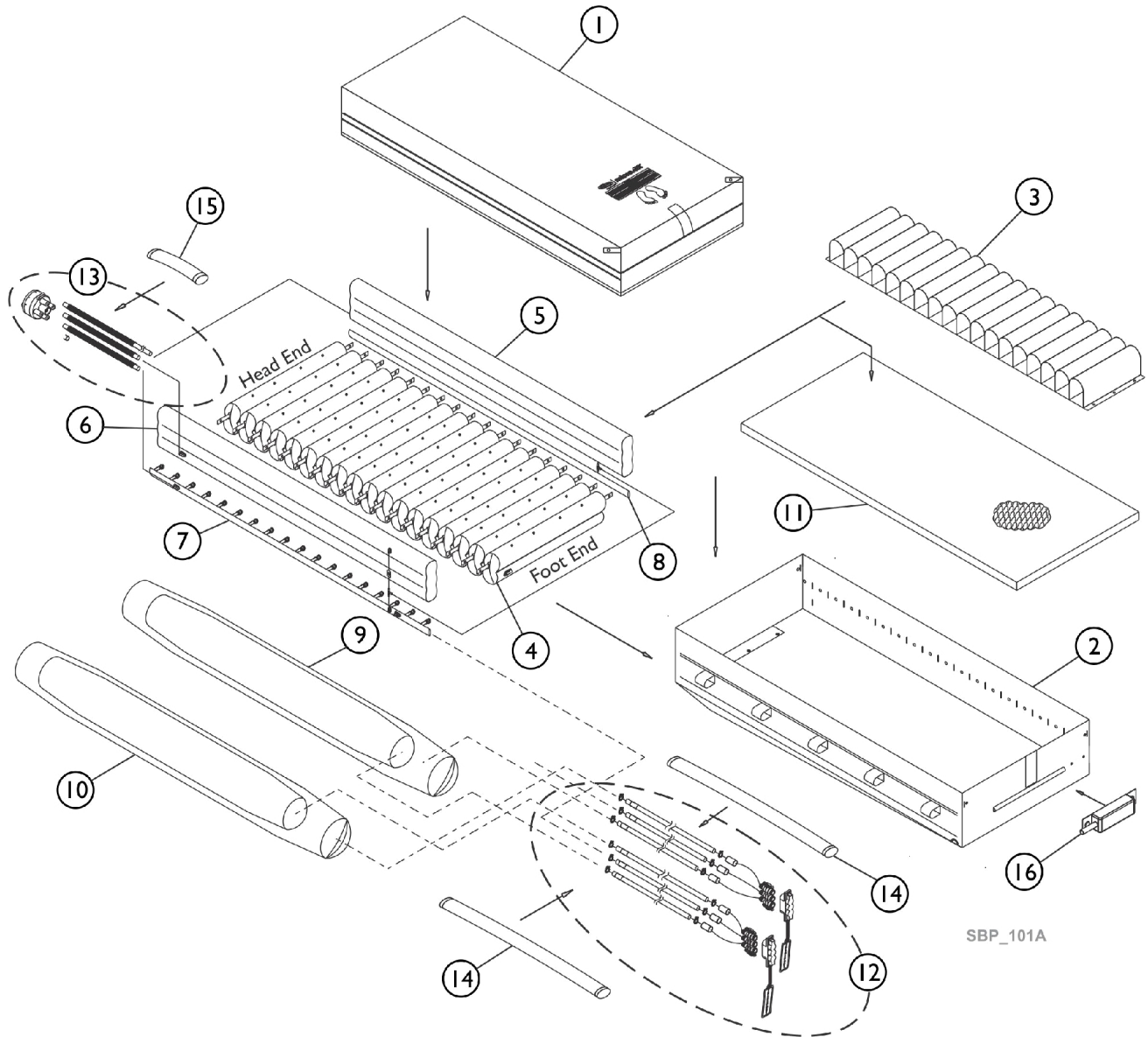


MicroAIR 900M48 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124774	Top Cover	1	1
2		60124775	Bottom Cover	1	1
4		60128458	MA900 48" Vent Cell	1	18
5		60124214	MA900 36" & 42" Left Side Bolster	1	1
6		60124215	MA900 36" & 42" Right Side Bolster	1	1
7		60124216	MA900 36" & 42" Right Side Rail	1	1
8		60124217	MA900 36" & 42" Left Side Rail	1	1
9,10		60124773	MA900 48" Left & Right Turn Bladder	1	2
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1
	A	60124213	Valve, One Way (SD / CRT)	1	1

NOTE: A - Not Shown

Item #3 (Belt Set), #11 (Mattress 900 48" Foam Set), #12 (Connecting Tube Set), #13 (CPR Set), #14 (Hose Sleeve, Fire Resistance CV 56" x 5") and #15 (Sleeve Cover) are not available to order

MicroAIR 1000 Mattress

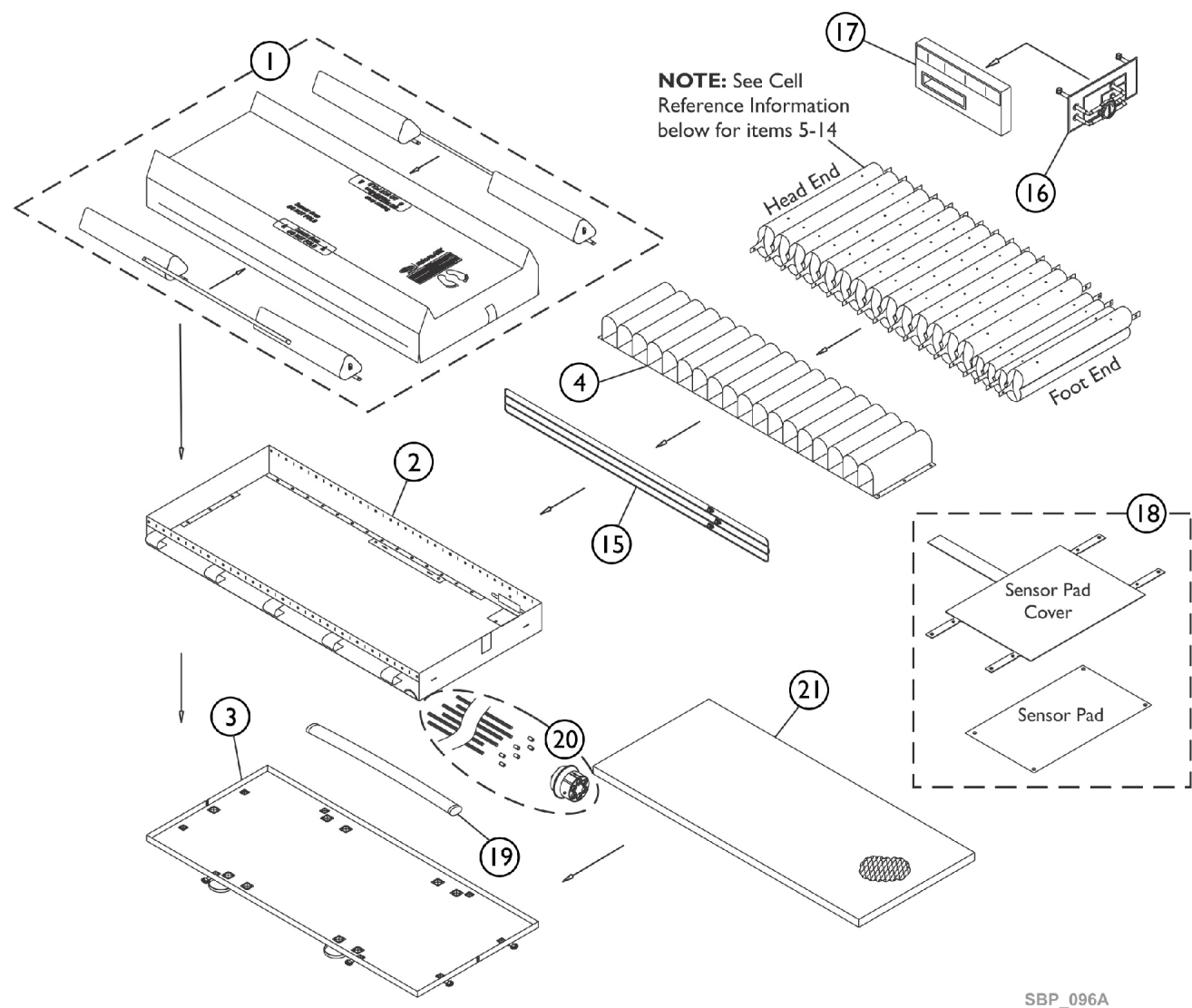

Cell Position	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18
Item Number	5	6	7	8	9	10	8	9	10	8	9	10	8	9	11	12	13	14
← Head End - Cell Reference Information - Foot End →																		

MicroAIR 1000 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124655	Top Cover w/ Right and Left Bolsters	1	1
2		60124656	Middle Cover	1	1
3		60124657	Bottom Cover	1	1
4		60124658	Belt Set	1	1
5,6,7		60124659	Head Vent Cell Set (3 Cells)	1	3
8		60124660	Vent Cell (Blue)	1	4
9		60124661	Vent Cell (White)	1	4
10		60124662	Vent Cell (Green)	1	3
11,12,13		60124663	Foot Cell Set (3 Cells)	1	3
14		60124664	Foot Cell (18th Single Cell)	1	1
18		60124667	Sensor Pad and Cover	1	1
Item #15 (Side Rail), #16 (Foot Side Rail 14-1/2" x 7"), #17 (Side Bolster Foam 17-3/4" x 8" x 2"), #19 (Hose Sleeve), #20 (Connecting Tube Set), and #21 (M1000 Foam Set 78" x 34" x 2") are not available to order					

Accessories, Miscellaneous

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60124263	MA800 36" / MA600 / MA500 Top Cover w/ Bolster	1	1
2	A	60124686	MA800 42" Top Cover w/ Bolster	1	1
3	A	60124752	MA800 48" Top Cover w/ Bolster	1	1
4	A,B	60124668	Quick Inflate Adapter (BA0001ENH) (MA600)	1	1
5	A,C	60124239	Quick Inflate Adapter (BA0001SAL) (MA500)	1	1
6	A,D	60124669	500 Watt; 4.0 Amp; 120 Volt Inflator/ Deflator w/o Adapter	1	1
NOTE: A - Not Shown B - Used for model MA600 C - Used for model MA500 D - Item #6 does not include the Quick Inflate Adapter. Must also order item #4 60124668 for model MA600 or item #5 60124239 for model MA500.					

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Literature Orders
800-333-6900

© 2021 Invacare Corporation, Elyria, Ohio.
All rights reserved. Republication,
duplication or modification in whole or in part
is prohibited without prior written permission
from Invacare. Trademarks are identified by
the symbols [™] and ®. All trademarks are
owned by or licensed to Invacare
corporation unless otherwise noted.
(Revised 7/29/21)

Form No. 19_057