

The Great Transport Chair

B30118, B30120

B10218, B10220

B30118LB, B30120LB

B30118LR, B30120LR

B10218LB, B10220LB

B10218LR, B10220LR



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalogue contains service parts for Form No. 19_111C. If you have any questions regarding this catalogue, please contact one of our Customer Service Representatives.

**For Customer Service Support Call:
1-800-668-5324 or 1-905-890-8300**
**For Faxed In Orders Call:
1-800-668-5478 or
1-905-890-5244**
**For Technical Service Call:
1-800-832-4707**
**Literature Can Be Found at:
www.invacare.ca or
1-800-668-5324**

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments
NOTE: = General information pertaining to that specific page
NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length
Washers Inside Diameter X Outside Diameter X Thickness
Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2019 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are registered trademarks of the Invacare Corporation. Information is subject to change without notification.

TABLE of CONTENTS

Description	Page Number
Back Components - Fold Down Back Release Tab and Hand Grip	11
Front Riggings and Adjustable Calf Strap	7
Upholstery - Back, Seat and Armpad	9
Wheel Locks	5
Wheels	3

Wheels




BLU-8CTR



BLU-12CTRBK



BLU-12CTR

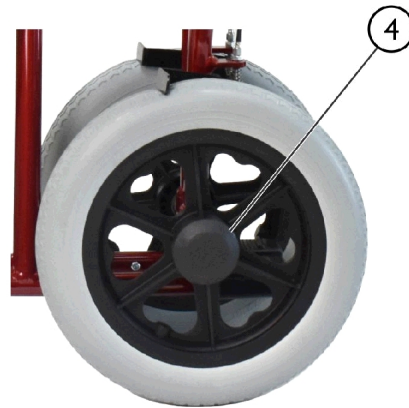

BLU-HC12W

SSM_047A

Wheels

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		BLU-8CTR	8" Urethane Front Wheel, Grey	1	2
2		BLU-12CTRBK	8" Urethane Front/ Rear Wheel, Black	1	4
3		BLU-12CTR	12" Urethane Rear Wheel, Grey	1	2
4		BLU-HC12W	Plastic Hub Cap 12" Rear Wheel	1	2

Wheel Locks




Left Side Shown

BLU-SIL




Right Side Shown

BLU-SIR

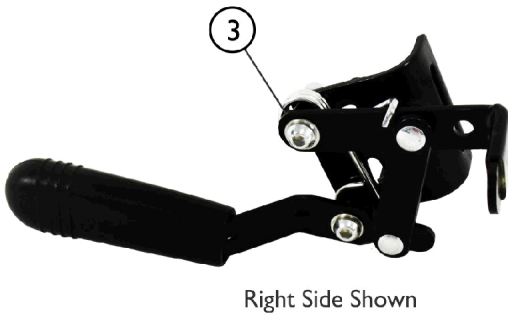

Right Side Shown

BLU-S10RBK
BLU-S10LBK

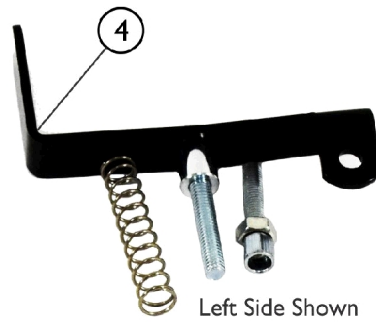

Left Side Shown




BLU-K12L
BLU-K12R

SSM_048A

Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		BLU-S1L	Attendant Brake Handle Big Wheel Chairs, Left	1	1
2		BLU-S1R	Attendant Brake Handle Big Wheel Chairs, Right	1	1
3		BLU-S10RBK	Wheel lock Great Aluminum Chairs, Right	1	1
3		BLU-S10LBK	Wheel lock Great Aluminum Chairs, Left	1	1
4		BLU-K12L	Brake Shoe Big Wheel Transport Chairs, Left	1	1
4		BLU-K12R	Brake Shoe Big Wheel Transport Chairs, Right	1	1

Front Riggings and Adjustable Calf Strap

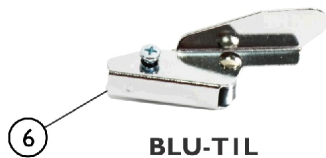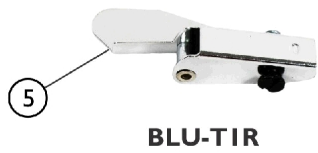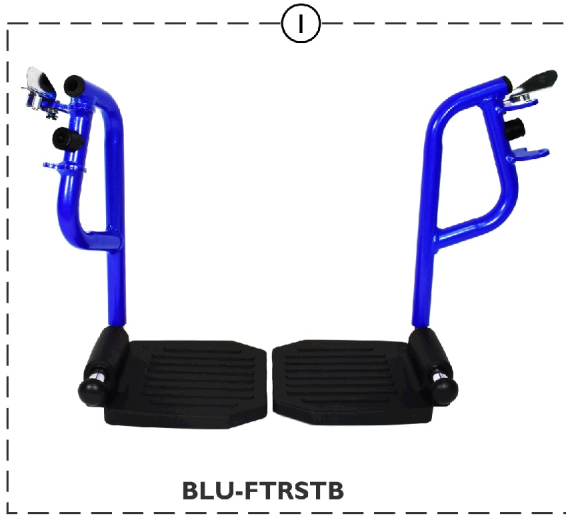


SSM_049A

Front Riggings and Adjustable Calf Strap

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		BLU-FTRSTB	Swingaway Legs composite foot, Electric Blue	1	1
2		BLU-FTRSTS	Swingaway Legs composite foot, Silver Vein	1	1
3		BLU-FTRSTR	Swingaway Legs composite foot, Electric Red	1	1
4		BLU-CALF	Calf Strap Transport Chair Adjustable	1	1
5		BLU-T1R	Hanger Lock Release, Right	1	1
6		BLU-T1L	Hanger Lock Release, Left	1	1
7		BLU-M5	Composite Footplate Left/Right	1	1

Upholstery

Back, Seat and Armpad

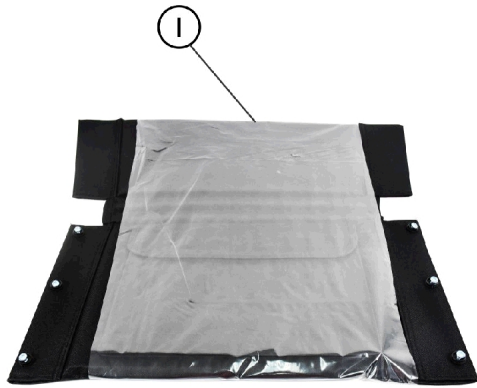

BLU-BACK18



BLU-SEAT18



BLU-BACK20



BLU-SEAT20



BLU-ARMPAD

SSM_050A

Upholstery Back, Seat and Armpad


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		BLU-BACK18	Transport Back Upholstery 18"	1	1
2		BLU-SEAT18	Transport Seat Upholstery 18"	1	1
3		BLU-BACK20	Transport Back Upholstery 20"	1	1
4		BLU-SEAT20	Transport Seat Upholstery 20"	1	1
5		BLU-ARMPAD	Armpad Univ. blue Transport Chair	1	2

Back Components

Fold Down Back Release Tab and Hand Grip



BLU-N6

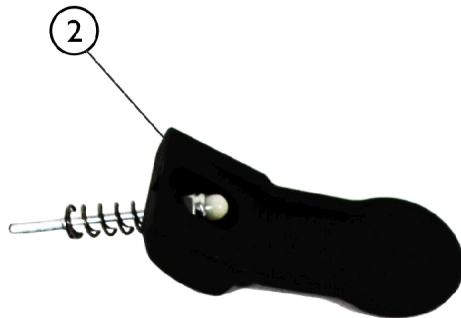

BLU-T2

SSM_051A

Back Components

Fold Down Back Release Tab and Hand Grip

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		BLU-N6	Hand Grip Blue Transport Chairs	1	2
2		BLU-T2	Fold Down Back Release tab	1	2

Invacare Canada L.P.

www.invacare.ca



Yes, you can.

570 Matheson Blvd E;8
Mississauga, ON
L4Z4G4
905-890-8300

Customer Service
800-668-5324
For Faxed Orders Call
800-668-5478

Technical Support
800-832-4707

© 2019 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ©. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. (Revised 9/13/19)

Form No. 19_111C