
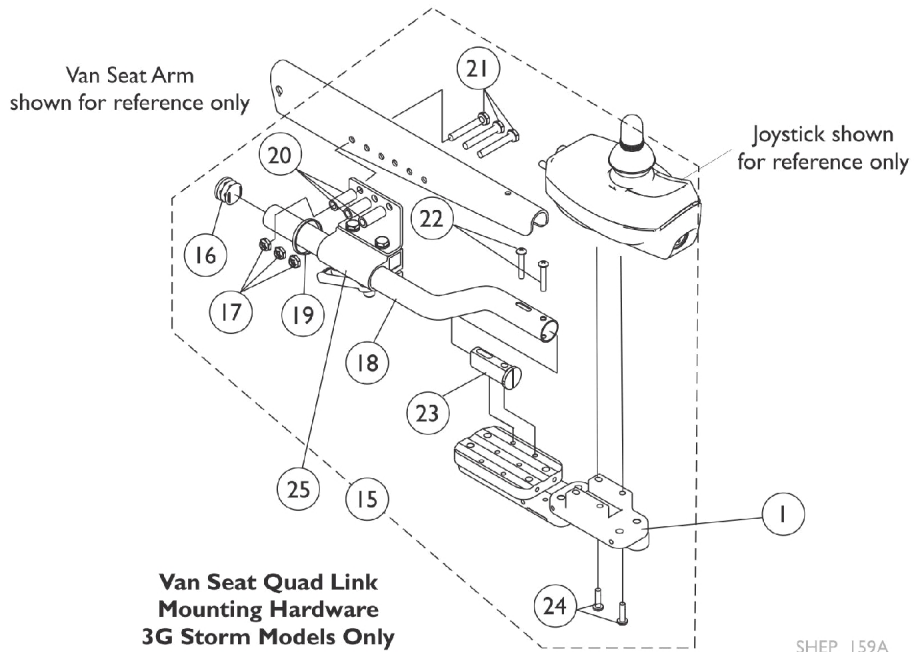


Quad Link (QL) - Van Seats with SPJ+ Joysticks

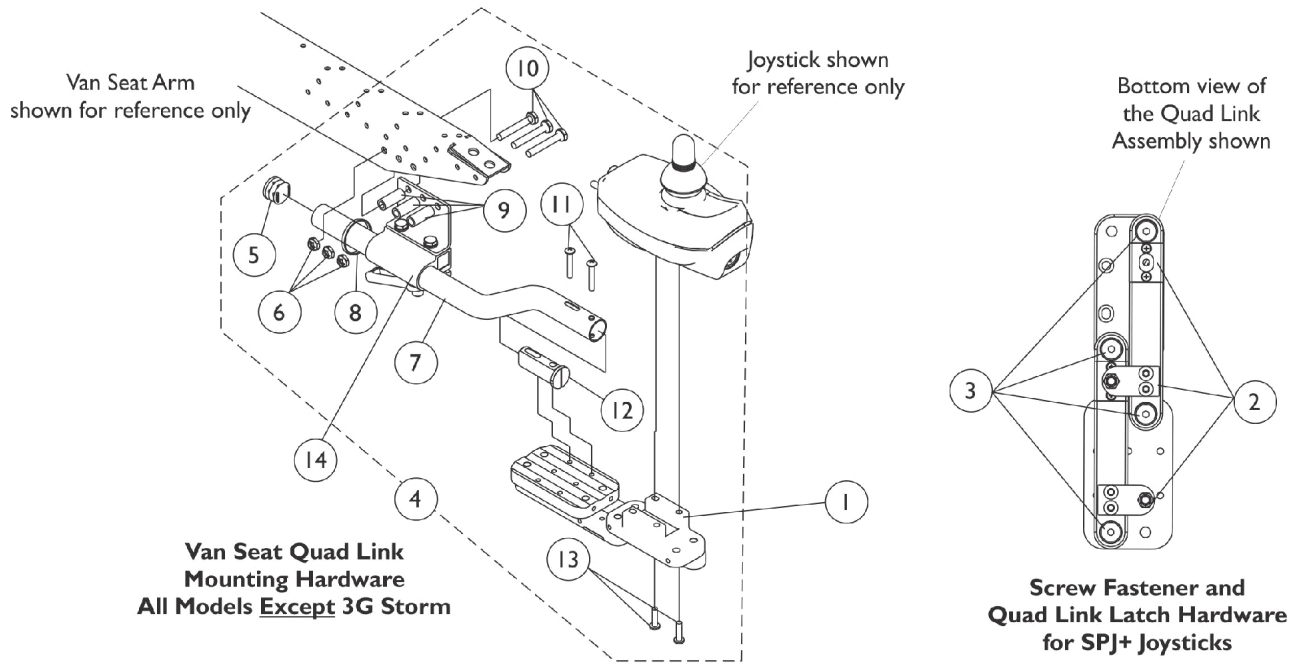

SHEP_159A

Quad Link (QL) - Van Seats with SPJ+ Joysticks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1140898	Quad Link (QL) with Instruction Sheet - Right	1	1
1		1140899	Quad Link (QL) with Instruction Sheet - Left	1	1
	E	1134844	Instruction Sheet, Quad Link	1	1
2	A	1140425	Kit, Quad Link Latch Hardware	1	1
3	B	1140424-NLA	NLA - Kit, Quad Link Screw Fastener Hardware	1	1
4	C	1143042	Kit, Quad Link Mounting Hardware - Van Seat	1	1
5			Plug Button, Black (7/8")	1	1
6			Locknut (1/4-20)	1	3
7			Tube, Joystick Offset (2")	1	1
8			Tie Wrap (11-1/2" L)	1	1
9			Spacer (1/4 x 3/8 x 1")	1	3
10			Screw, Hex Head (1/4-20 x 1-5/8")	1	3
11			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
12			Spacer, Joystick Tube, Black	1	1
13			Screw, Phillips Pan Head w/ Patch (#10-32 x 3/4")	1	2
13		1105414	Screw, Phillips Pan Head w/ Patch (#10-32 x 3/4")	1	2
14		1139681	Clamp Assembly, Fixed Height, Captain/ Van Seat	1	1
15	D	1143041-NLA	NLA - Kit, Quad Link Mounting Hardware - Van Seat	1	1
16			Plug Button, Black (7/8")	1	1
17			Locknut (1/4-20)	1	3
18			Tube, Joystick Offset (2")	1	1
19			Tie Wrap (11-1/2" L)	1	1
20			NLA - Spacer (1/4 x 7/16 x 5/8")	1	3
21			Screw, Hex Head (1/4-20 x 1-1/4")	1	3
22			Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
23			Spacer, Joystick Tube, Black	1	1
24			Screw, Phillips Pan Head w/ Patch (#10-32 x 3/4")	1	2
24		1105414	Screw, Phillips Pan Head w/ Patch (#10-32 x 3/4")	1	2
25		1105049	Clamp Assembly, Fixed Height, Captain/ Van Seat	1	1

NOTE: A - Latch Kit includes 4 ea. Button Head Cap Screws (10-32 x 3/8"), 2 ea. Latch Spring Assembly, 4 ea. Phillips Flat C'Sunk Head Screws, 2 ea. Spring Catch Plates, 2 ea. Socket Head Set Screws (10-32 x 3/8")
B - Screw Fastener Kit includes 4 ea. Socket Head Shoulder Screws (3/8 x 3/8"), 2 ea. Socket Head Cap Screws (1/4-20 x 3/8"), 2 ea. Socket Head Set Screws (1/4-20 x 1/2"), 4 ea. Socket Head Set Screws (10-32 x 1/8")
C - Includes items 5-13.
D - Includes items 16-24.
E - Not Shown