


3" Anti-Tip Wheel Assembly

All Models

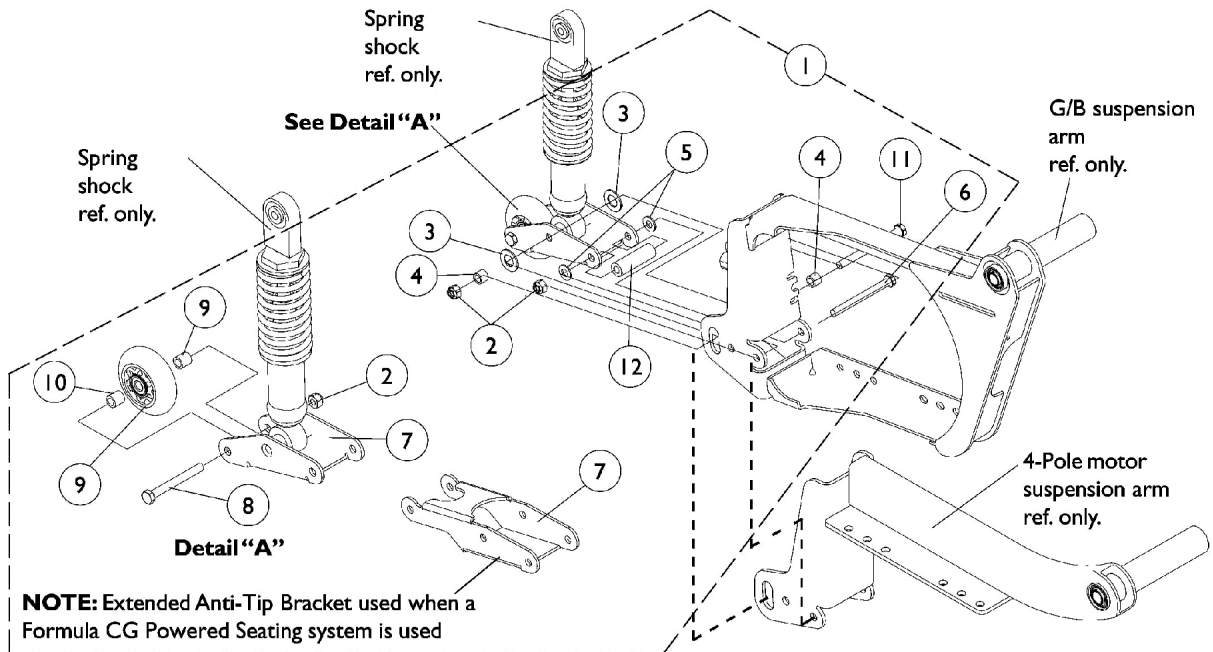

3" Anti-Tip Wheel Assembly All Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1104811	Anti-Tip Wheel Assembly (Pair)	2	1
2			Locknut (5/16-18)	1	6
3			NLA - Washer, Nylon (35/64 x 1 x 1/16")	1	4
4			Spacer (5/16 x 1/2 x 3/8")	1	4
5			Washer, Nylon (11/32 x 11/16 x 1/16")	1	4
6			NLA - Screw, Hex Head (5/16-18 x 3-1/4")	1	2
7			NLA - Bracket, Anti-Tipper, Black	1	2
8			Screw, Hex Head (5/16-18 x 2-3/4")	1	2
9			Wheel Assembly (3" x 1")	1	2
10			Spacer (5/16 x 1/2 x 1/2")	1	4
11			NLA - Screw, Hex Head (5/16-18 x 3-1/2)	1	2
12			Spacer (5/16 x 5/8 x 2")	1	2
1	A	1143792	Kit, Anti-Tip Wheel Assembly (Pair) (For Formula CG Powered Seating Only)	2	1
2			Locknut (5/16-18)	1	6
3			NLA - Washer, Nylon (35/64 x 1 x 1/16")	1	4
4			Spacer (5/16 x 1/2 x 3/8")	1	4
5			Washer, Nylon (11/32 x 11/16 x 1/16")	1	4
6			NLA - Screw, Hex Head (5/16-18 x 3-1/4")	1	2
7			Bracket, Anti-Tipper Extension, Black	1	2
8			Screw, Hex Head (5/16-18 x 2-3/4")	1	2
9			Wheel Assembly (3" x 1")	1	2
10			Spacer (5/16 x 1/2 x 1/2")	1	4
11			NLA - Screw, Hex Head (5/16-18 x 3-1/2)	1	2
12			Spacer (5/16 x 5/8 x 2")	1	2
9		1101757	Wheel Assembly (3" x 1")	1	2

NOTE: A - Includes items 2-12 (Pair)