
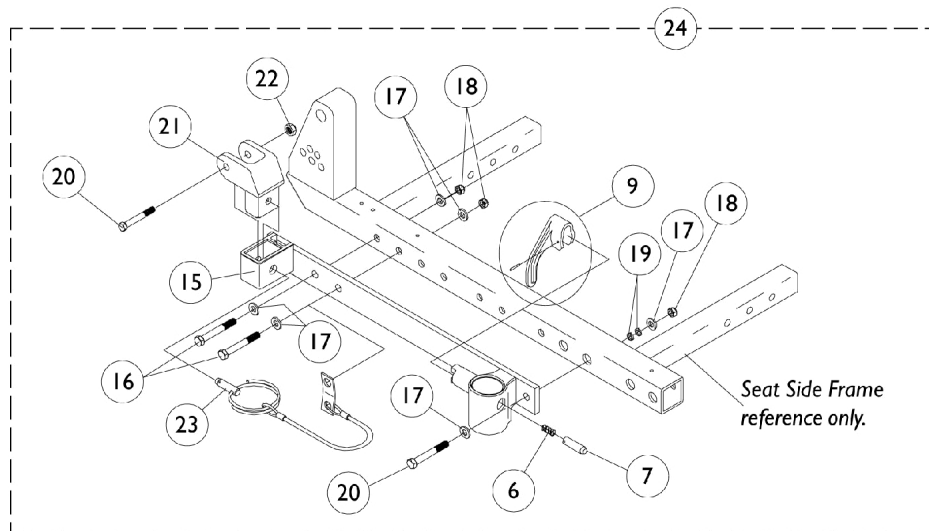


Armrest Assembly

Flip Back Arms After 10/14/98

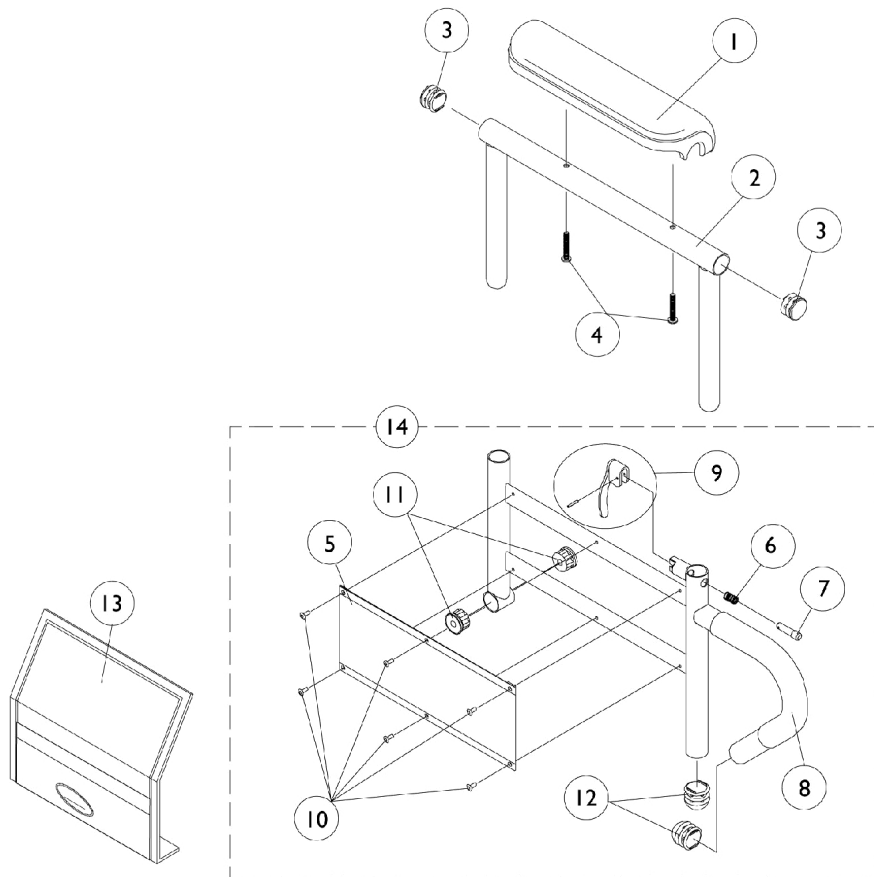

Armrest Assembly

Flip Back Arms After 10/14/98

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
1	B	1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
1	A,C	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
1	C	1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
2		1071253	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
2		1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
3		1027095	Plug Button, Black (7/8")	1	2
4		1055949	Package, Screw, Phillips Pan Head (#10-32 x 1-3/8")	2	1
4	D	1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
5		1064130-NLA	NLA - Guard, Clothing	1	1
6		47471X074	Spring, Arm Lock	1	1
7		1028594	Pin, Arm Lock, Black	1	1
8		1032244	Grip, Foam Hand (9" L)	1	1
9		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
10		1028645	Rivet	1	6
11		1065936	Plug, Pivot	1	2
12		1067734	Plug Button, Black (5/8")	1	2
13		1035065	Cover, Clothing Guard w/ Pocket, FXDESK & ADJFULL/ADJDESK	1	1
14		1068416	Arm Base Assembly, Black - Right (16-19" D)	1	1
14		1068417	Arm Base Assembly, Black - Left (16-19" D)	1	1
14		1068418	Arm Base Assembly, Black - Right (20" and Deeper)	1	1
14		1068419	Arm Base Assembly, Black - Left (20" and Deeper)	1	1
14		1067761	Arm Base Assembly, 1" Lower, Black - Right (16-19" D)	1	1
14		1067762	Arm Base Assembly, 1" Lower, Black - Left (16-19" D)	1	1
14		1067755-NLA	NLA - Arm Base Assembly, 1" Lower, Black - Right (20" and Deeper)	1	1
14		1067756-NLA	NLA - Arm Base Assembly, 1" Lower, Black - Left (20" and Deeper)	1	1
15		1078053-NLA	NLA - Receptacle, Flip Back Arm, Black - Right	1	1
15		1078052-NLA	NLA - Receptacle, Flip Back Arm, Black - Left	1	1
16		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	3
17		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	6
18		7930333	Locknut (1/4-20)	25	3
19		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	2
20		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	1

Armrest Assembly Flip Back Arms After 10/14/98

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
21		1068025	Clevis, Pivot	1	1
22		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
23		1068039	Pin, Pull	1	1
24	E	1081427-NLA	NLA - Hardware, Armrest Mounting - Right	1	1
24	E	1081428-NLA	NLA - Hardware, Armrest Mounting - Left	1	1
<p>NOTE: A - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs with screws for chairs serviced before 1/3/02.</p> <p>B - Full Length</p> <p>C - Desk Length</p> <p>D - For Black Urethane Waterfall Arm Pads Only</p> <p>E - Includes Items 6, 7, 9 and 15-23</p>					