


Shroud - Bottom

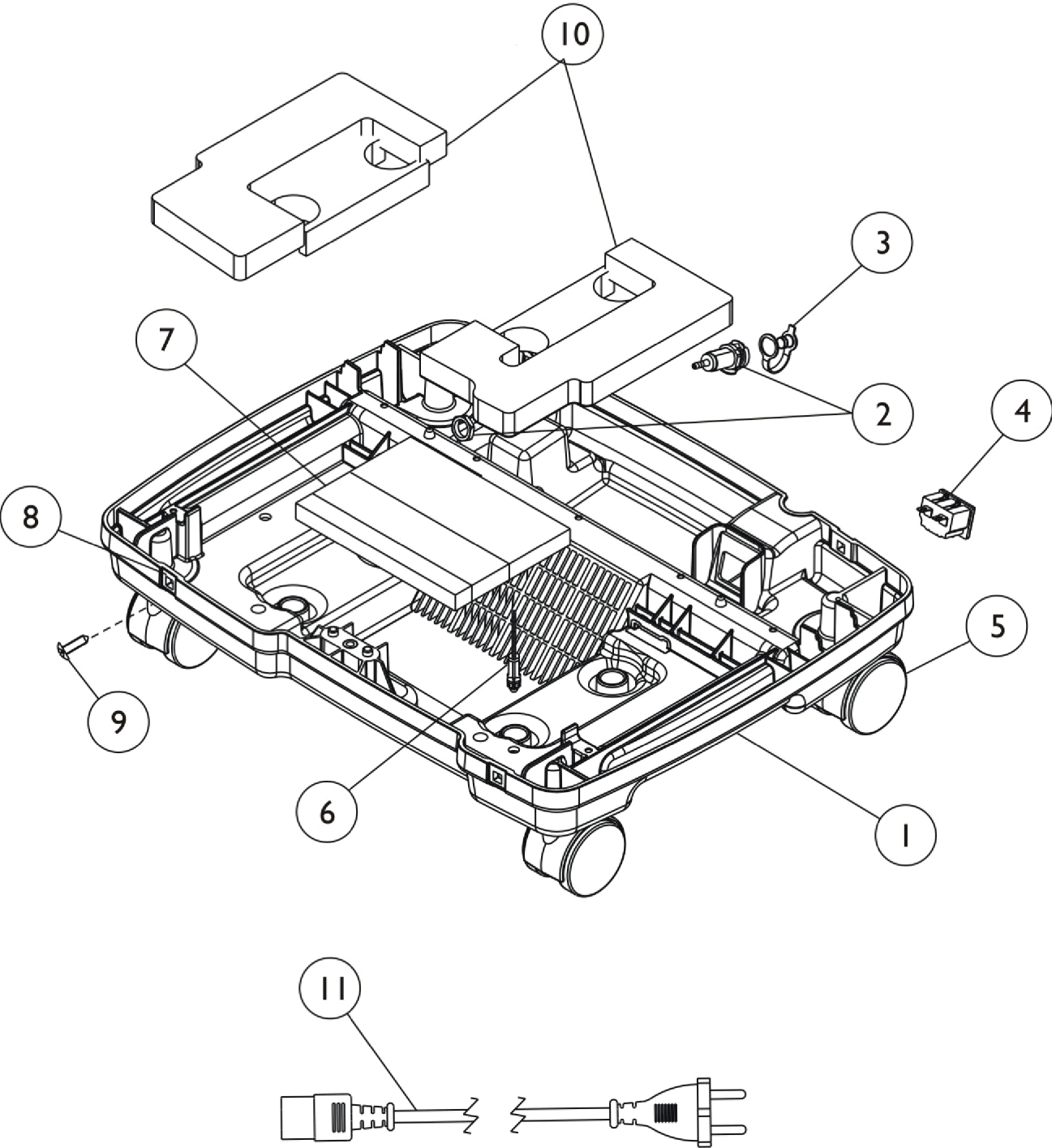

Shroud - Bottom

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1108804-NLA	NLA - Base, Compressor, Maroon	1	1
1	C	1134174-NLA	NLA - Tech Only - Base, Compressor, Black	1	1
2		1104151	Fitting, Hose Barb with Nut (1/8")	1	1
3		1108211	Cap, End, Dust, Green	1	1
4		1105515-NLA	NLA - Tech Only - Module, Inlet (15A/ 250 VAC)	1	1
5		1110426	Caster Assembly (2")	1	4
6		2000576	Stand-Off, Twist-Lok (1/2")	1	1
7		1137257-NLA	NLA - Tech Only - Foam, Base - AWQ	1	1
8		2000360	Grommet, Plastic	1	4
9		2000508	Screw, Phillips Truss Head Self Tap (#8-32 x 5/8")	1	1
10	B	1119595-NLA	NLA - Tech Only - Kit, Foam Base	1	1
11	A	1131308-NLA	NLA - Tech Only - Cord, Power, Continental Europe	1	1
11	A	1062794-NLA	NLA - Cord, Power - Argentina, Australia, Peoples Republic of China and New Zealand	1	1
11	A	1062793-NLA	NLA - Cord, Power - Hong Kong, Singapore and Vietnam	1	1
11	A	1071398	Cord, Power - Israel	1	1
11	A	1102634	Cord, Power - India	1	1
NOTE: A - Select Line Cord desired. B - Kit includes Right Inner, Right Outer and Left Outer Base Foam. C - Color has changed to black as of 5/10/07.					